

GCX Mounting Assembly Installation Guide

Philips MP60/70 IntelliVue – Roll Stand Mount for Flexible Module Server (FMS)

The purpose of this guide is to describe mounting the FMS on the Roll Stand Post.

Parts Reference

The following parts and hardware are provided for this procedure:

Item #	Description	Qty
1	Clamping Block with Mounting Adapter	1
2	M4 x 12 mm Pan Head Machine Screw (PHMS)	2
3	#10-32 x 9/16" PHMS	2



Clamping Block with Mounting Adapter



Tools Required: Phillips screwdriver.

Mounting the Flexible Module Server

1. Fasten Mounting Adapter to two (2) mounting holes on rear of FMS with two (2) M4 x 12 mm PHMS.

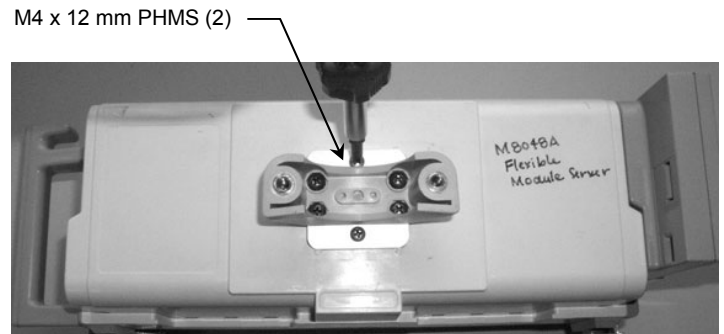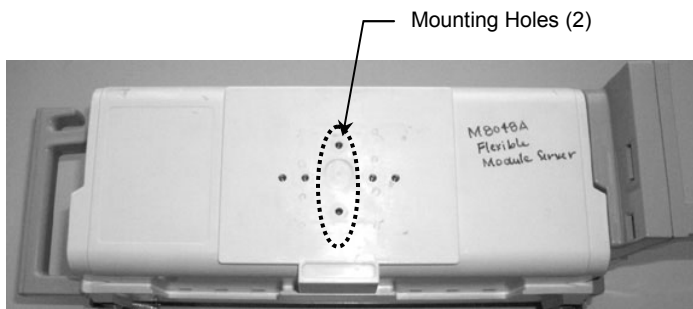

2. Fasten the FMS to the Post with the rear half of the Clamping Block and two (2) #10-32 x 9/16" PHMS.

