

## GCX Mounting Assembly Operation/Installation Manual Dräger Model Fabius CE<sup>®</sup> with Siemens 6002/7000/9000XL Patient Monitors



**Patient Monitor – Top Shelf / Power Supply on Right Side**

The purpose of these instructions is to:

1. Describe attachment of GCX mounting hardware to the Fabius anesthesia machine.
2. Describe attachment of the Siemens monitor to the GCX mounting hardware.



**WARNING:** USE OF MOUNTING HARDWARE AND MONITORING COMPONENTS OTHER THAN THOSE DESCRIBED IN THIS DOCUMENT MAY RESULT IN SERIOUS INJURY DUE TO TIPPING OF THE ANESTHESIA MACHINE.

**THIS FABIUS CE CONFIGURATION REQUIRES A COUNTERWEIGHT (DRAGER P/N: 8604076).**

**Installer:** When installation is completed, provide these instructions to the end user for future reference.


### INSTALLATION

#### RS Top Shelf Pan & Low Profile Tilt Swivel Mount

Install RS Housing Top Shelf Pan and Low Profile Tilt Swivel mount according to instructions included with those assemblies.


#### Slide Pick & Go Base into Mounting Plate

- 1) Align the mounting adapter located at the bottom of the Pick & Go base with the edge of Mounting Plate. Slide the Pick & Go base into the plate until it stops at the Spring Plunger (located below the Mounting Plate). Pull the Spring Plunger and continue to slide the base until the mounting adapter engages the Spring Plunger. Tighten the (4) nylon thumb screws on the back of the Mounting Plate.

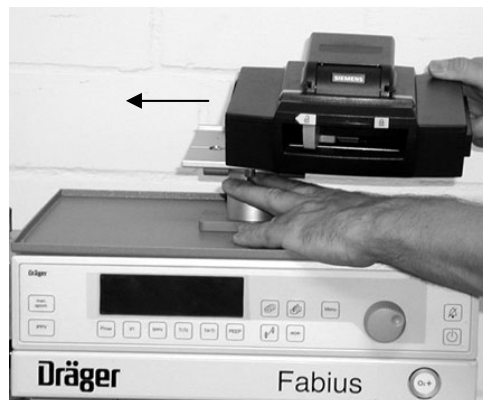

## Install Power Supply Mounting Adapter to Siemens Power Supply Mount (Siemens P/N: 4720095)

- 1) Install according to the instructions included with the adapter.

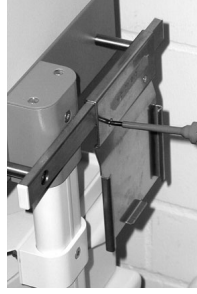

## Insert Power Supply


- 1) Align the Slide of the mount with edge of the Channel insert into channel.



## INSTRUCTIONS FOR USE – Low Profile Tilt Swivel Mount


### Swivel

- 1) To rotate, simply push or pull the corners of the device.
- 2) To adjust the tension of the swivel action, tighten or loosen the swivel adjustment screw.

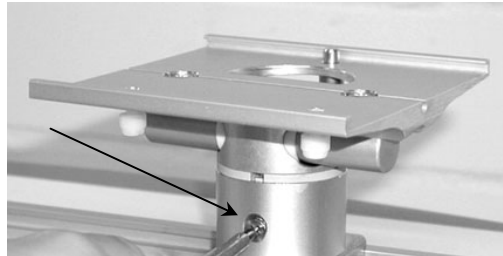

### Tilt

- 1) Grasp the top and bottom of the device and tilt to the desired angle.
- 2) Adjust the overall tilting tension by evenly tightening or loosening the #8-32 X 1/2" SHCS - (Tension Adjustment Screw) with the hex key provided.

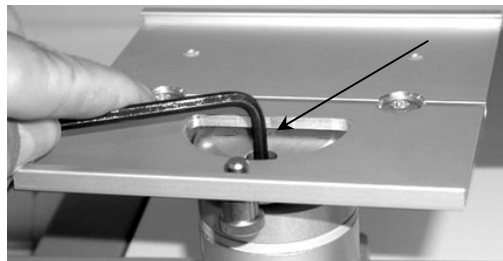

## Cleaning

- 1) The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
- 2) The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
- 3) Steel wool or other abrasive material should never be used.
- 4) Damage caused by the use of unapproved substances or processes will not be warranted. To verify compatibility, it is recommended that any cleaning solution be tested on a small area of the arm that is not visible.
- 5) Never submerge the mounting arm. Wipe any cleaning agents off of the arm immediately, using a water-dampened cloth. Dry the arm thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions supplied with those products.

## PARTS LIST

GCX P/N	Siemens P/N	Dräger P/N	DESCRIPTION
DR-0021-07	4720095	MM15690	RS Top Shelf Pan w/ tilt swivel
DR-0025-81		6870777	Siemens Power Supply Holster
		6870776	Vamos – RSHousing Mounting Kit w/10" Arm
		6870841	Vamos – Adaptor Plate
			Vamos – Power Supply Holster, rail mounted