

GCX Mounting Assembly Installation Guide


M Series Utility Hook



The purpose of these instructions is to:

1. Describe attachment of Utility Hook to M Series Arm.
2. Describe adjustment of Utility Hook.

1. Align the screw holes in the Utility Hook with the threaded holes on the Swivel Fitting, located on the underside of the M Series Arm. Using the supplied 5/32 (4 mm) hex key, secure the #10-32 x 5/16" SHCS (3).

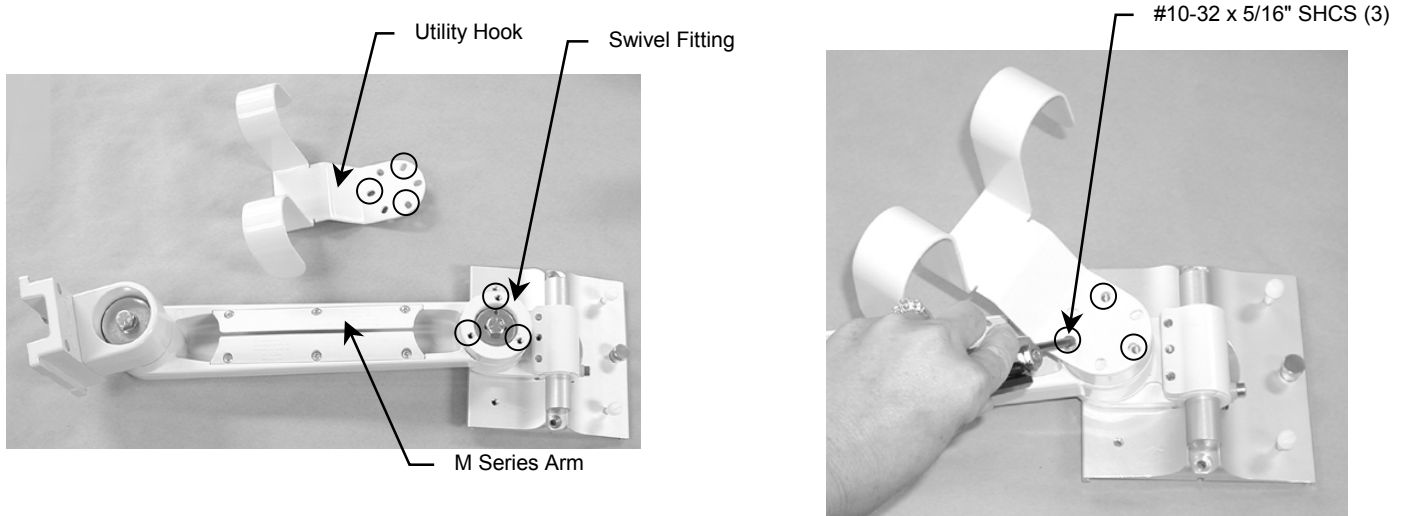

2. Adjust the location of the Utility Hook by grasping the hook and rotating to the desired position.

