

**GCX Mounting Assembly Installation Guide**

## Utility Basket Hanger for M-Series or VHM Arm

The purpose of this guide is to describe attachment of Hanger to Basket and Arm.

**Parts Reference**

The following parts and hardware are included with this installation kit (photos of parts – hardware not shown):

Item #	Description	Qty
1	Basket Hanger	1
2	#10-32 x 3/8" Pan Head Machine Screw (PHMS)	4
3	#10-32 x 3/8" Socket Head Cap Screw (SHCS)	3
4	5/32" Hex Key	1

Basket Hanger



**Tools Required:** Phillips screwdriver, 5/32" hex key (provided).

**Installation Note:** The Utility Basket is equipped with a Wall-Mounting Slide that must be removed prior to attachment of the Hanger.

**Attaching Hanger to Utility Basket**

1. Remove the Slide from the Basket by unscrewing four (4) pan head screws (below left).
2. Align four (4) threaded mounting holes in Hanger with mounting holes in Basket. Fasten Hanger to Basket with four (4) #10-32 x 3/8" PHMS (below center).
3. Using the 5/32" hex key provided, fasten Hanger to Swivel Fitting (M-Series or VHM Arm) with three (3) #10-32 x 3/8" SHCS (below right). **Installation Note:** Attachment is shown on M-Series Arm. Procedure for VHM Arm is identical.

