

GCX Mounting Assembly Installation Guide

Pedestal Base Mount for Philips Intellivue MP20/30/40/50/60/70 – Monitor Only

The purpose of this guide is to:

1. Describe procedure for attaching the table top mount to Base Plate.
2. Describe procedure for mounting Base Plate on counter top.
3. Describe procedure for applying rubber bumpers to Base Plate.
4. Describe procedure for covering unused holes in Base Plate.


Installation Note

This mounting assembly may be mounted securely to a rigid surface (counter top), or used for other purposes not requiring permanent attachment. Procedures for both applications are covered in this installation guide.

Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts; hardware not shown):

| Item # | Description | Qty |
|--------|--|-----|
| 1 | Pedestal Base Mount | 1 |
| 2 | M6 x 10mm Flat Head Machine Screw (FHMS) | 3 |
| *3 | 1/4-20 x 1-3/4" Carriage Bolt | 4 |
| *4 | 1/4-20 x 3/4" Carriage Bolt | 4 |
| *5 | 1-1/4" (OD) Flat Washer | 4 |
| *6 | 1/4-20 Kep Nut | 4 |
| **7 | Rubber Bumper | 9 |
| 8 | Hole Cover | 10 |

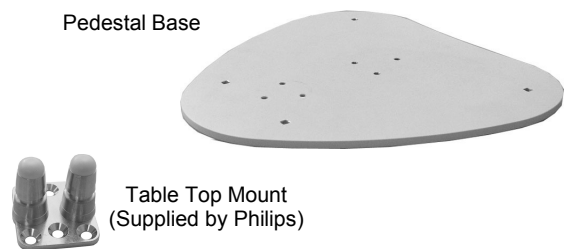


Tools Required: *Drill, *17/64" drill bit, *7/16" wrench, Phillips screwdriver.

*Required only if mounting Pedestal Mount on a rigid surface (counter top).

**Required if not attaching Pedestal Mount to rigid surface.


Attaching the Table Top Mount to the Pedestal Base

1. Fasten the Table Top Mount to the Base Plate with three (3) M6 x 10mm FHMS as shown below.

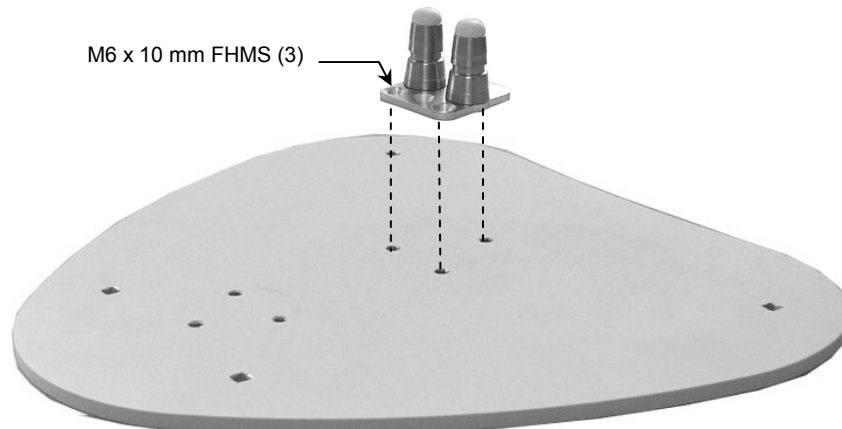

Mounting the Pedestal Mount on a Counter Top


1. Using the Base Plate as a template, drill four (4) holes in the counter top with a 17/64" drill bit.
2. Fasten Base Plate to counter top using four (4) 1/4-20 carriage bolts, four (4) 1-1/4" flat washers, and four (4) 1/4-20 keps nuts. Carriage Bolts are inserted through the Base Plate from the top.

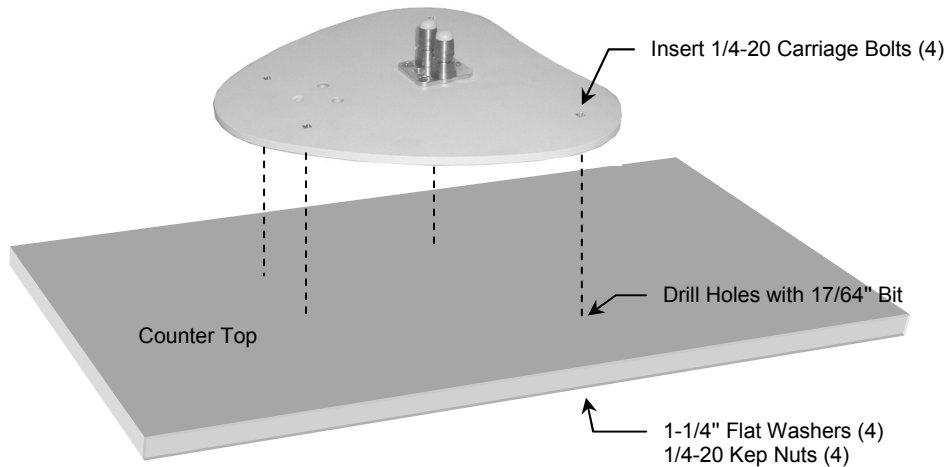

Pedestal Mount not Mounted to Counter Top

If the Pedestal Mount will not be mounted to a counter top, rubber bumpers (provided) must be attached to the bottom of the Base Plate.


1. Using a mild, non-abrasive cleaning solution, clean surface area where bumpers will be applied.
2. Peel nine (9) bumpers from the adhesive backing and apply to Base Plate in the pattern shown below.

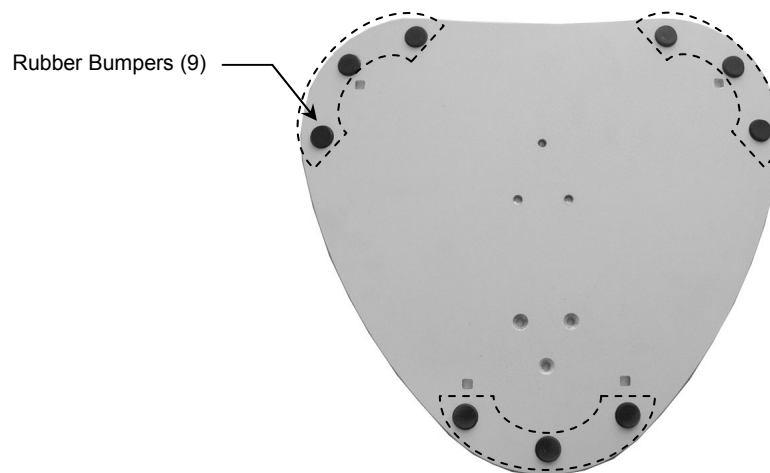

Covering Unused Holes in the Base Plate

Round, color-matching Hole Covers are provided for covering the unused holes in the Base Plate.

1. Using a mild, non-abrasive cleaning solution, clean surface area around holes to be covered.
2. Peel Covers from adhesive backing and apply over holes in Base Plate. **Installation Note:** The maximum usage of Hole Covers is shown in photo below right. Your actual application may require fewer Covers.

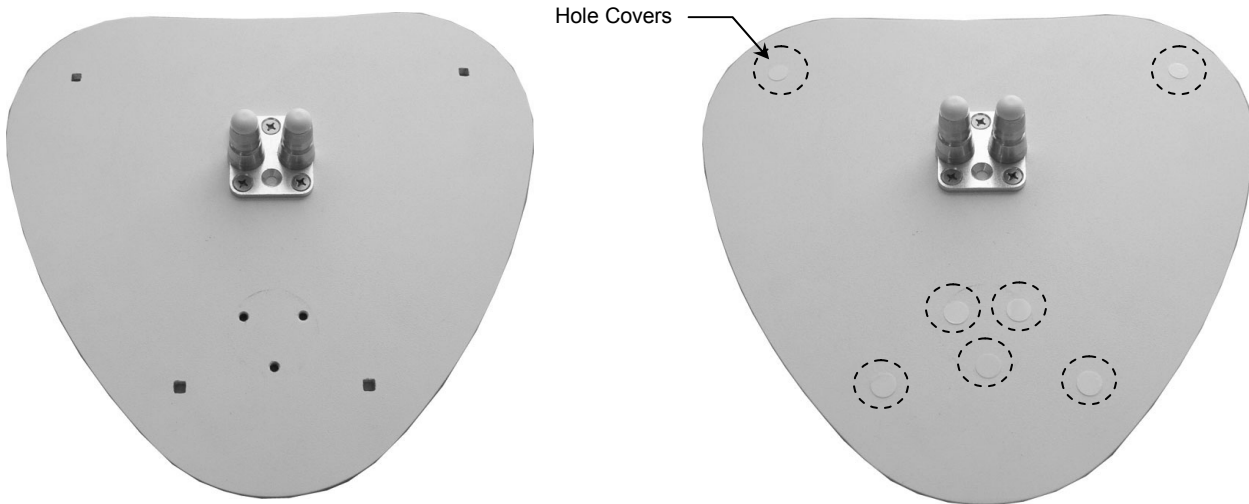

Mounting the Monitor

Mount the monitor in accordance with the Philips Table Top Mount instructions.