


## Installation Guide

### Barcode Scanner Adapter


Confirm Barcode Scanner Compatibility with Dimensional Drawings below

The purpose of this guide is to:

1. Describe the attachment of the Barcode Scanner to the Barcode Scanner Adapter.

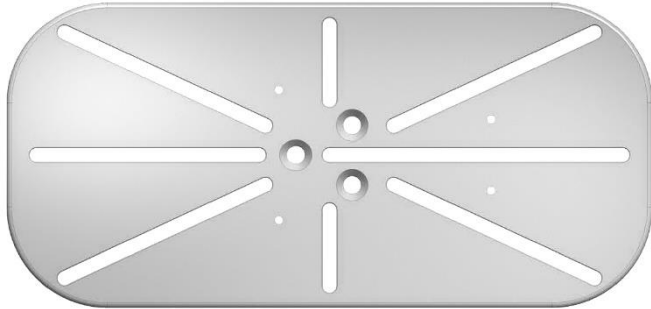

BCR-0001-17

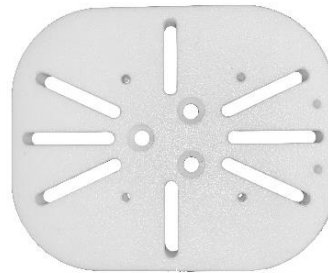

BCR-0001-18

### Parts Reference

The following parts and hardware are included in this installation kit:

Item #	Description	Qty
1	BARCODE SCANNER ADAPTER WITH SLOT	1
2	M3 x 6 PHMS	2
3	M3 x 8 PHMS	4
4	M4 x 20 FHMS	4
5	M4 X 4.3 x 9 FLAT WASHER	4
6	M4 x 16MM FHMS	4
7	10-32 x 3/4" FHMS	3
8	6-32 x 5/8" PHMS	4
9	8-32 x 5/8" FHMS	4
10	8-32 x 3/4" FHMS	4
11	8-32 KEP NUT	4


### Tools required



- Philips #2 screwdriver. (not provided)

**Swivel Head Attachment:** Using the Barcode Scanner Adapter, align with the holes on the swivel head using **(3)** 10-32 X ¾ FHMS. Determine the screws required then, align the slots with the two holes on the bottom of the barcode scanner and tighten.



**L Bracket Barcode Scanner Mount Attachment:** Locate the two holes on the Barcode Scanner that line up with the slots and secure with **(2)** M3 x 8 PHMS. Determine screws required then, align the Barcode Scanner holes to the holes on the mount and tighten.

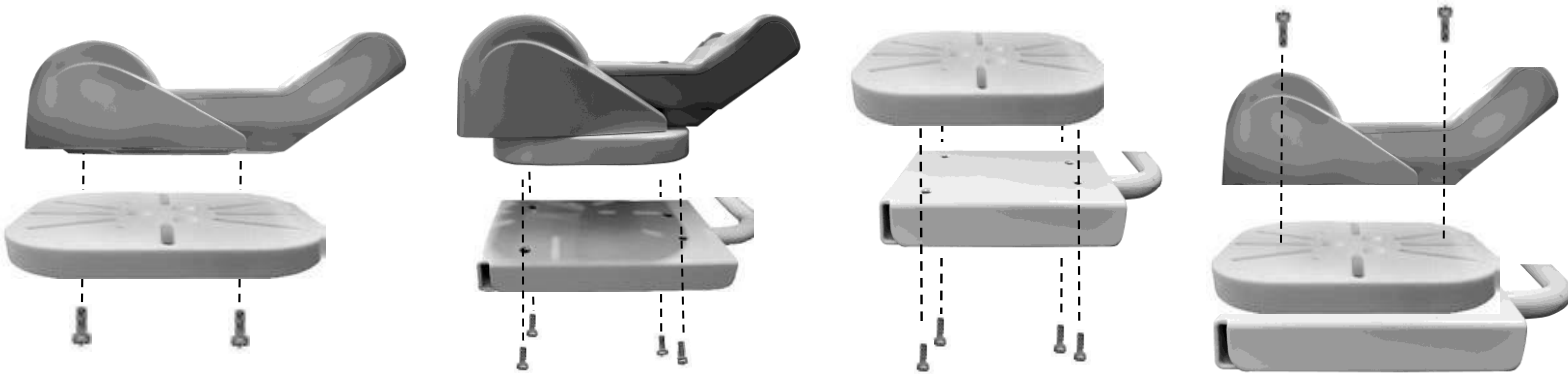

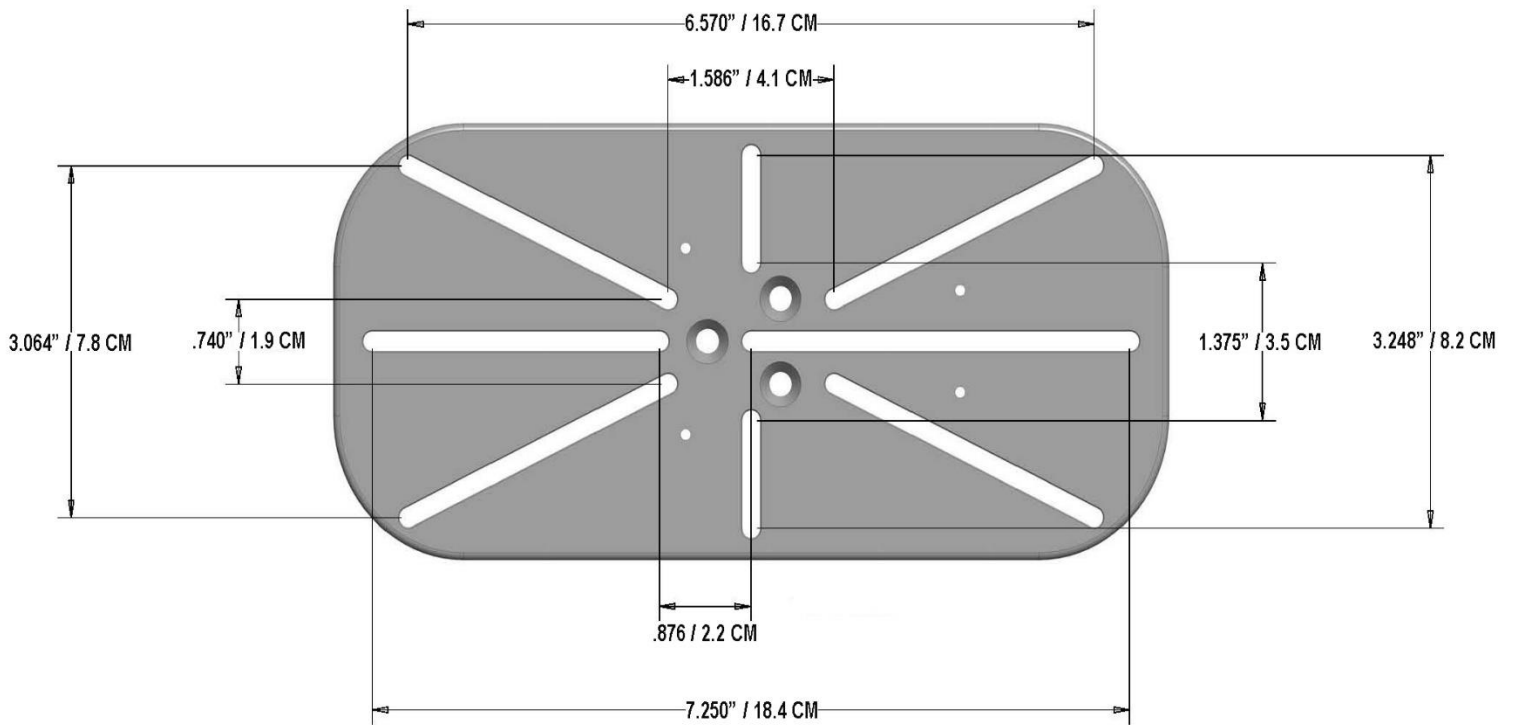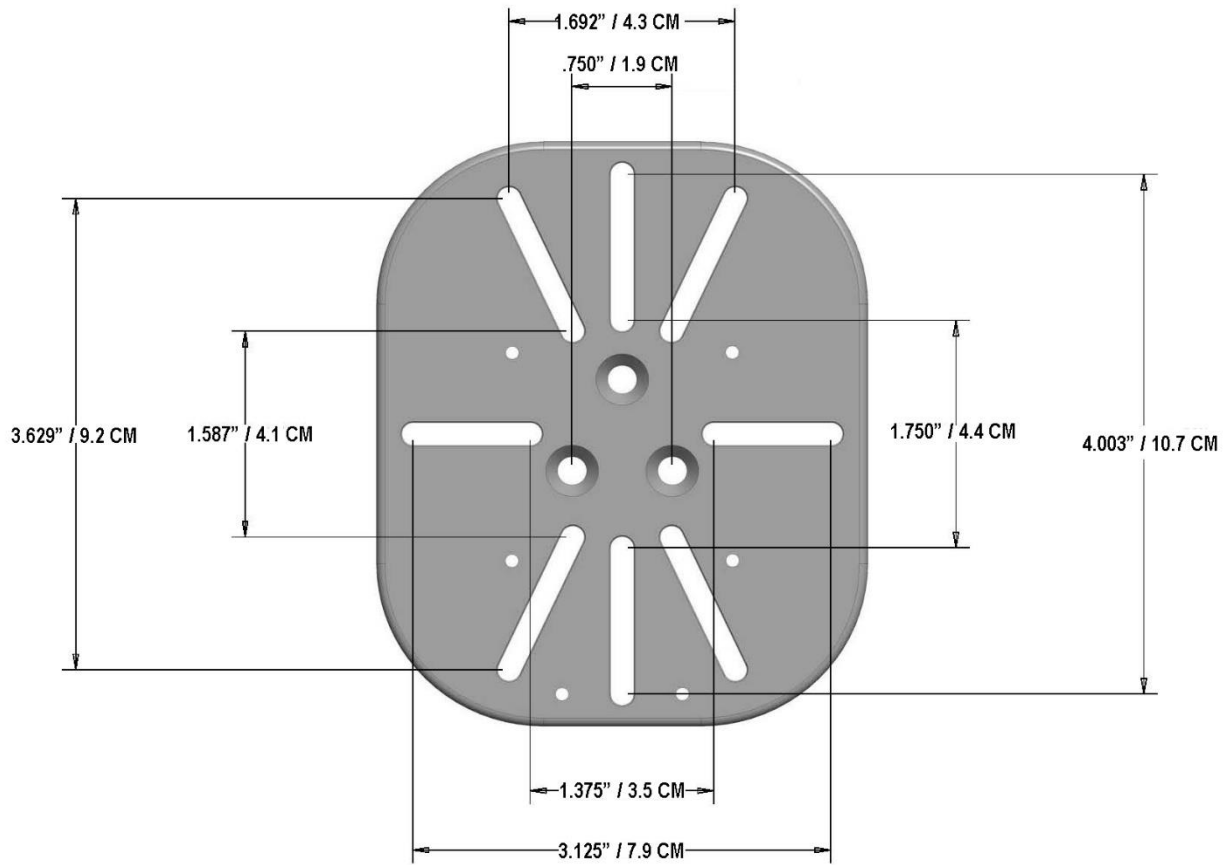

## **Routine Maintenance**

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

## **Cleaning the Mounting Assembly**

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately, using water dampened cloth. Dry the mounting assembly thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products before installing the arm in the Channel.