

Installation Guide

Datacaptor Capsuletech Flush Wall Mount

The purpose of this guide is to:

- Describe mounting the Datacaptor Capsuletech Flush Wall Mount.




Parts Reference

The following parts are included in this installation kit :

Item #	Description	Qty
1	Flush Wall Mount	1
2	8/32 x 2 1/2 Pan Head Machine Screw (PHMS)	2
3	2" Spacers	2
4	#8 Flat Washers	2
5	8-32 ESNA Nut	2
3	1/8" Hex Wrench	1

(Hardware not shown)

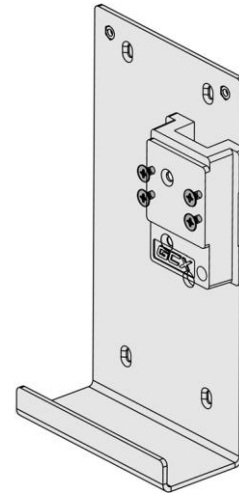


Tools Required:

- 1/8" Hex Wrench (supplied)
- Phillips Screwdriver

Installing Flush Wall Mount in Channel

1. Guide the Slide (rear of Flush Mount) into top of Channel and move to desired location (below left).
2. Using a 1/8" Hex wrench (provided), tighten set screw to secure position in Channel (below right).

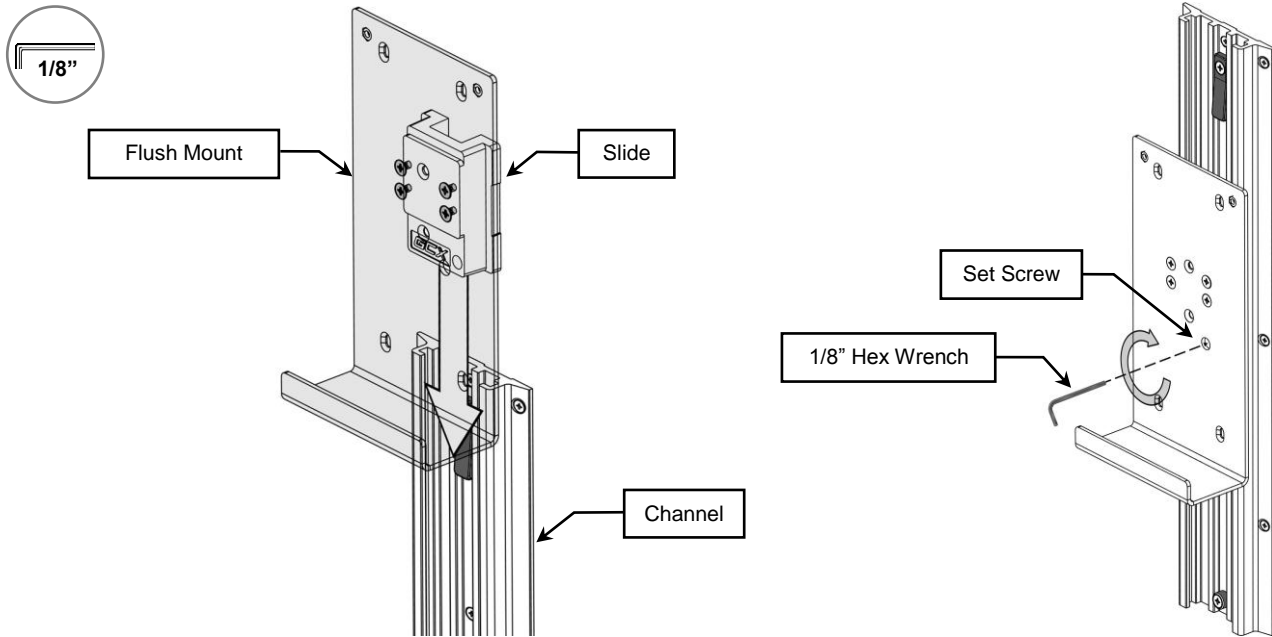

Installing Datacaptror Capsuletech into Flush Wall Mount

1. Lower Datacaptror Capsuletech into Flush Wall Mount as shown (below left).
2. Install the (2) 8-32 x 2 1/2" PHMS and (2) #8 Washers through the Datacaptror Capsuletech mounting holes and (2) 2" Spacers and thread into the Flush Wall Mount (below right).

Installation Note: 8-32 Esna nuts (2) are provided and can be used on the 8-32 x 2 1/2" PHMS to cover any exposed thread.

