

GCX Mounting Assembly Installation Guide

Dräger Julian Top Shelf Base Plate with Horizontal Channel

The purpose of these instructions is to:

1. Describe attachment of the GCX top shelf base plate with 10" or 20" Horizontal Channel to the Julian anesthesia machine.

Warning: Do not mount equipment such that its center of gravity is outside the perimeter of the Top Plate.

Refer to the pictures below

Note: Horizontal Channel is preinstalled.



Julian Top Shelf




GCX Top Shelf Base Plate



Attach Top Shelf Spacer Bars using
(4) #10-32 x 3/4" FHMS



View of installed Spacer Bars. Bars have double sided VHB tape to secure to Julian top shelf.

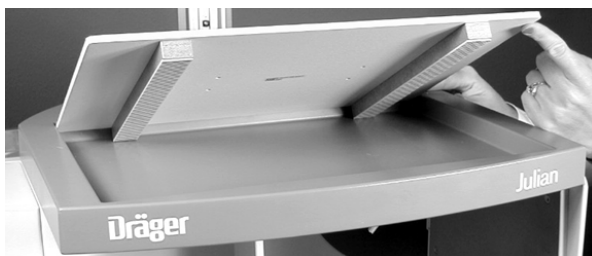

Set the Base Plate on the Julian top shelf. Peel paper from VHB, lower the Base Plate, and press into place



The Base Plate installed on the Julian top shelf
(20" Horizontal Channel/10" bottom)