

GCX Mounting Assembly Installation Guide

Dräger – Fabius Tiro Top Plate with 10.5" Channel


The purpose of this guide is to describe attachment of the Top Plate/Channel to the Fabius Tiro anesthesia machine.

Warning: Do not mount equipment such that its center of gravity is outside the perimeter of the Top Plate.

Parts Reference

The parts list below includes only the parts and hardware that will be used in this installation procedure (see photo for parts; hardware not shown).

Item #	Description	Qty
1	Fabius Tiro Top Plate with 10.5" Channel	1
2	M5 x 35mm Pan Head Machine Screw (PHMS)	3

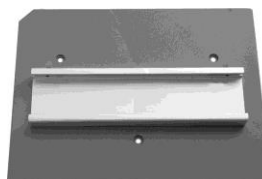



Top Plate with 10.5" Channel

Tools Required: Phillips screwdriver, drill and 1/4" drill bit.




Attaching the Top Plate

1. Find the three (3) threaded inserts by locating the three (3) dimples toward the top front of the anesthesia machine (photo below left). **Note:** The fourth dimple at the rear is not used for this installation.
2. Drill holes precisely in the center of the dimples using a 1/4" diameter drill bit (photo below right). **CAUTION:** Do not allow the drill bit to penetrate more than 1/4" into the top surface. Drilling too deeply will damage insert threads and internal components of the machine. Remove all drilling chips.

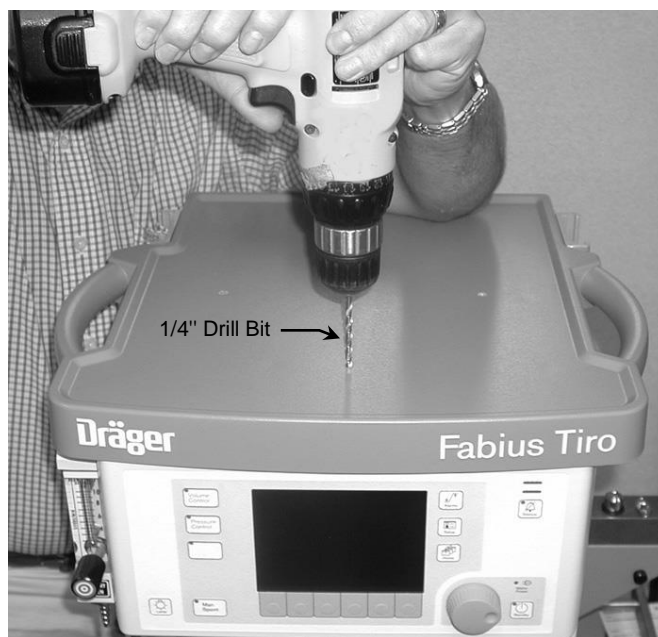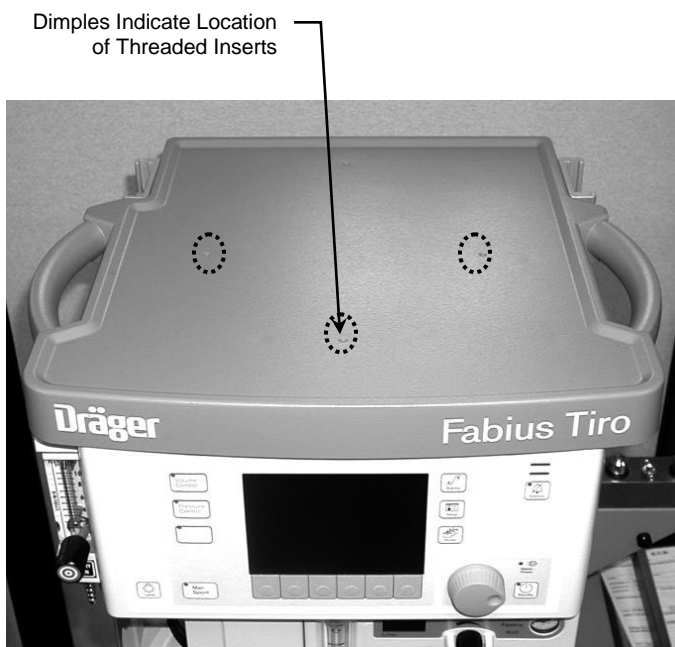

3. Align the Top Plate with the three (3) mounting holes toward the front of the machine (photo below left).
4. Thread the three (3) M5 x 35mm PHMS through the Top Plate and into the holes drilled in step #2 (photo below right).

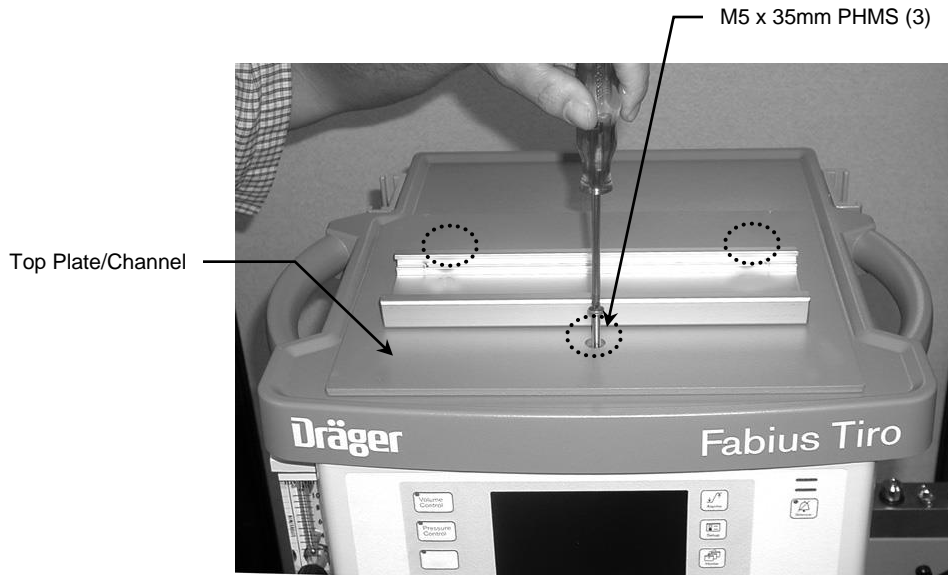

5. Attach the Tip Caution Label (shown below) to the front center of the blue top shelf bezel (see photo).

