

Installation Guide

Dräger Primus IE Top Plate with 15" Horizontal Channel

The purpose of this guide is to:

- Describe installation of the Top Plate on the machine.



Warning: Top Plate for Primus IE only, and will not work on Primus.


Warning: Do not mount equipment such that its center of gravity is outside the perimeter of the Top Plate.

Warning: Carefully tighten screws by hand only. Do not use power tools.

Parts Reference

The following parts are included in this installation kit :

Item #	Description	Qty
1	Top Plate with 15" Horizontal Channel	1
2	M4 x 10mm Flat Head Machine Screw (FHMS)	4

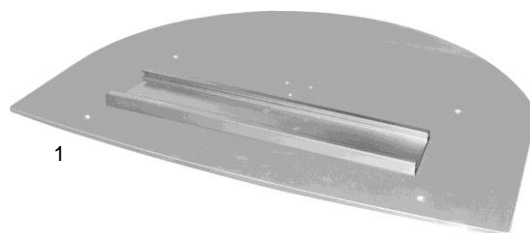


Tools Required:

- Phillips screwdriver (not provided)

Attaching Top Plate to Primus IE

Align Top Plate over mounting holes and fasten with four (4) M4 x 10mm FHMS as shown below.

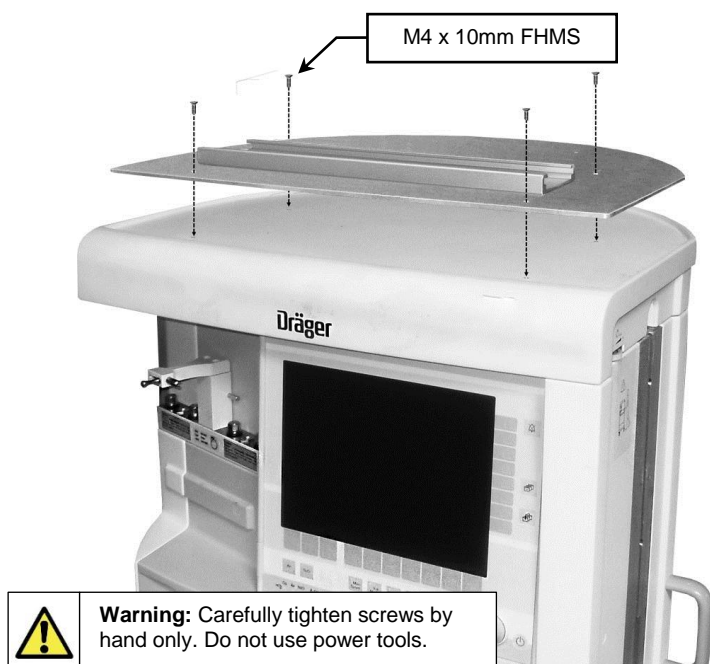

Warning: Carefully tighten screws by hand only. Do not use power tools.

