

Installation Guide

Top Plate/Channel for Datex-Ohmeda Aespire Anesthesia Machine

The purpose of this guide is to describe attachment of Top Plate to anesthesia machine.

Parts Reference

The following parts and hardware will be used in this procedure (see photo for parts, hardware not shown):

Item #	Description	Qty
1	Top Plate/Channel Assembly	1
2	M6 x 12mm Flat Head Machine Screw (FHMS)	4




Top Plate/Channel Assembly

Tools Required

- Phillips screwdriver (not provided)

Attaching Top Plate/Channel Assembly

1. Remove all instruments attached to top shelf of machine.
2. Align four (4) mounting holes in Top Plate/Channel Assembly with threaded mounting holes in top of machine. Fasten Plate to machine with four (4) M6 x 12mm FHMS as shown below.

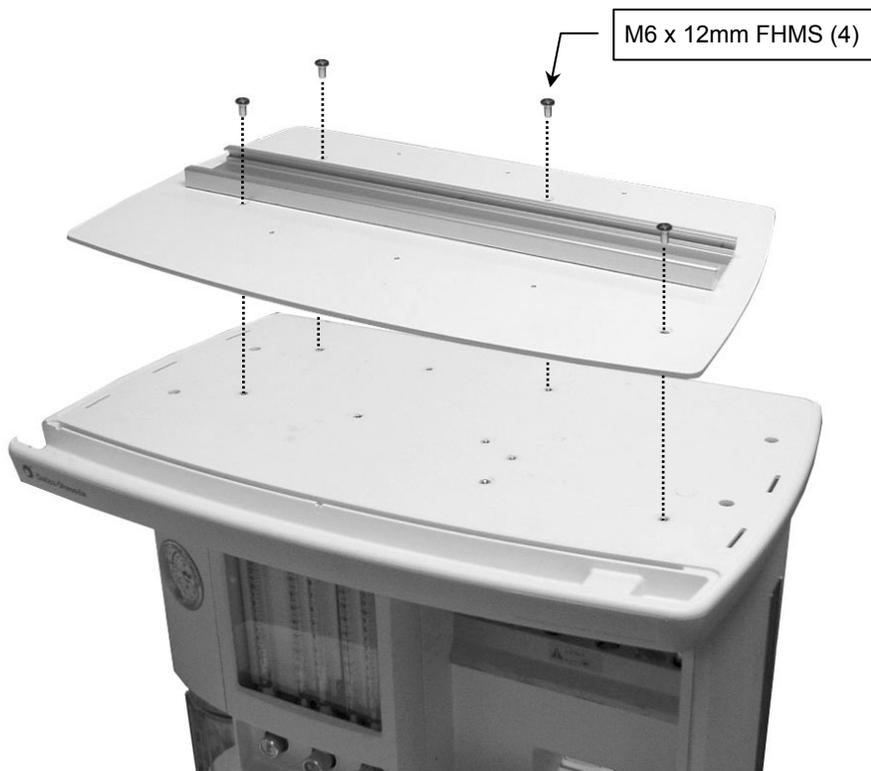

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened or adjusted as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the arm that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the arm. Wipe any cleaning agents off the arm immediately using a water-dampened cloth. Dry mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.