



GCX Mounting Assembly Installation Guide

M Series Flush Mount for Horizontal Channel

The purpose of these instructions is to:

1. Describe attachment of Flush Mount to Horizontal Channel.

1. Insert the Flush Mount into the Horizontal Channel by supporting the bottom of the mount with one hand and guide the Slide into the side of the Channel. Once the Flush Mount is at the desired location, using the supplied 1/8" (3mm) hex key to tighten the 1/4-20 x 3/4" SHSS (2) set screws.

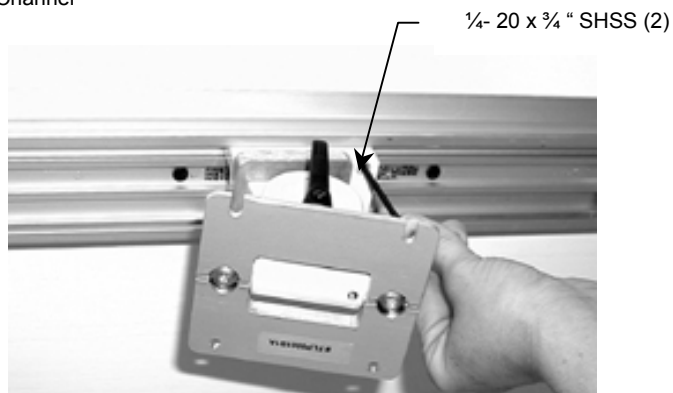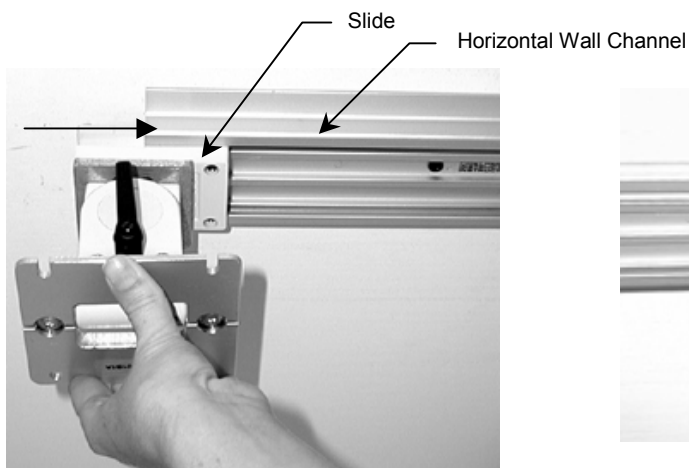

2. Complete the installation in accordance with the separately printed instructions, DU-WMM-0002.