

GCX Mounting Assembly Operation/Installation Manual Channel for Hill Rom Accessory Track



The purpose of these instructions is to:


1. Describe attachment of GCX Channel to the Hill Rom Accessory Track.




WARNING: EQUIPMENT MOUNTED MUST NOT EXCEED THE DUTY RATING OF THE ACCESSORY CHANNEL.

INSTALLATION


1. Insert the Slide into the Accessory Track using the opening closest to the location where the Channel will be positioned.

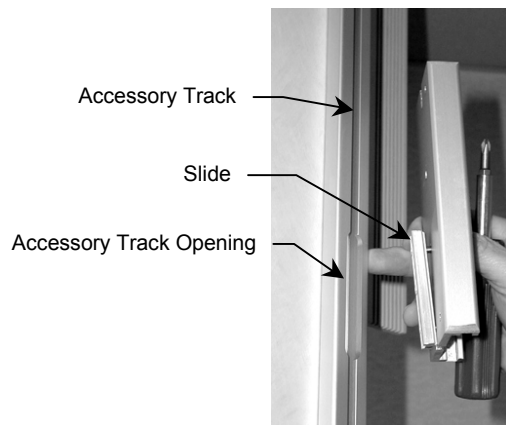

3. Insert the second (top) Slide into the Accessory Track opening and slide behind the Channel so the top screw enters the slot at the top of the Channel.

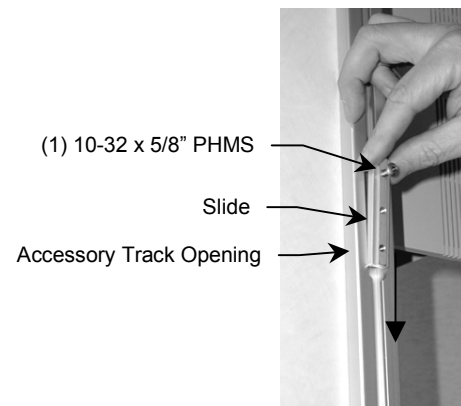

2. Position the Channel and tighten the (2) 10-32 x 5/8" PHMS to secure the Slide in the Accessory Track.

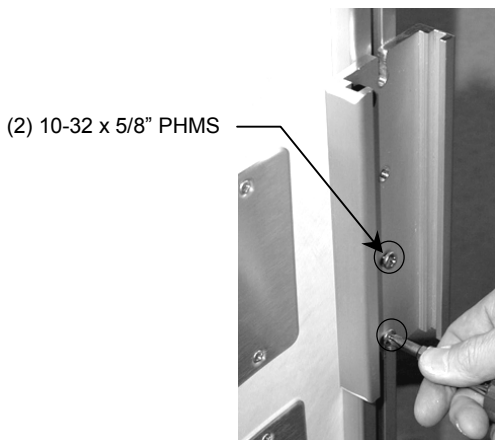

4. Insert the second 10-32 x 5/8" PHMS into the Slide and tighten both to secure Channel in position.

