

GCX Mounting Assembly Operation/Installation Manual Channels for Hill-Rom Horizon Headwall – Single & Dual Rail

The purpose of these instructions is to:



1. Describe attachment of GCX Channel to the Hill-Rom Horizon headwall.



WARNING: EQUIPMENT MOUNTED MUST NOT EXCEED THE DUTY RATING OF THE HEADWALL.



6 Inch Single Rail Channel - HR-0007-02

1. Position the Channel as desired and insert the Rail Fitting onto the Rail.
2. Secure Channel by tightening the (2) 10-32 x 1/2" OHMS and (1) 10-32 x 1/2" PHMS.

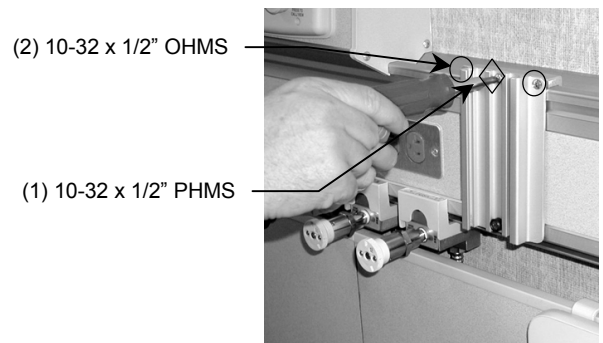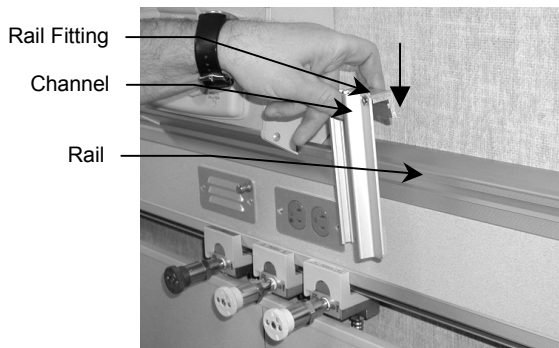

23" Dual Rail Channel - HR-0007-06

1. Position the Channel as desired and insert the Rail Fittings onto the top and bottom Rails.
2. Secure Channel by tightening the top Rail Fitting (2) 10-32 x 3/4" OHMS & (1) 10-32 x 3/4" PHMS, and the bottom Rail Fitting (2) 10-32 x 3/4" PHMS

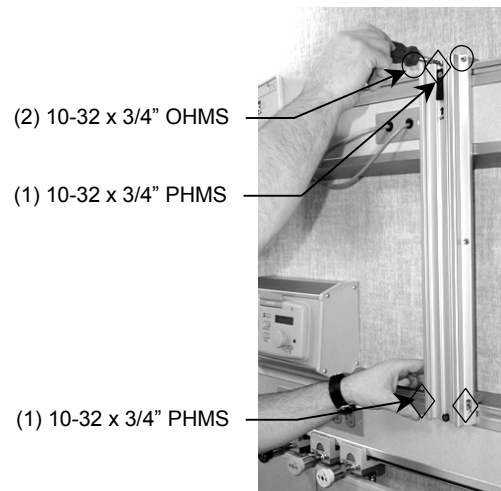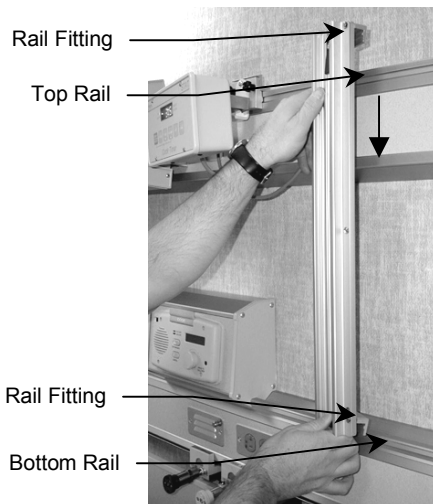


35.56" Dual Rail Channel - HR-0007-07

1. Position the Channel as desired and insert the Rail Fittings onto the top and bottom Rails.



2. Secure Channel by tightening the top Rail Fitting (2) 10-32 x 3/4" OHMS & (1) 10-32 x 3/4" PHMS.

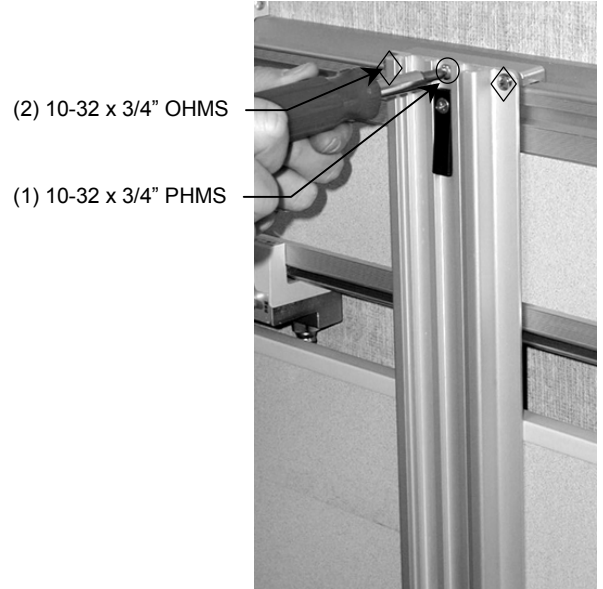

3. Tighten the bottom Rail Fitting (2) 10-32 x 3/4" PHMS.

