

Installation Guide

Mounting Adapter for Mindray TM5 or TM8 Monitor

The purpose of this guide is to describe attachment of Mounting Adapter to monitors (pages 1 – 2), and mounting monitor of slide-on mounting plate (page 2).

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):

Item #	Description	Qty
1	Mounting Adapter	1
2	M4 x 20 mm Pan Head Machine Screw (PHMS)	4




Tools required


Phillips screwdriver (not provided).

Attaching Mounting Adapter to Monitor

1. Fasten Mounting Adapter to bottom of T5 monitor with four (4) M4 x 20 mm PHMS as shown below. T8 Monitor shown on page 2.

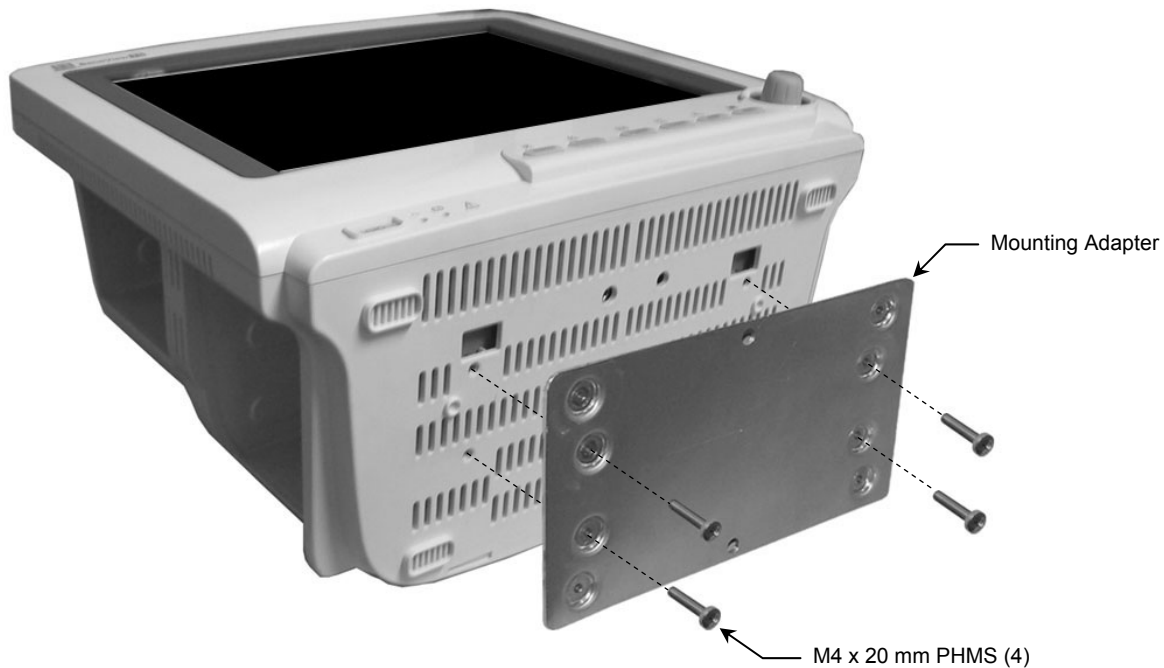

Attachment of Mounting Adapter to T8 Monitor

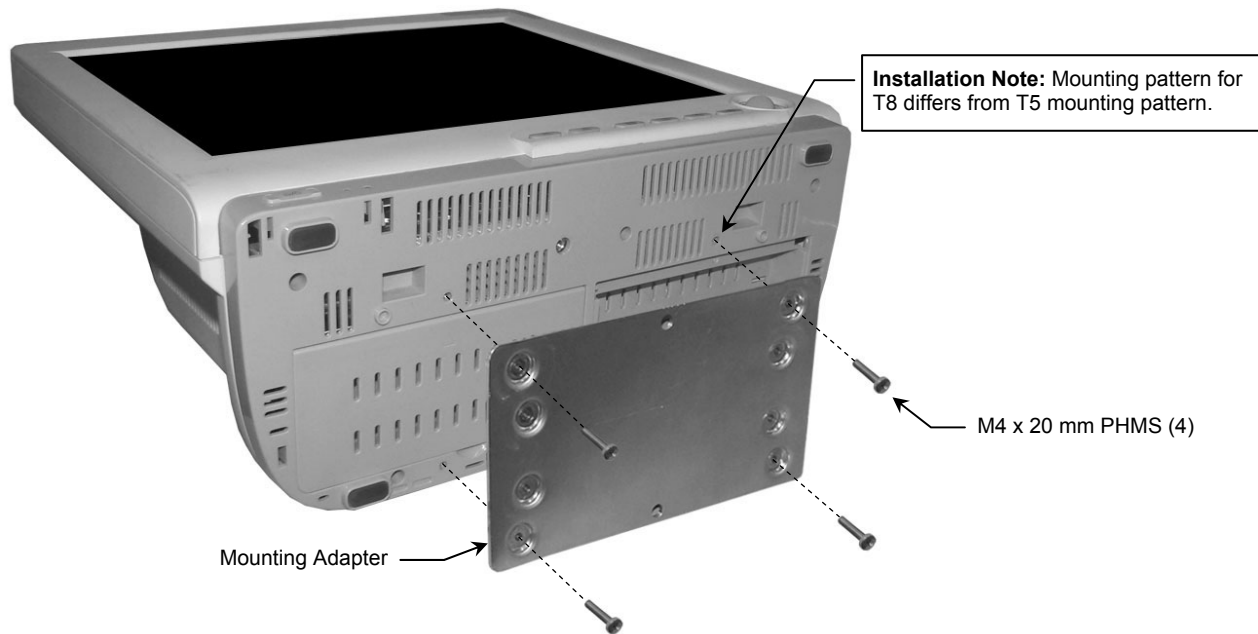

Mounting Monitor on Slide-on Mounting Plate

1. Slide Mounting Adapter onto mounting plate while pulling spring-loaded plunger. Plunger will snap into clearance hole when monitor is correctly mounted. Tighten Nylon screws on bottom of Mounting Plate.

