

### Installation Guide Utility 3-Hook for Philips MP2 Pole Mounts

*The purpose of this guide is to describe attachment of Utility Hook Assembly to Philips MP2 Tilt/Swivel Post/Rail Clamp and Docking Station.*

**Installation Note:** Docking Station supplied by Philips.

#### Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):

Item #	Description	Qty
1	Utility Hooks	1
2	M4 x 12mm Pan Head Machine Screw (PHMS)	2



**Tools Required:** Phillips screwdriver (not provided).

#### Attaching Utility Hooks to MP2 Tilt/Swivel Post/Rail Clamp

