

Counter Top Mount and Horizontal Channel Mount with Tilt & Swivel Adjustments


The purpose of this guide:

- 1) Describe installation of Counter Top Mount.
- 2) Describe installation of Horizontal Channel Mount
- 3) Describe Tilt & Swivel adjustments

Installing the Counter Top Mount

Tools Required: Drill, Adjustable wrench.


- 1) Verify the counter top is suitable to hold the mount and expected load.
- 2) Use the base of the counter top mount as a template to mark holes for drilling.
- 3) Drill four (4) 9/32" / 7mm holes.
- 4) Use either the included 1/4"-20 X 3/4" or 1/4"-20 X 1 3/4" carriage bolts (with kep nuts and washers) to secure mount to counter top. Orient the mount so the Spring Plunger is at the rear – refer to picture below:

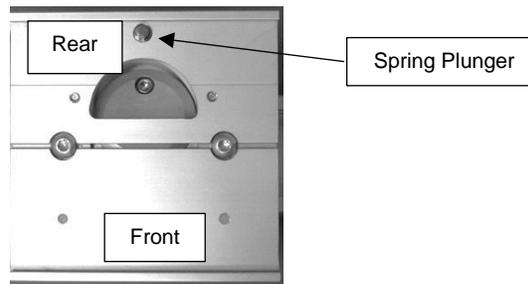


Installing the Horizontal Channel Mount

Tools Required: Phillips screwdriver

- 1) Orient the mount so the Spring Plunger is at the rear – refer to picture below:

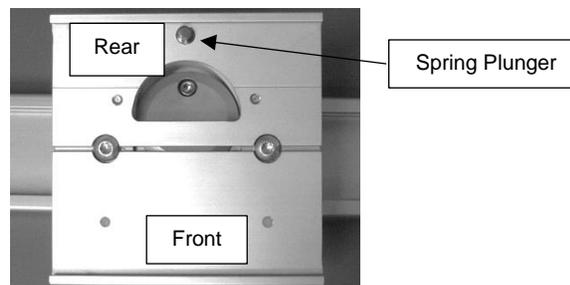

- 2) Align the Slide of the mount with edge of the Horizontal Channel and insert into Channel -- slide Mount to desired position in Channel.

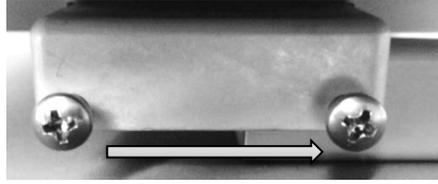

- 3) Using a Phillips screwdriver, tighten two (2) screws at rear of Mount to secure position of the mount in the channel.



- 4) Mount device in accordance with separate instructions.

Tilt and Swivel Tension Adjustments


Tools Required: 5/32 Hex Wrench (included), Phillips Screwdriver

Tilt tension

- 1) Tighten or loosen the socket set screw on the top using the included 5/32" hex wrench.

Swivel tension

- 1) Tighten or loose screw on side of post immediately below Mounting Plate.

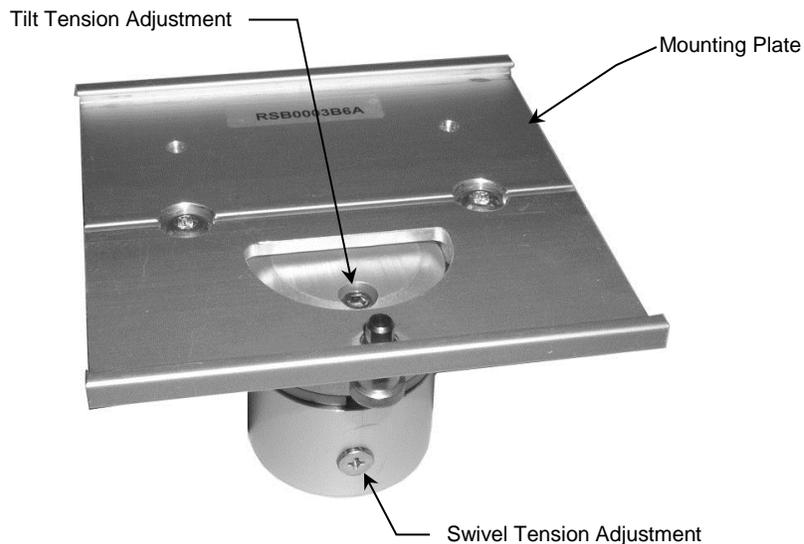

Routine Maintenance of the Mounting Assembly

Periodically inspect all mounting hardware. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately using a water-dampened cloth. Dry mounting assembly thoroughly after cleaning.