

Installation Guide

Crossmember for Mounting Dual Spacelabs Module Hangers (Yokes)

The purpose of this guide is to:


1. Describe attachment of Crossmember to M-Series Arm or *down post (page 1).
2. Describe attachment of yokes to Crossmember (page 2).

*See **WARNING** below.

Parts Reference

The following parts and hardware are included with this installation kit (major components shown):

Item #	Description	Qty
1	Crossmember	1
2	#10-32 x 1/2" Flat Head Machine Screw (FHMS)	3
3	#10-32 x 7/8" FHMS	3
4	#10-32 x 5/16" Pan Head Machine Screw (PHMS)	6

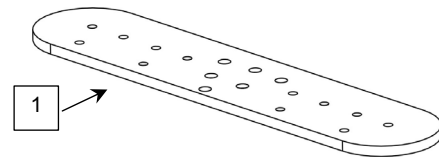


Tools Required

Phillips screwdriver (not provided).



Attaching Crossmember to M-Series Arm or *Down Post

WARNING: Do not attach Crossmember directly to VHM Arm. Do Not exceed load rating of VHM or M-Series Arm.

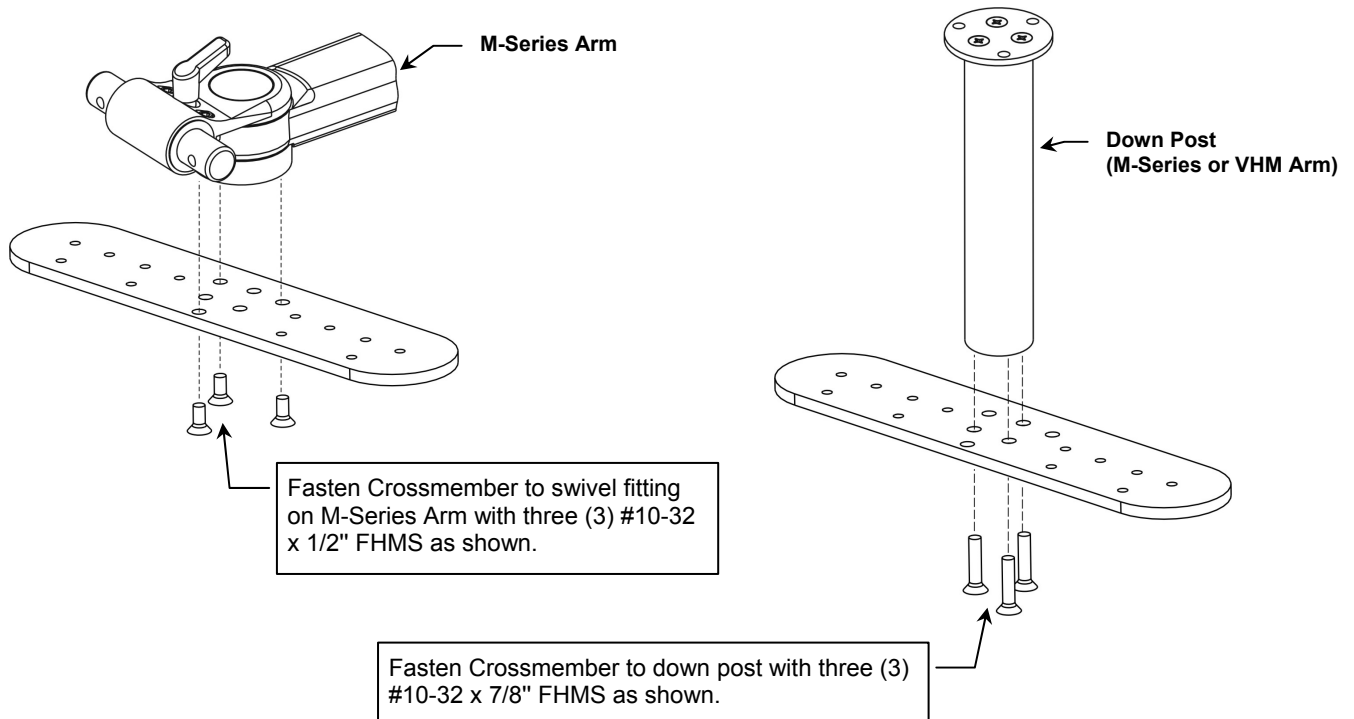

Attaching Yokes to Crossmember

Six (6) #10-32 x 5/16" PHMS are provided for attachment of two (2) yokes to Crossmember. Attachment of (2) wide yokes is shown in upper illustration. The lower illustration shows attachment of one narrow yoke and one wide yoke. Yoke attachment is not necessarily limited to the examples shown on this page.

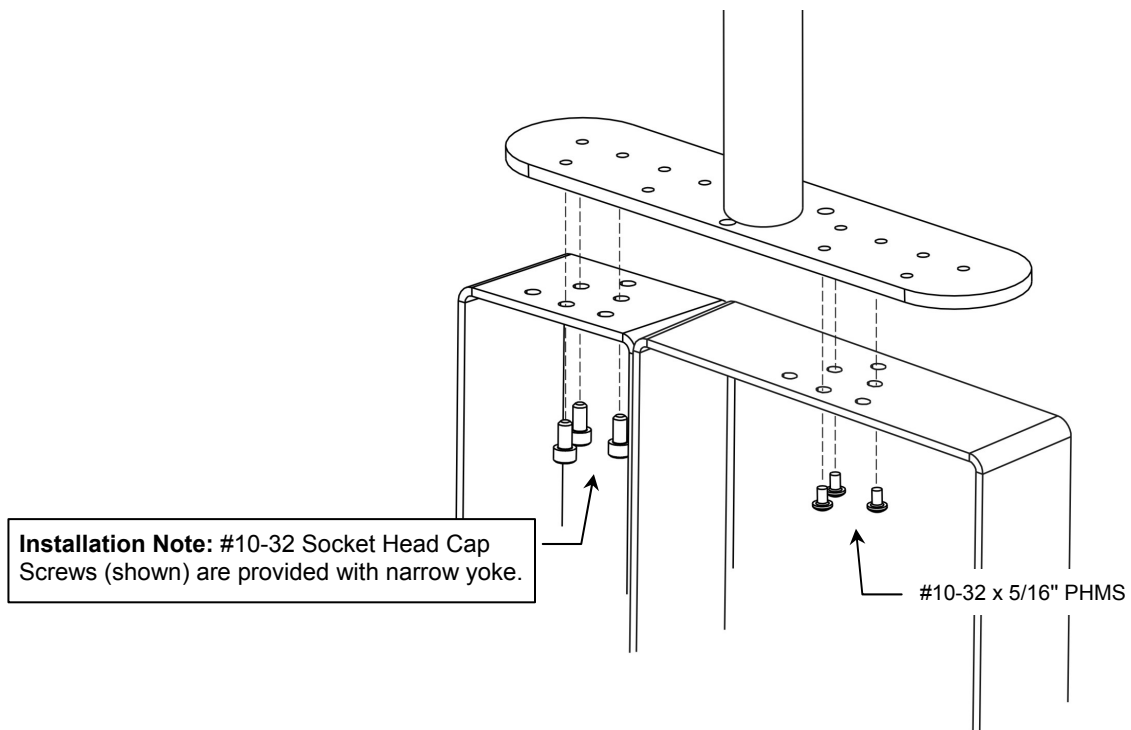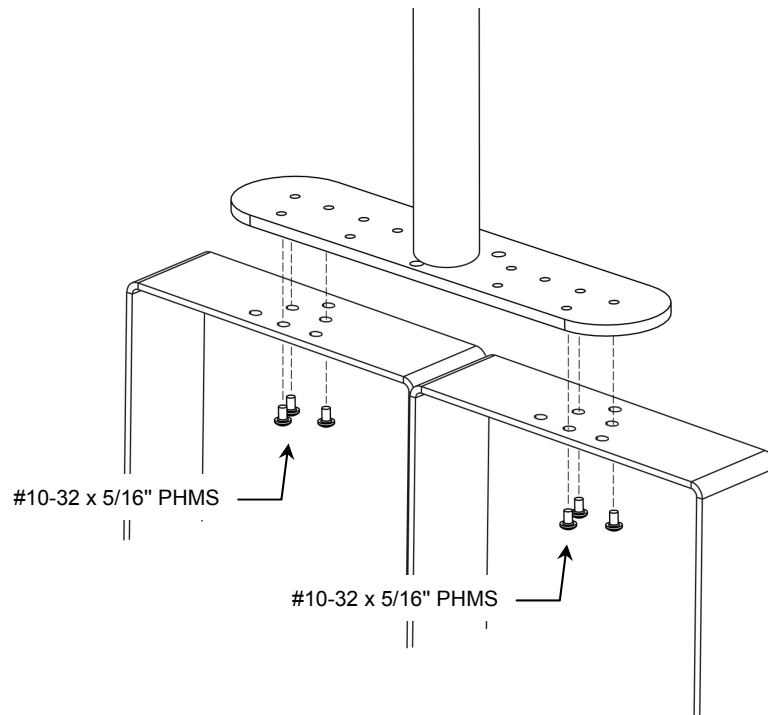

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened as necessary.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately, using a water-dampened cloth. Dry the mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.