

## Installation Guide

### Slide-on Mounting Plate Attachment to Swivel-only M-Series Arm

*The purpose of this guide is to describe attachment of a slide-on mounting plate to a swivel-only M-Series Arm.*

**Installation Note:** This installation guide shows attachment of a 5" Mounting Plate to a Swivel-only M-Series Arm. This installation procedure (including hardware) also applies to attachment of the 7.6", 9", 11", and 14" Mounting Plates.

#### Parts Reference

The parts list below includes the parts and hardware that will be used in this installation procedure (see photos for parts; hardware not shown).

Item #	Description	Qty
1	Mounting Plate (5", 7.6", 9", 11", or 14")	1
2	#10-32 x 5/8" Flat Head Machine Screw (FHMS)	3

Mounting Plate  
(5" Shown)



**Tools Required:** Phillips screwdriver.

#### Attaching Mounting Plate to M-Series Arm

1. Align the 3-hole mounting pattern in the Mounting Plate with the corresponding threaded holes in the swivel-only head (front end of M-Series Arm). Fasten Mounting Plate to head with three (3) #10-32 x 5/8" FHMS.

