



**Installation Guide**  
**VHM-P (Non-Locking) Variable Height Arm (Slide-Below-Arm Configuration)**  
**with Slide-In Mounting Plate**  
 (Refer to qualified personnel)

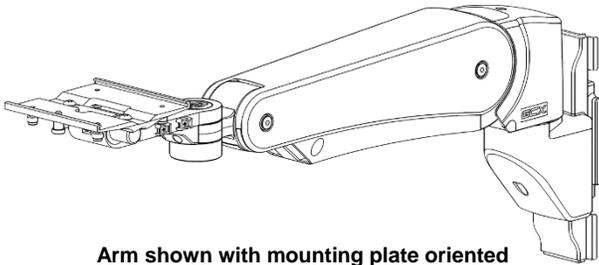

Arm shown with mounting plate oriented as shipped

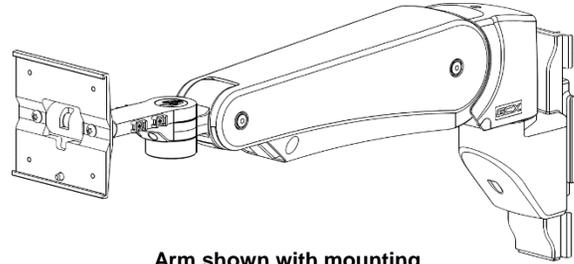

Arm shown with mounting plate reoriented

VHM-P (Non-Locking)		
Part Number	Arm Load Range	Max Tilt Load
WS-0012-04	15-30 lb / 6.8-13.6 kg	30 lb / 13.6 kg
WS-0012-06	25-45 lb / 11.3-20.4 kg	30 lb / 13.6 kg

**Installation Time**

Approximately 10-15 minutes

*The purpose of this manual is to describe general installation, operation, and adjustment procedures for VHM-P Series Arms. This manual should be used in conjunction with any instrument-specific installation guides. Please read this manual and all instrument-specific installation material before installing or using this product.*


**Table of Contents**

1.0	Parts Reference.....	2
2.0	Tools Required.....	2
3.0	Product Warning / Safety Labels.....	3
4.0	Installation, Operation, and Maintenance Warnings.....	4
5.0	Installing the Arm in a Channel .....	5
6.0	Mounting the Device to the Arm.....	6
7.0	Adjusting the Arm .....	7
8.0	Cable Management.....	10
9.0	Routine Maintenance.....	18
10.0	Cleaning the Arm.....	18
11.0	Troubleshooting the Arm.....	19

## 1.0 Parts Reference

The following parts and hardware are included and labeled accordingly:

Item #	Description	Quantity
1	VHM-P Channel Mount Arm	1
2	Cable Cover	1
3	Pivot Cover (upper)	1
4	Pivot Cover (lower)	1
5	Pivot Cover Cap	1
6	Slide to Channel Transition Cover	1
7	16" (40.6 cm) Channel Cover	1
8	30.5" (77.5 cm) Cable Sleeve	2
9	Cable Tie	8
10	Adjustable Stop	1
11	Tilt Adjustment Lever	1
12	1/2" (13mm) x 3/4" (19mm) Wrench	1
13	1/8" Hex Wrench ( <i>not shown</i> )	1
14	5/32" Hex Wrench ( <i>not shown</i> )	1

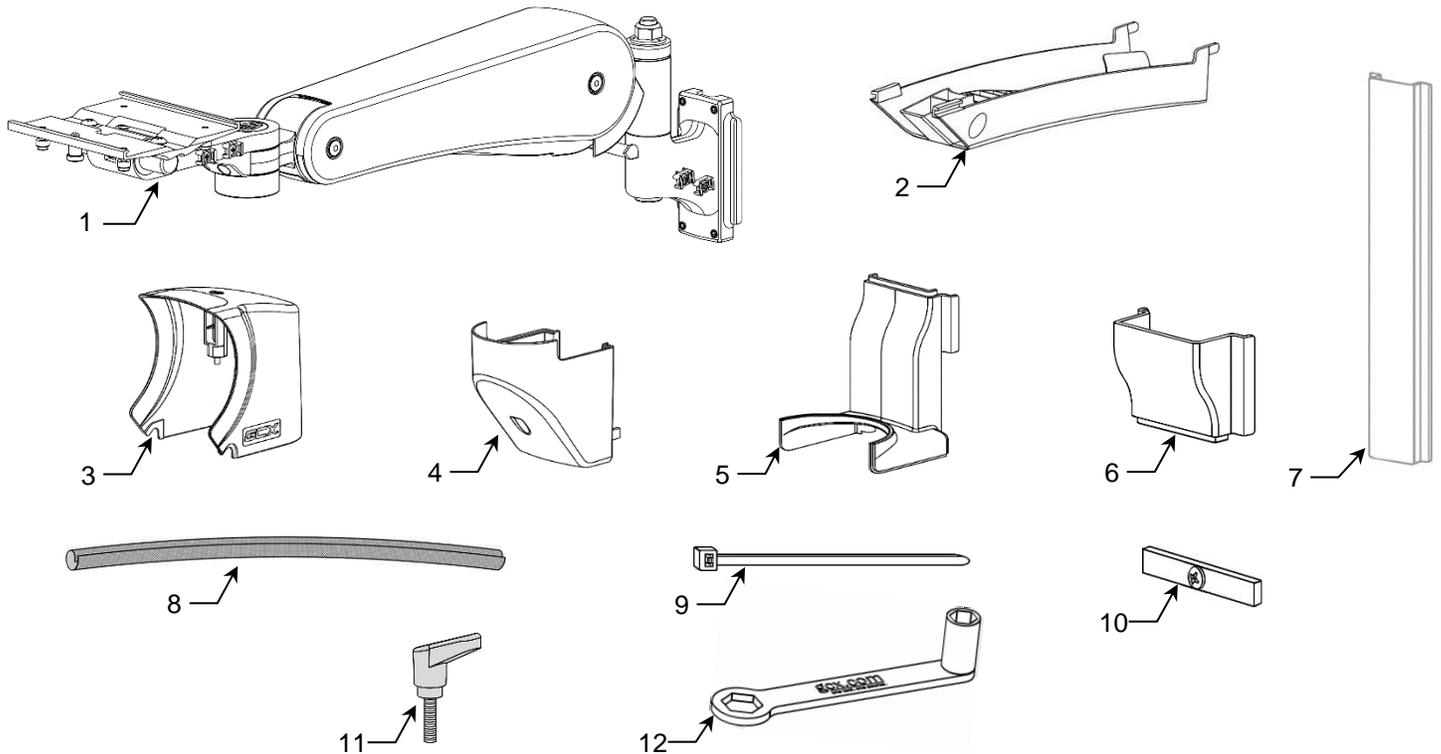

## 2.0 Tools Required

The tools listed below are required to install and adjust the arm.

### Provided



1. 1/8" Hex Wrench
2. 5/32" Hex Wrench
3. 1/2" (13mm) / 3/4" (19mm) Wrench

### Not Provided

4. #2 and #3 Phillips Screwdrivers
5. Scissors, utility knife or similar cutting tool
6. Wire Cutters (for cable ties)

### 3.0 Product Warning / Safety Labels

#### Safety Label Locations

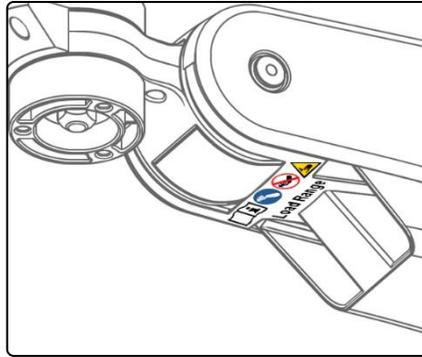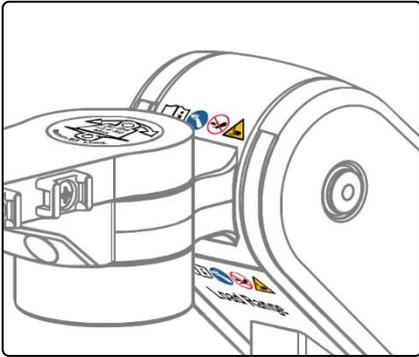

Refer to Installation Guide for additional information.



Remove the device only when the arm is at the highest position.



Do not remove the device when the arm is in a lowered position.



Potential pinch point that may cause personal injury.



Maximum weight for mounting plate with tilt function.

**Counter Balance Load Range:**  
**25 - 45 lb / 11.3 - 20.4 kg**  
or  
**15 - 30 lb / 6.8 - 13.6 kg**


## 4.0 Installation, Operation, and Maintenance Warnings

This section contains warnings regarding the installation, operation, and maintenance of the arm. This section must be read in its entirety before installing, operating, and maintaining the arm. Failure to follow these warnings may result in damage to equipment or injury to personnel.




### Warnings

- Do not position the arm or mounted device above a patient. Note that the arm has a wide range of motion both up/down and side to side. Please consider carefully the device being mounted and the proximity of the mounting assembly to other equipment, hospital personnel, and the patient. GCX recommends that the hospital's risk management personnel verify that the application is appropriate prior to installation and use of this arm.
- Before the arm is channel mounted, verify that the channel has been installed and approved in accordance with the channel installation guide. Documents are available on our website: [www.gcx.com](http://www.gcx.com)
- Ensure that the total weight of the devices being mounted does not exceed the load rating of the arm. Check the bottom surface of the arm for the load range label. Refer to Section 3.0.
- Do not use power tools to make any arm adjustments.
- The mounted device or arm may move suddenly due to normal wear or improper adjustment of the tilt, swivel, and pivot functions (see Sections 7.2, 7.3 and 7.4), improper counterbalance (Section 7.1) or ultimately, gas spring end of life. The gas spring has a limited life span and will lose some strength over a long period of time. **The arm must be inspected and maintained at least once a year.** This inspection must include the steps outlined in Section 9.0.
- Failure to periodically inspect and adjust the arm as instructed may result in damage to equipment or injury. If you are unable to adjust the arm or require service, contact GCX technical support at 800.228.2555 or + 1 707.773.1100.
- The VHM-P arm has a wide range of motion both up/down and side to side. Please consider potential pinch points that may cause personal injury.

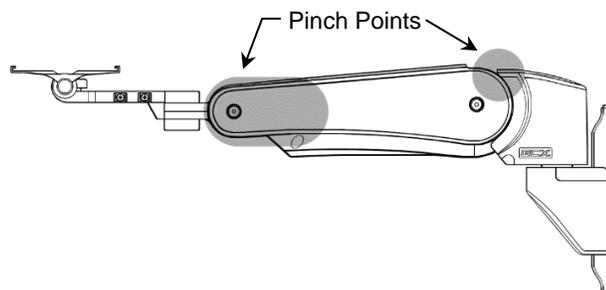

- Remove the device only when the arm is at the highest position. Due to the counterbalance function, the arm will naturally rise to the highest position when weight is removed. This can happen suddenly if the weight is removed at any height other than the highest point.

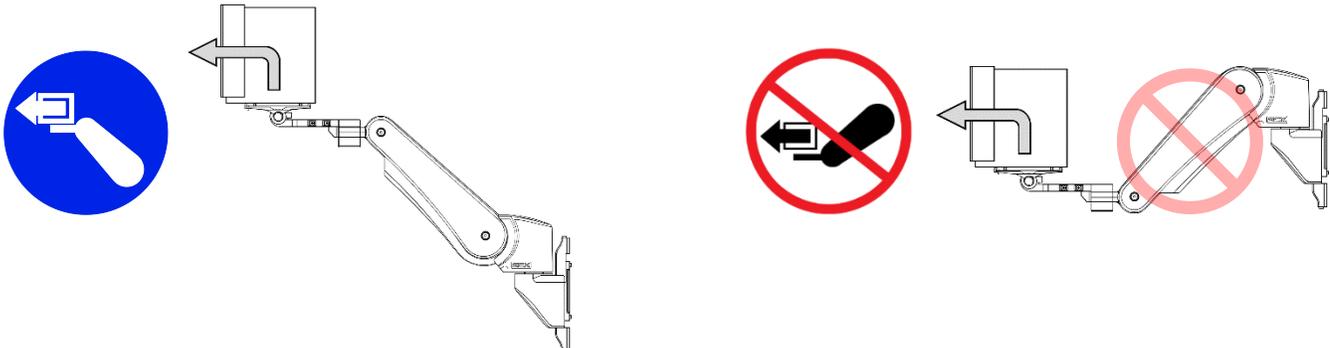

- Due to risk of personal injury or damage to the arm, the arm housing must never be disassembled by non-GCX personnel. Failure to comply will void the warranty.



## 5.0 Installing the Arm in a Channel



Before mounting the arm, verify that the channel has been installed and approved in accordance with the channel installation guide included with the channel.



5.1 Insert the adjustable stop into the top of the channel and slide it to the desired mounting position.

5.2 Using a #3 Phillips screwdriver, tighten the center screw to secure the position of the adjustable stop.

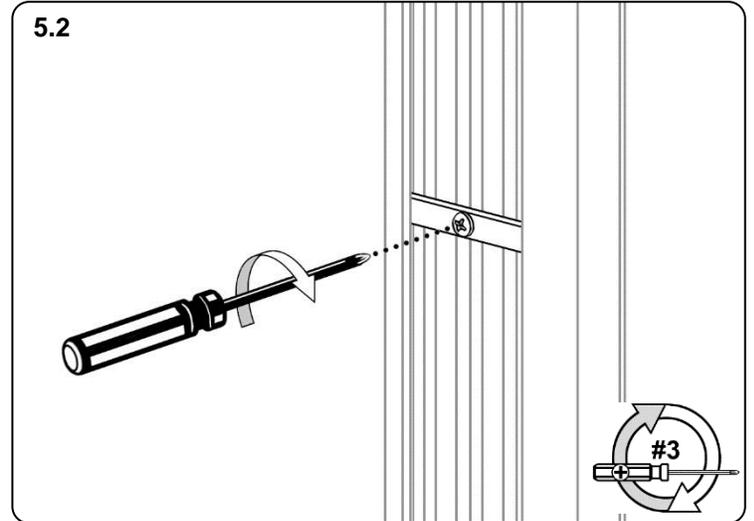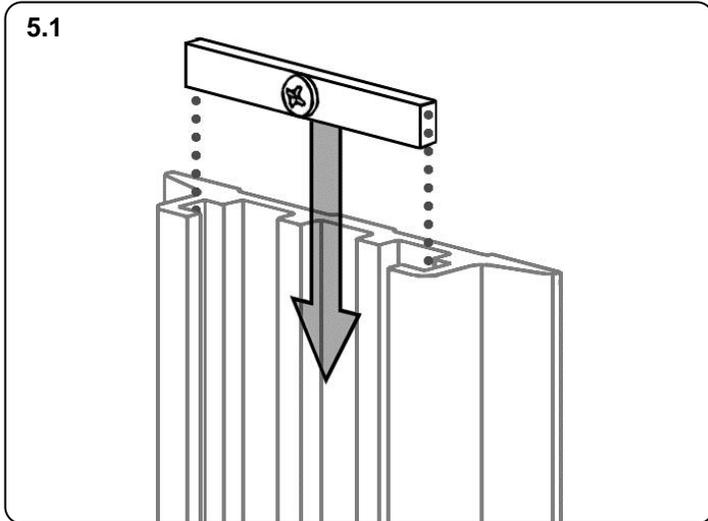

5.3 While supporting the arm, guide the slide into the top of the channel and position it against the adjustable stop.

5.4 Using a 1/8" hex wrench, tighten the (2) upper and the (2) lower set screws in the slide to secure the position of the arm.

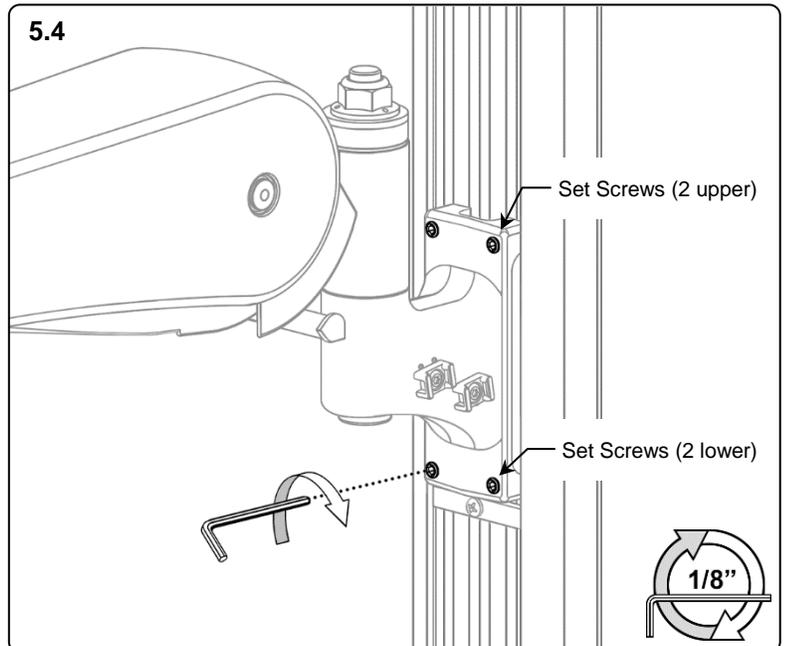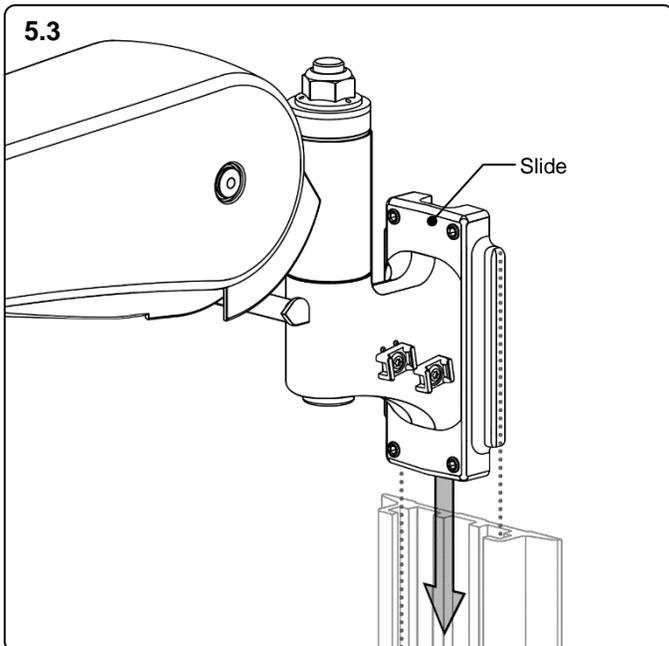


## 6.0 Mounting the Device to the Arm

### 6.1 Reorienting the Mounting Plate

To install a back-mounted device the arm's mounting plate must first be reoriented. Follow this section to reorient the mounting plate. For a bottom mounted device skip to Section 6.2


#### 6.1.1

Use a 1/8" hex wrench to remove the tilt stop and screw.

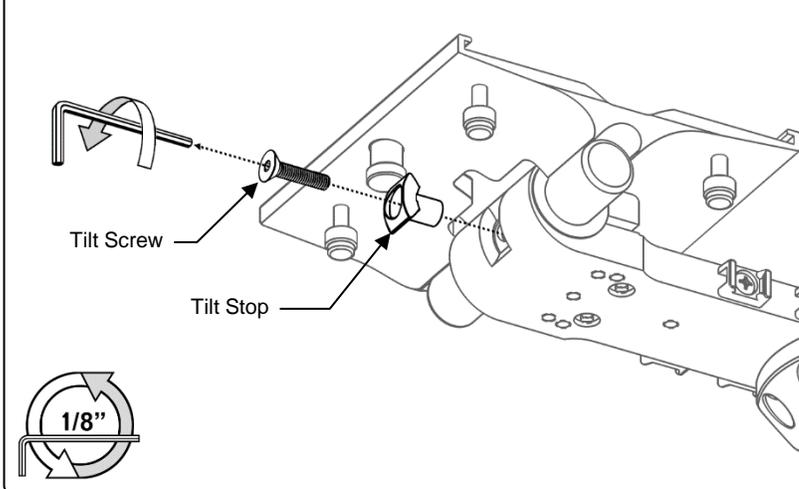

#### 6.1.2


Rotate the mounting plate to a vertical orientation and install the tilt adjustment lever.

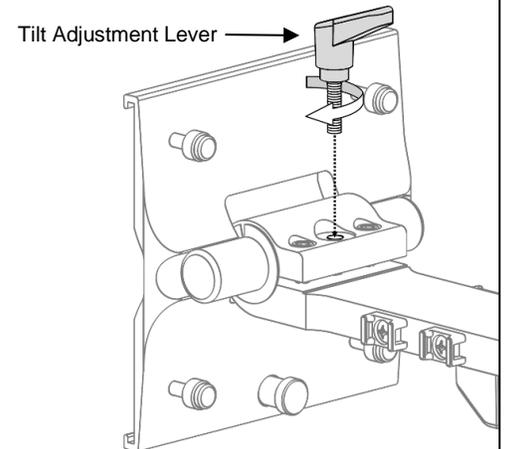

### 6.2 Mounting the Device

**Installation Note:** Most devices will require the attachment of a device-specific mounting adapter provided by GCX. Attach the mounting adapter in accordance with the instructions provided with the mounting adapter.

Pull the spring plunger at the front of the mounting plate and slide the device's mounting adapter laterally into the mounting plate until the plunger snaps into the clearance hole at the front of the mounting adapter. After the device is installed, tighten the (4) nylon locking screws to secure the device in position.

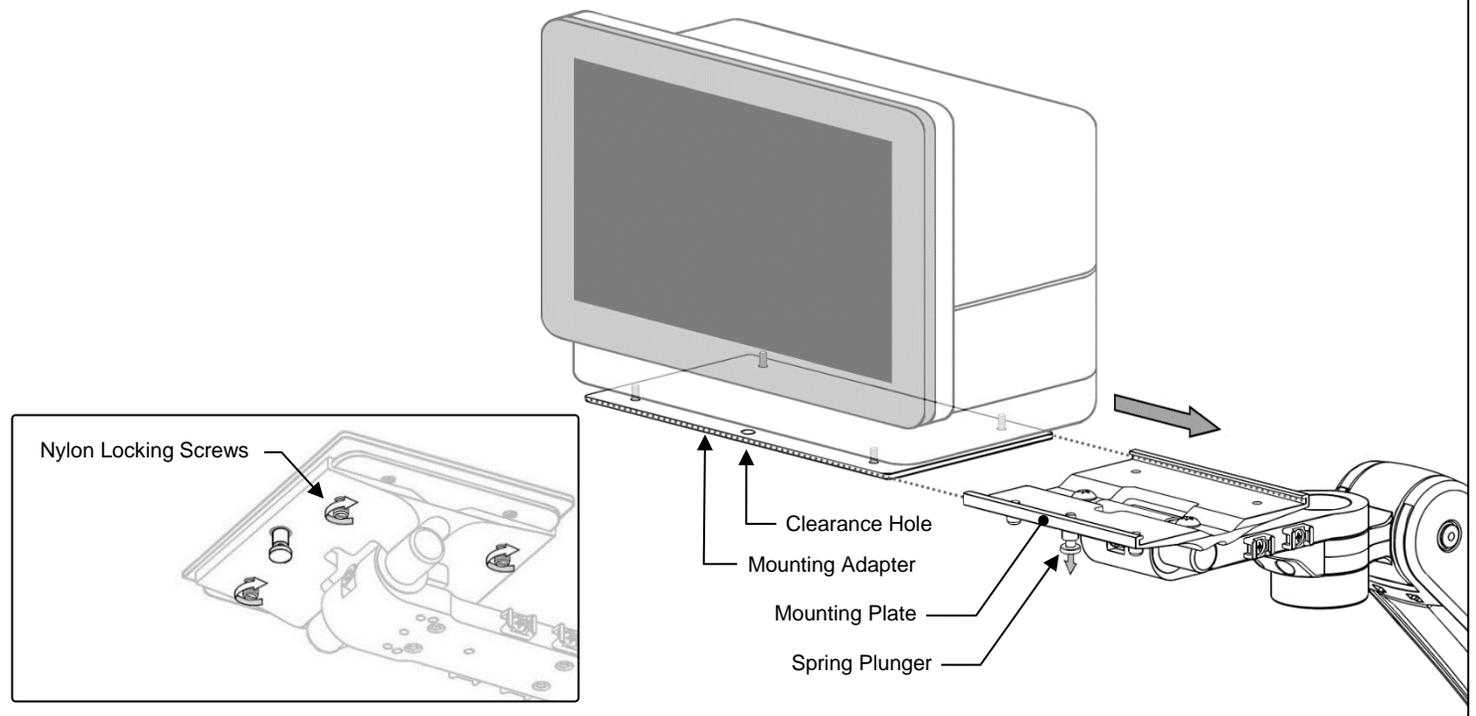

## 7.0 Adjusting the Arm


**Installation Note:** Adjustments are factory pre-set. However, adjustments to counterbalance, tilt tension, swivel tension and pivot tension may all be required to achieve optimal arm performance. Detailed instructions for making adjustments follow in the section below. When properly adjusted, the mounted device will move easily throughout the arms full range of motion. Refer to the Routine Maintenance Check List in Section 9 for a quick guide to these functional checks.



**Caution:** The counterbalance procedure is performed with the device(s) and accessories installed on the arm. Use caution as it may be necessary to support the weight of the arm/mounted device(s). This process may require an additional person. Use caution while performing this procedure.

### 7.1 Counterbalance Adjustment

Move the arm to a level horizontal position and use a 5/32" hex wrench to increase (+) or decrease (-) the counterbalance setting. Counterbalance is correctly set when the arm can be moved up or down with minimal force and the arm does not float up or down after releasing the arm.


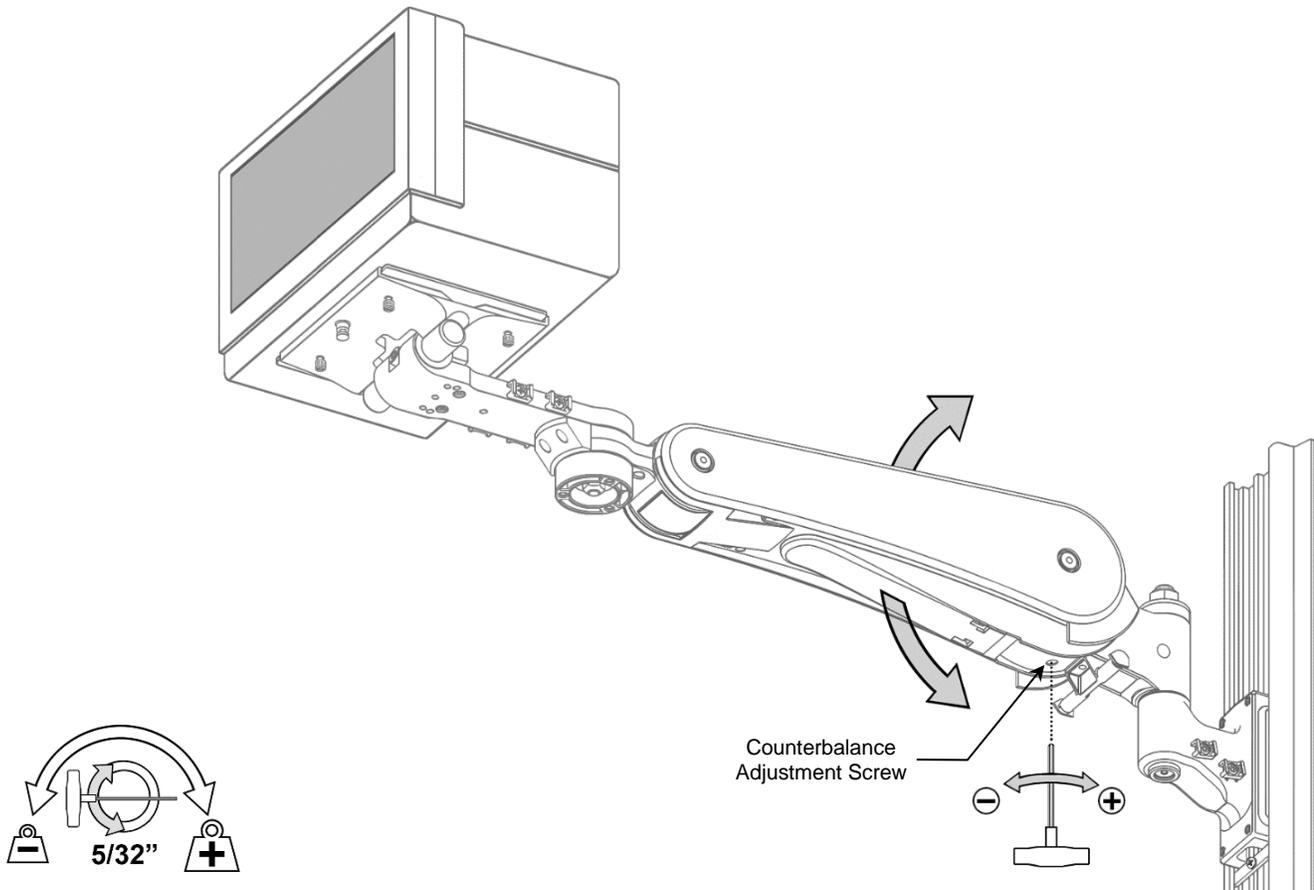


### 7.2A Tilt Tension Adjustment (Horizontal Mounting Plate)

**Note:** Tension is factory pre-set to accommodate many installations and therefore may not need adjustment.

Adjustment is performed with the device removed. Re-mount the device and check tilt tension after adjustment is made, repeat as necessary.

**Caution:** Do not remove the adjustment screws.


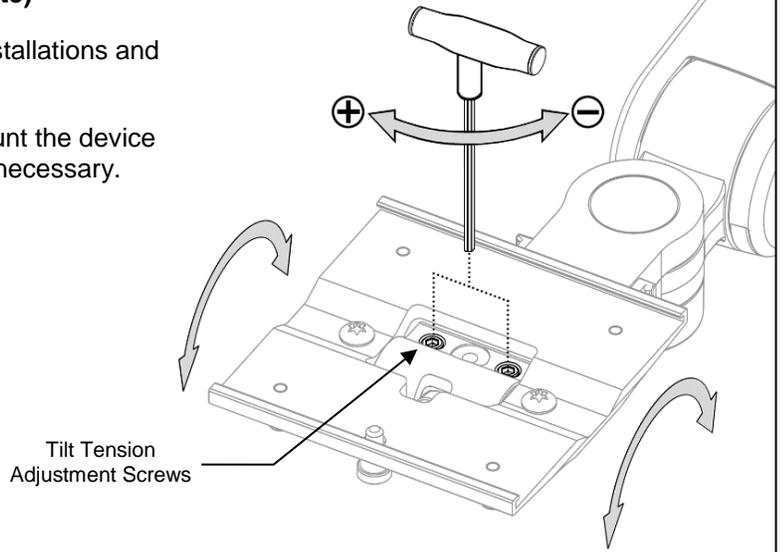
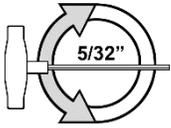


### 7.2B Tilt Tension Adjustment (Vertical Mounting Plate)

**Note:** Tension is factory pre-set to accommodate many installations and therefore may not need adjustment.

Adjustment is performed with the device installed.


**Caution:** Do not remove the adjustment screws.

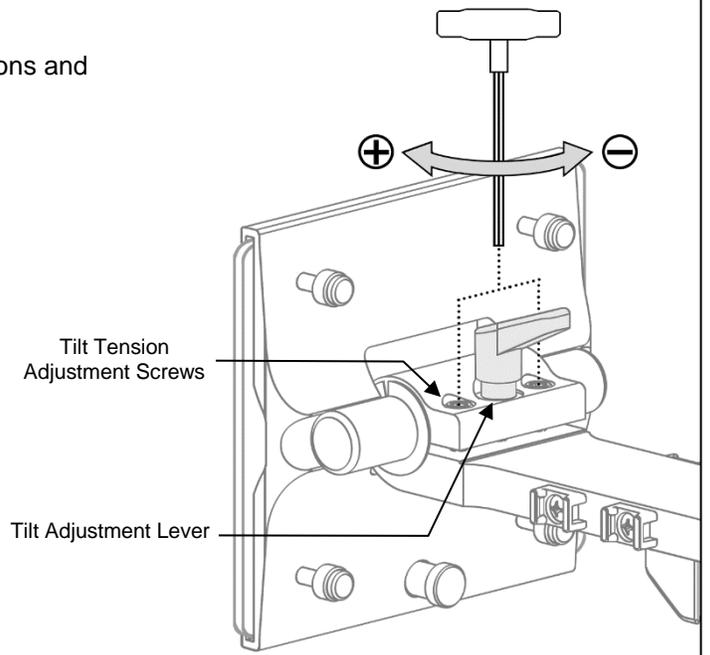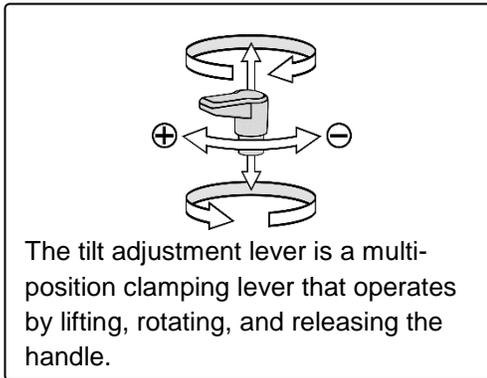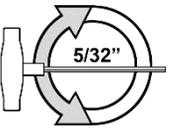

### 7.3 Swivel Tension Adjustment

**Note:** Tension is factory pre-set to accommodate many installations and therefore may not need adjustment.


**Caution:** Do not remove the swivel tension nut.

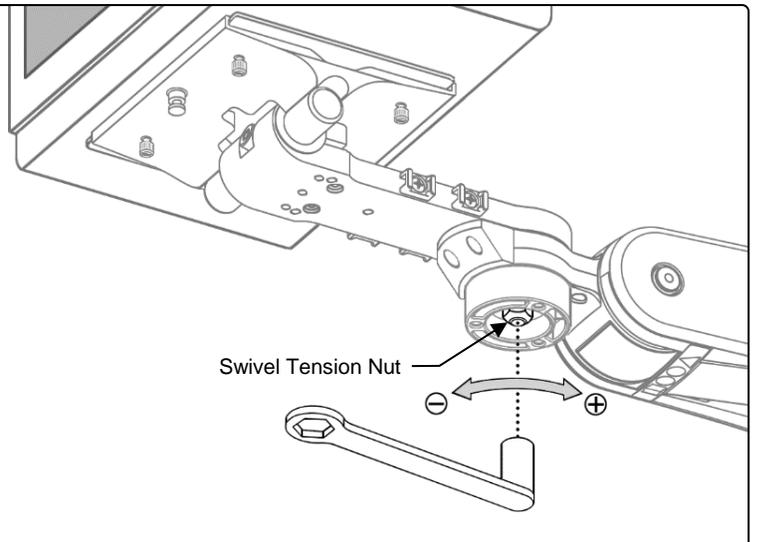

#### 7.4 Pivot Tension Adjustment

**Note:** Tension is factory pre-set to accommodate many installations and therefore may not need adjustment.

**Caution:** Do not remove the pivot tension nut.

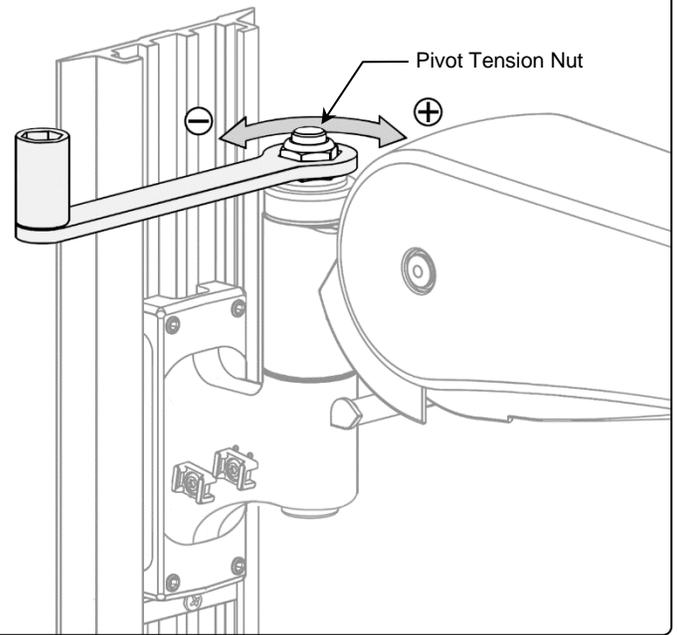


## 8.0 Cable Management

**Attention:** Follow these cable routing instructions to ensure unrestricted arm range of motion and to avoid possible damage to cables or the mounted device.

**Cable Management Overview:** Cables are routed from the device connections through the arm and into the channel. Cable service loops (cable slack) are established to prevent restricting range of motion when the arm or mounted device is repositioned. Cables are anchored at the front of the arm to preserve the necessary service loops. Cable sleeves are provided to assist in determining the correct service loops and to help protect and organize the cables. Cables routed through the arm are obscured from view and protected by plastic covers.


### 8.1 Preparing Service Loop 1

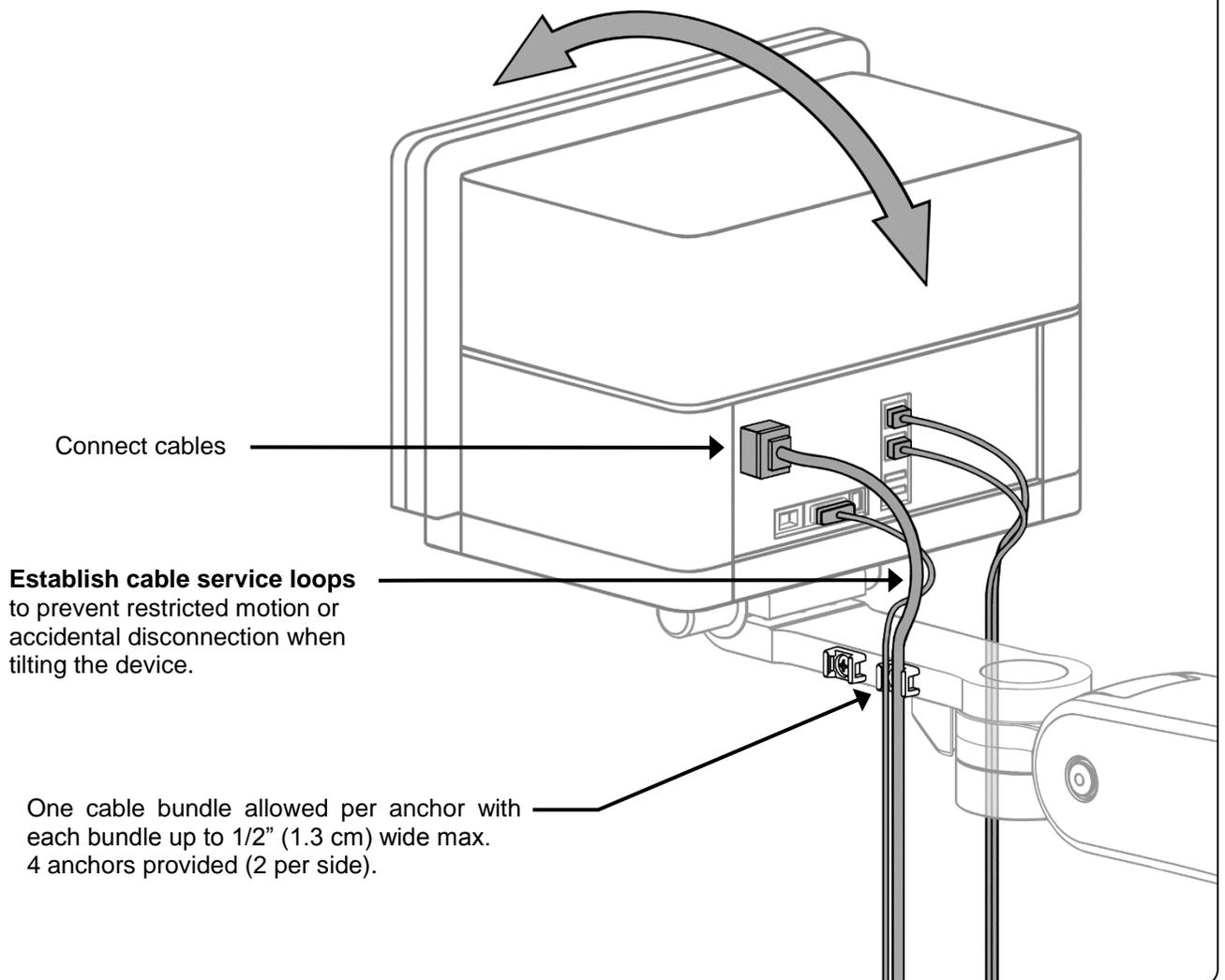

## 8.2 Setting Service Loop 1

Using the provided cable sleeves is recommended but optional.

Insert cables into the sleeves as shown below.

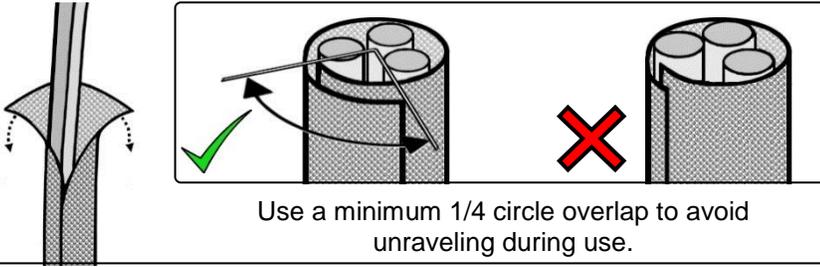

Use a minimum 1/4 circle overlap to avoid unraveling during use.



If using cable sleeves different than those provided with the arm then cut the sleeves to 30-1/2" (77.5 cm) in length and verify function within the arm prior to final product installation.

**Position cable tie heads flush against the arm while tightening.**

The provided cable ties have a release tab at the head.

Pliers may be used to help pull ties taut to secure cable bundles.


Clip excess length of cable ties as flush as possible to avoid creation of a sharp edge.

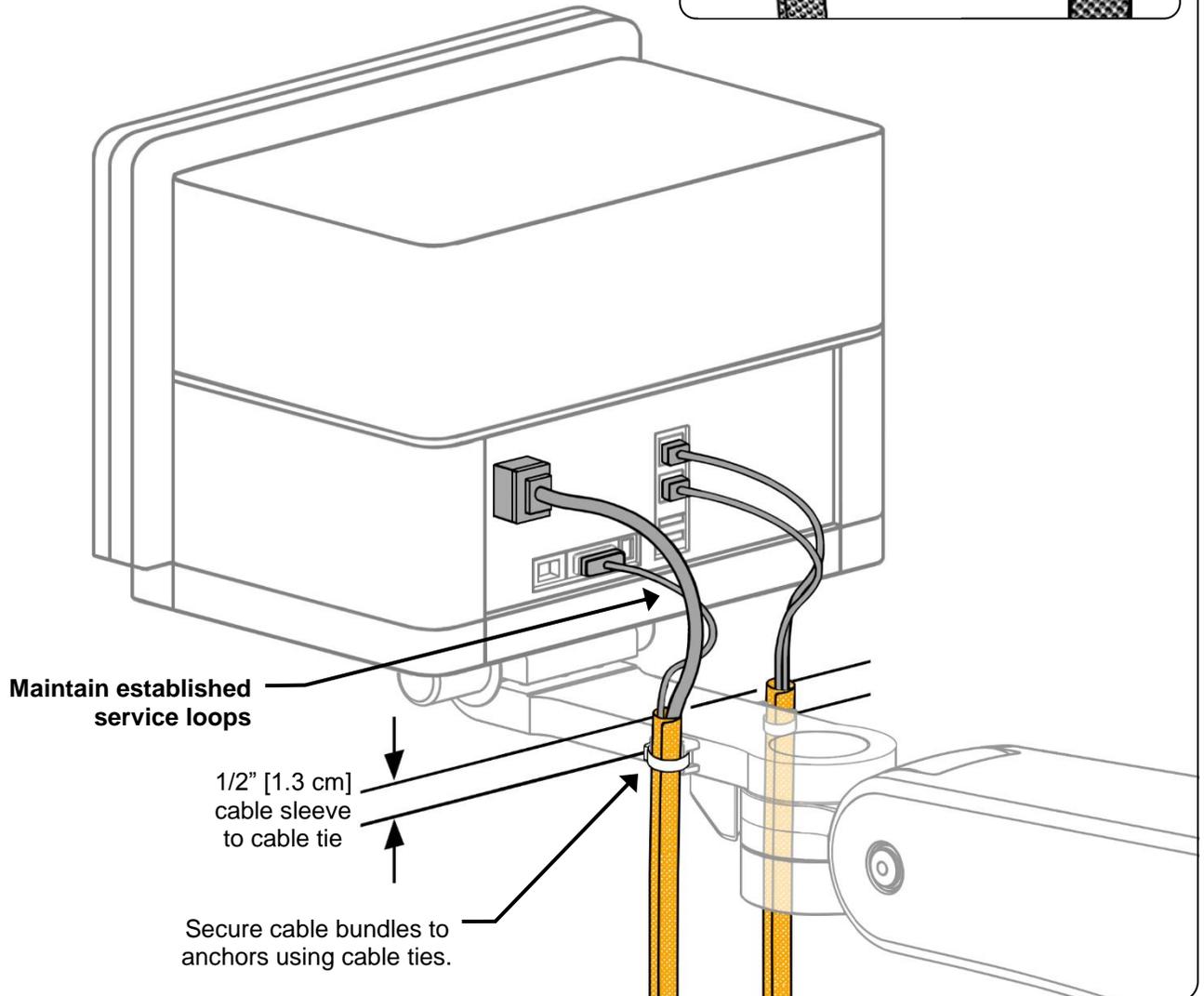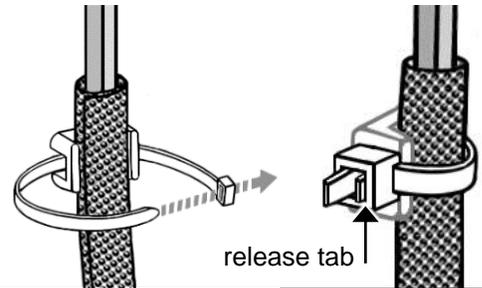

### 8.3 Preparing Service Loop 2


**8.3.1** If not using cable sleeves then use this section to establish service loop 2. If using sleeves then skip to the next section.

**NOTE:** By using and positioning cable sleeves as recommended, the necessary length of service loop 2 will automatically be set by the length of the sleeves.

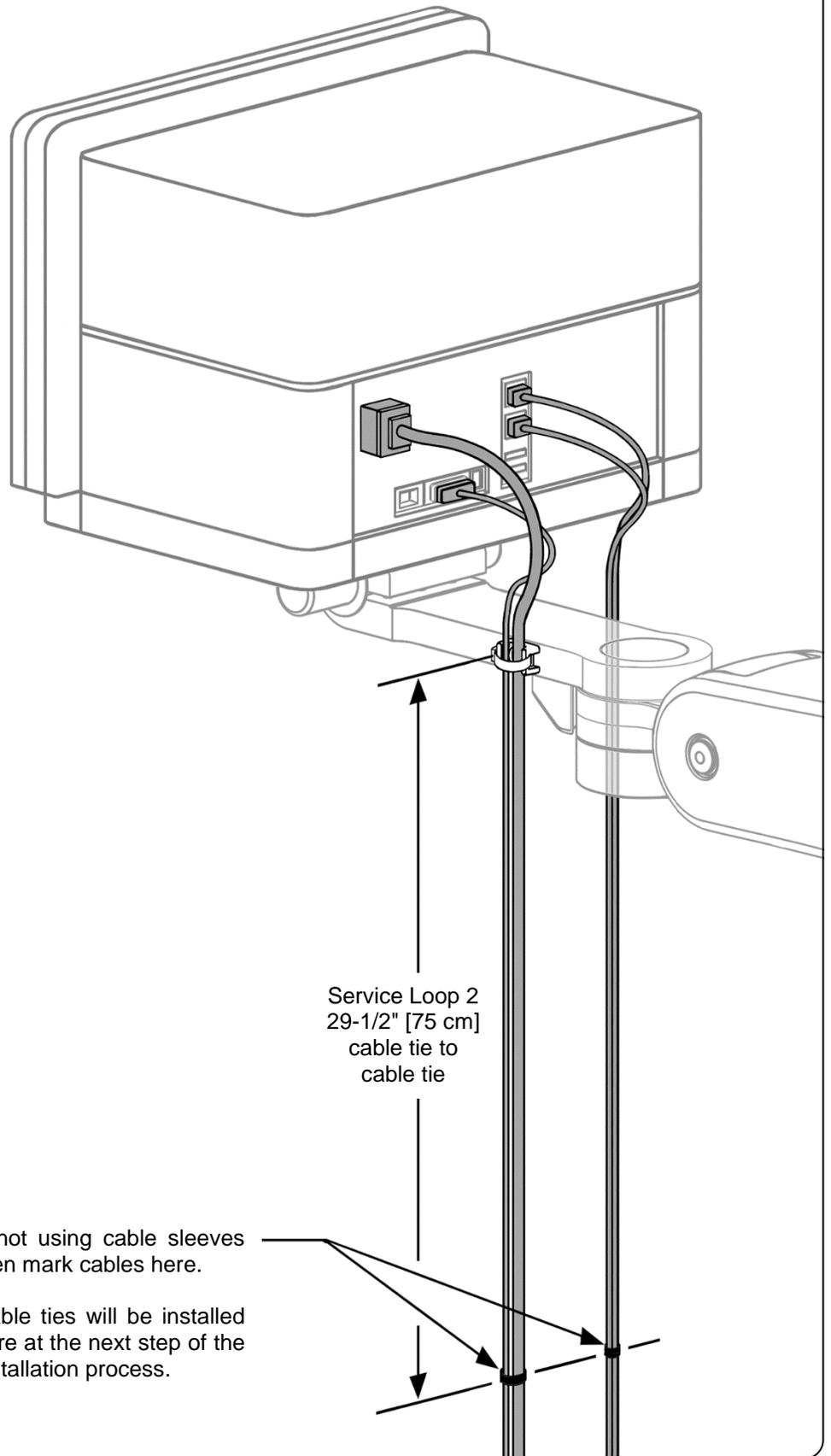

## 8.4 Setting Service Loop 2



**8.4.1 Maintain the established 29-1/2" (75 cm) service loop** between anchor points and anchor the cables (and sleeves) at the rear of the arm using cable ties. Secure only one cable bundle, each a maximum 1/2" (1.3 cm) wide, per anchor. Cable anchors further from the channel should be used prior to the anchors closer to the channel.

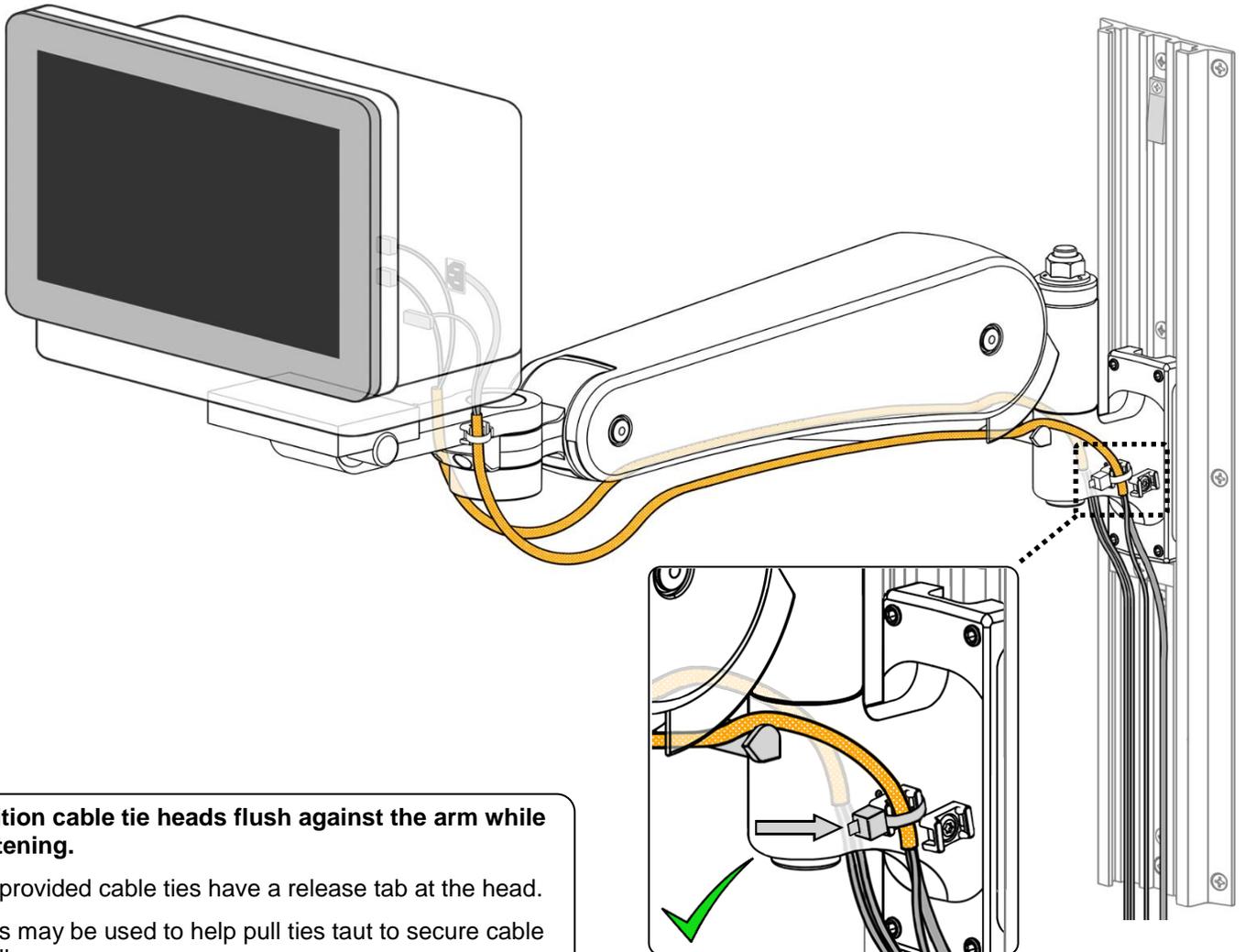





**Position cable tie heads flush against the arm while tightening.**

The provided cable ties have a release tab at the head. Pliers may be used to help pull ties taut to secure cable bundles.

Clip excess length of cable ties as flush as possible to avoid creation of a sharp edge.

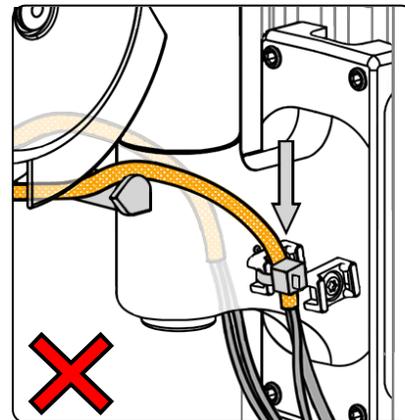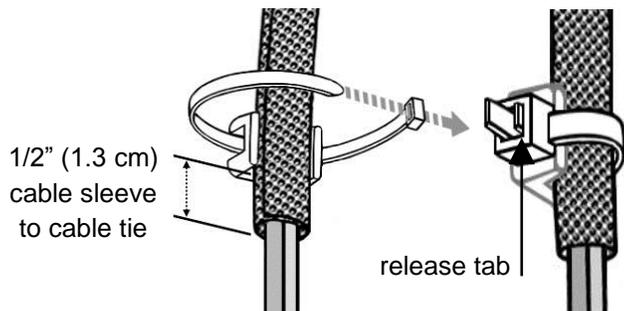

## 8.5 Installing the Cable Cover

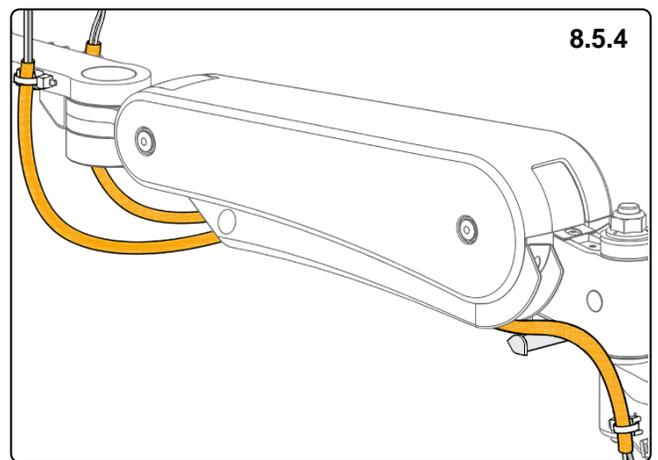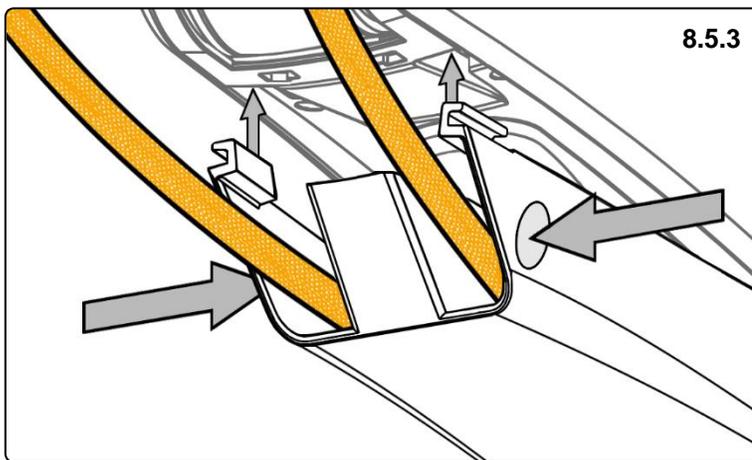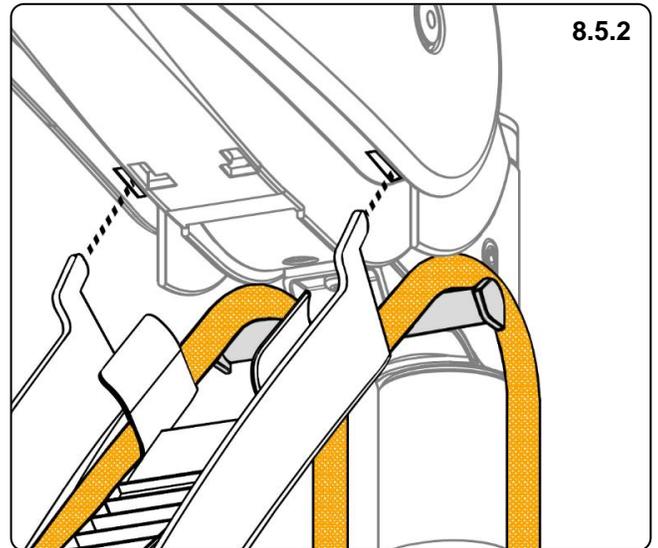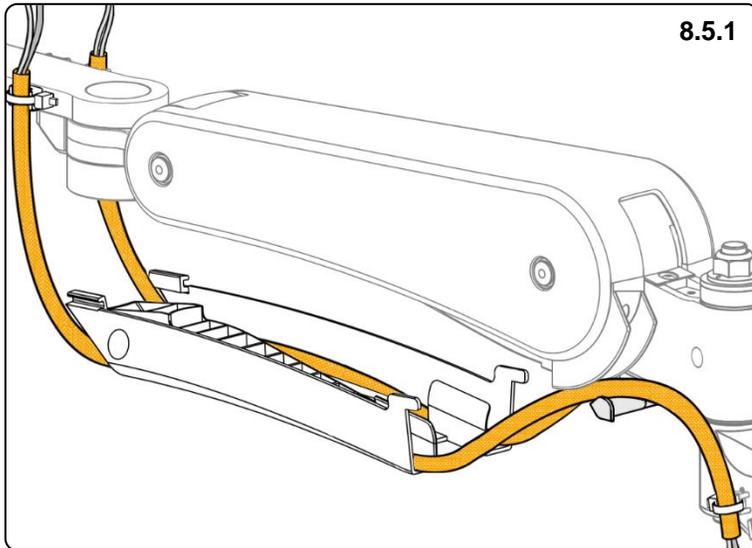

## 8.6 Routing Cables Upward into the Channel


**8.6.1** If all cables will route downward into the channel below the arm then simply let them hang from the rear cable anchors and proceed to the next section.




**8.6.2. For cables routing upwards into the channel above the arm.**

**8.6.2.1** First wrap each cable bundle from the downward-going side, underneath the slide, and then back upward along the opposite (upward-going) side.


**8.6.2.2** On the upward-going side, secure the bundle in the unused cable anchor closest to the channel.

**ATTENTION:** If both cable anchors on the upward-going side are being used then it is not recommended to route cables upward on that side. Attempting to do so may prevent the slide covers from proper fit and function.



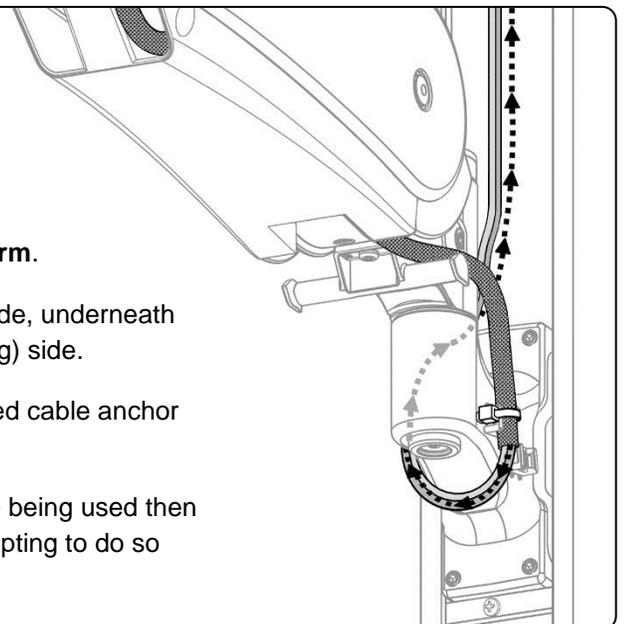


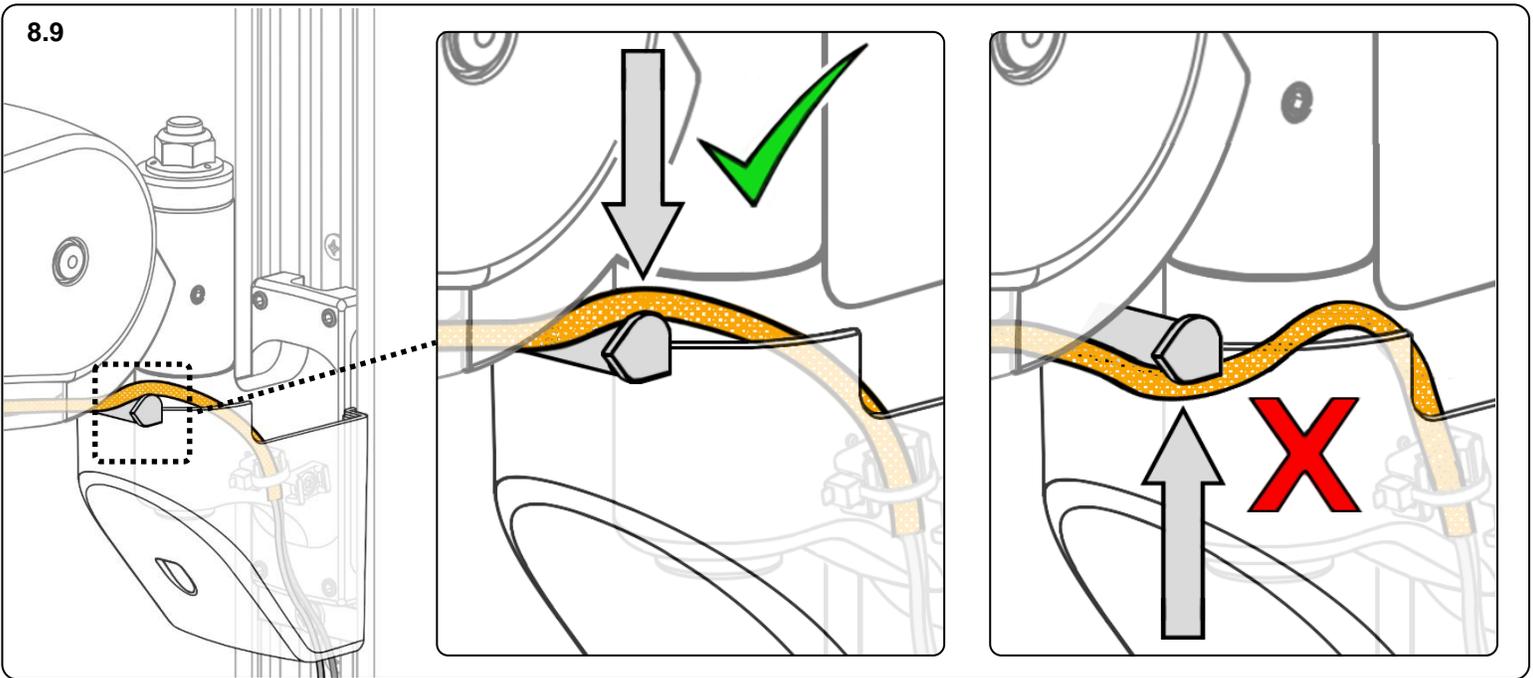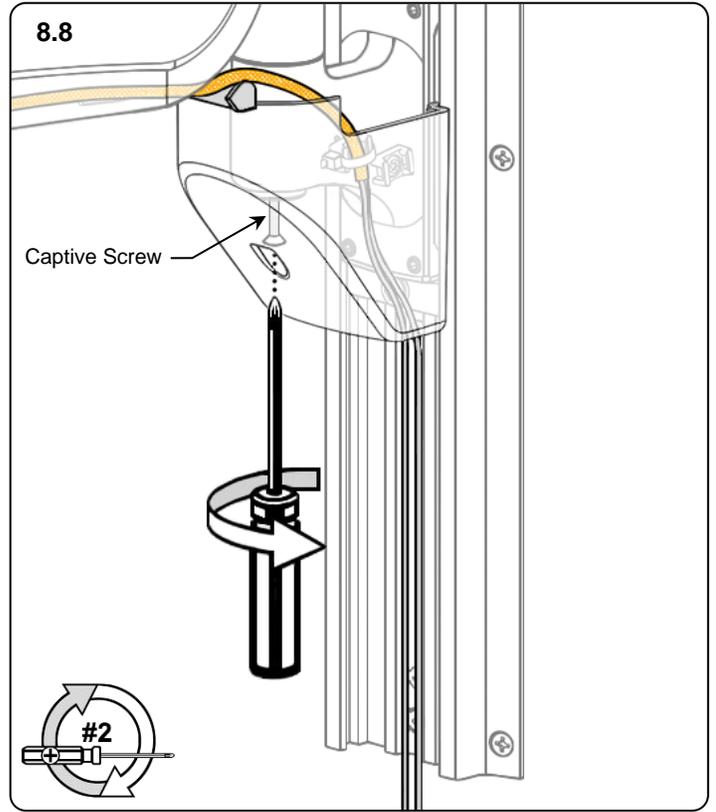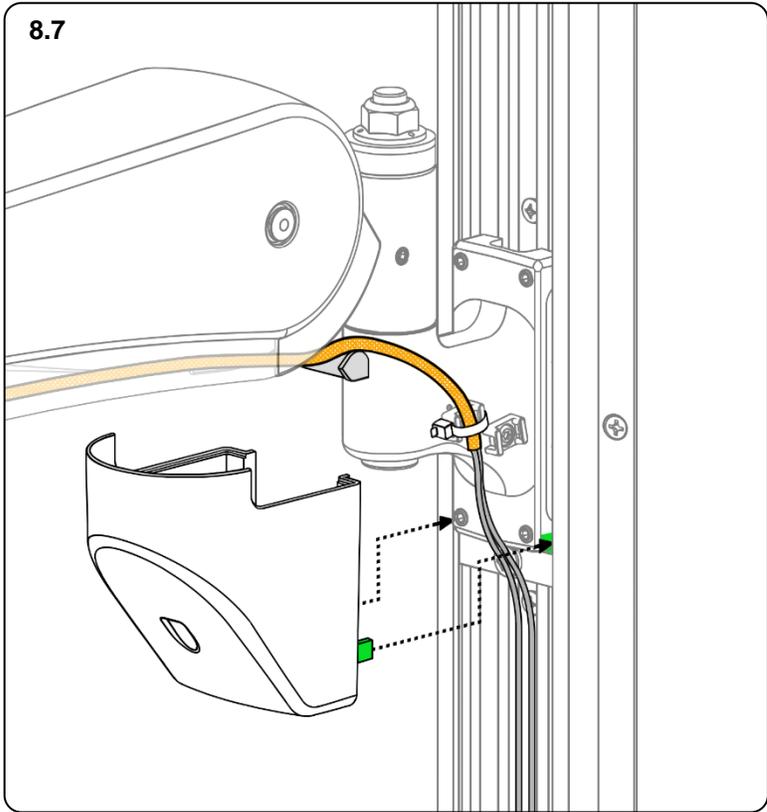


8.10

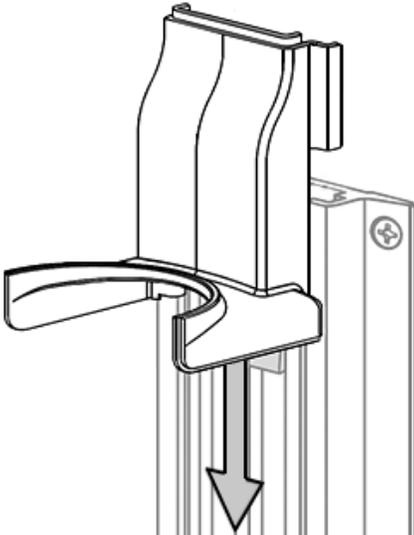




8.11

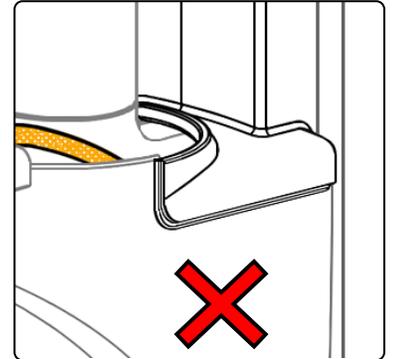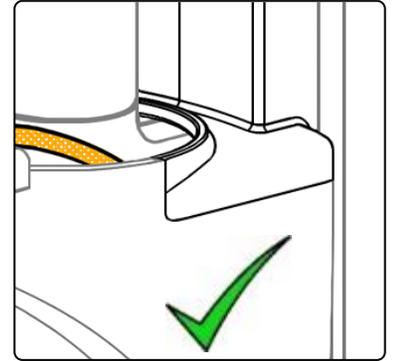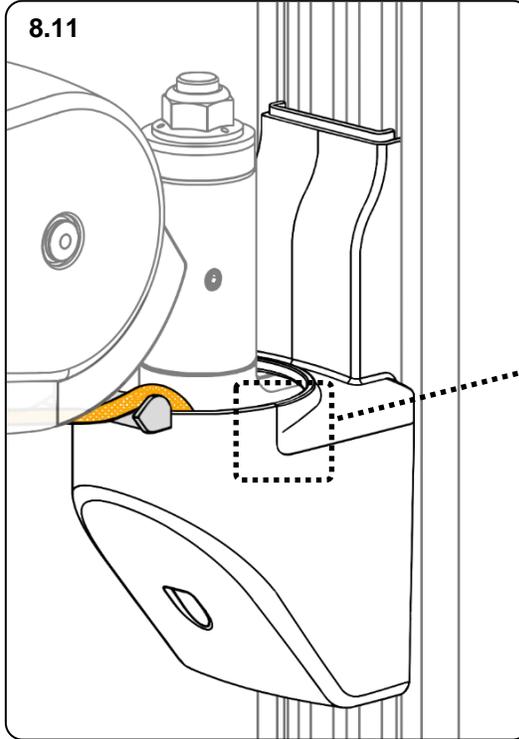



8.12

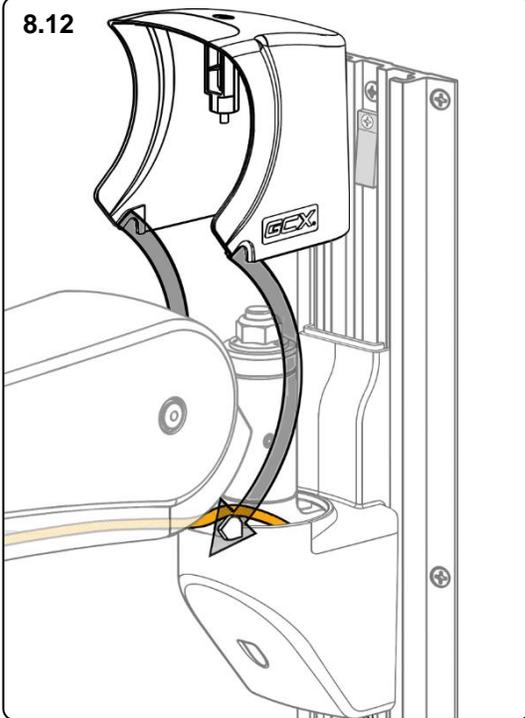


8.13

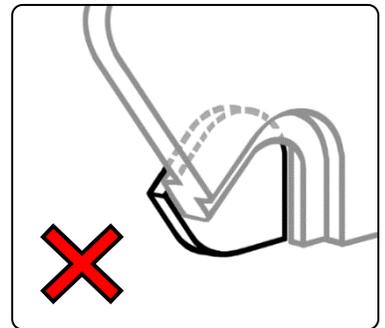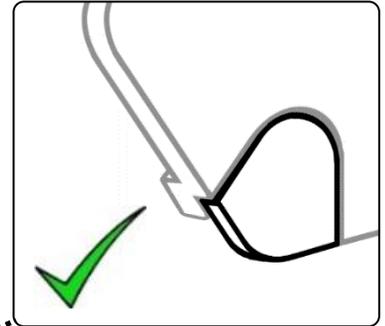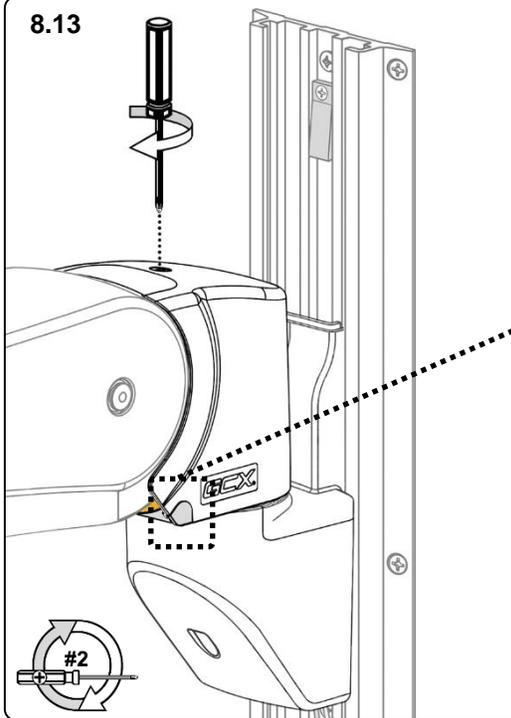


8.14

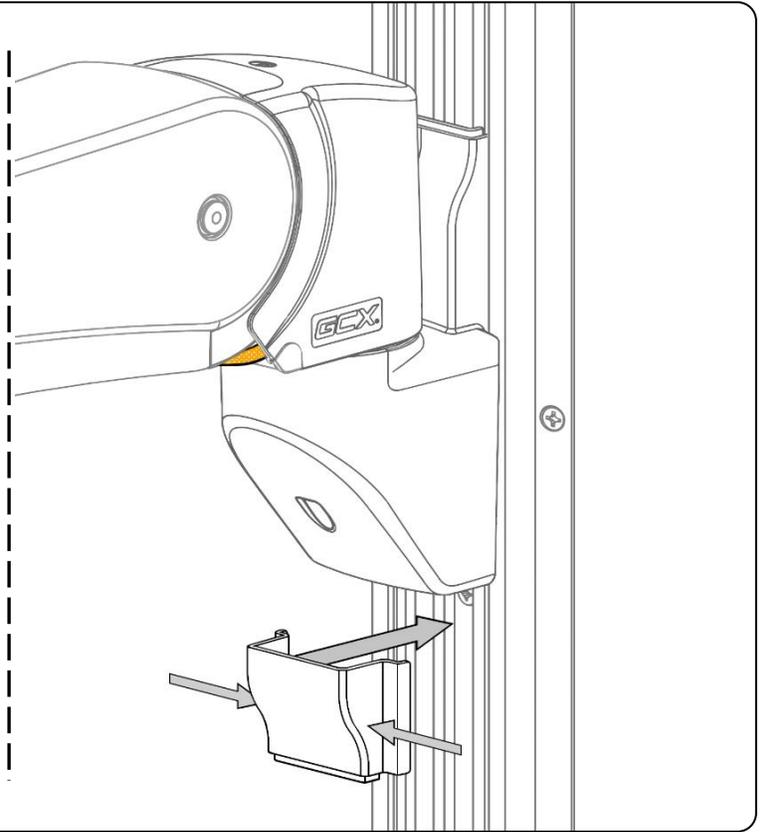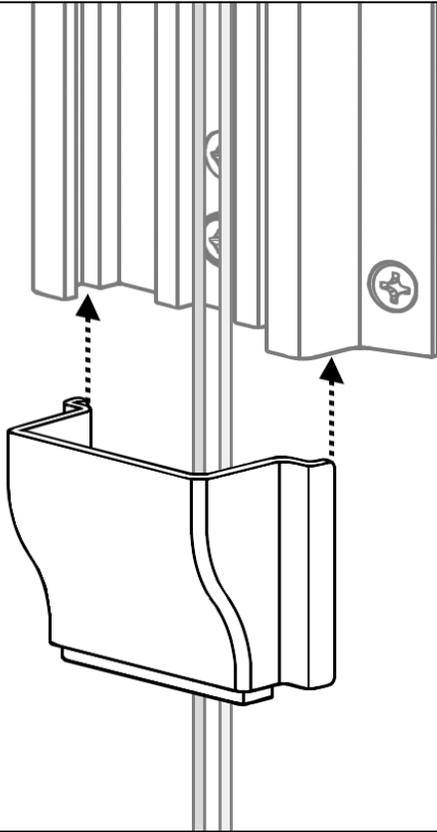

8.15

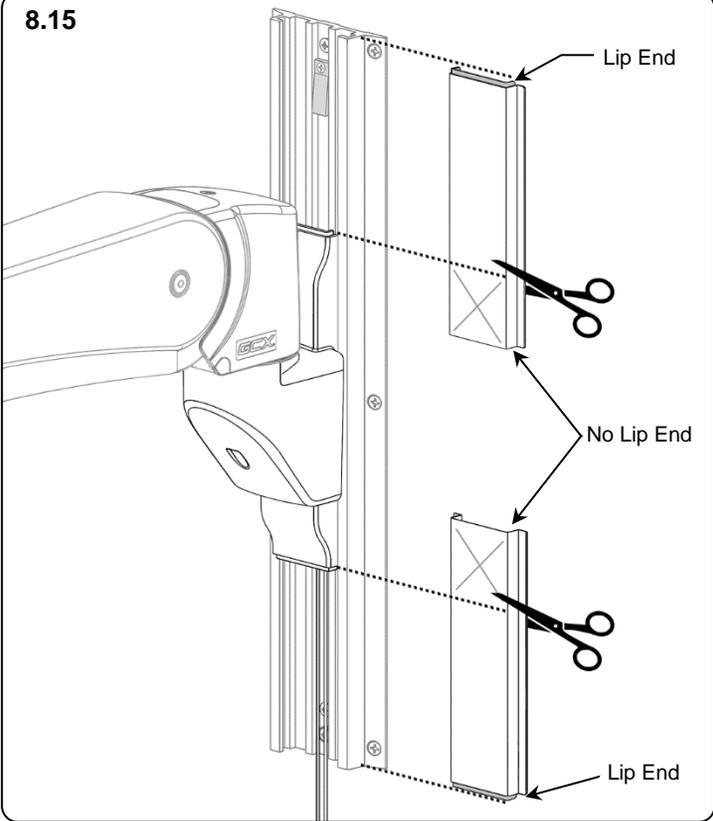


8.16

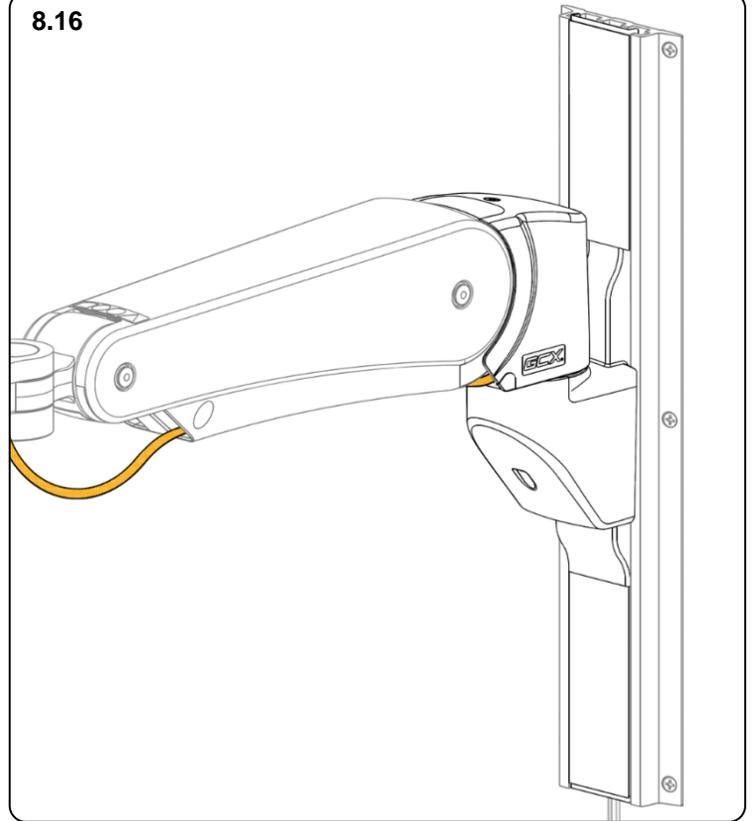

## 9.0 Routine Maintenance

**The arm must be inspected and maintained at least once a year.**

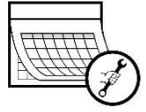

✓	Routine Maintenance Check List	Section
<b>Counterbalance</b>	With the device mounted, move the arm through its entire vertical range of motion. The load should maintain its position at every point in the travel of the arm. If necessary, the counterbalance mechanism may be adjusted.	7.1
<b>Tilt</b>	Grasp the mounted device and tilt it forward and back, through its entire range of motion. There should be enough tension or resistance in the tilt mechanism to prevent the device from tilting forward unexpectedly when in use. If necessary, the tilt tension may be adjusted.	7.2
<b>Swivel</b>	Grasp the mounted device and swivel it from side to side at the front swivel joint. The device should swivel with some tension or resistance, not loosely. If necessary, the swivel tension may be adjusted.	7.3
<b>Pivot</b>	Grasp the arm and pivot it from side to side at the rear attachment. The arm should pivot with some tension or resistance, not loosely. If necessary, the pivot tension may be adjusted.	7.4
<b>Hardware</b>	Inspect fasteners for looseness. Tighten as required for optimal operation and safety.	5.2,5.4, 6.2
<b>Cable Management</b>	Inspect cable management and ensure proper anchorage, service loops, and path routing; adjust as needed. Inspect cables and sleeves for damage and replace as needed.	8.0

## 10.0 Cleaning the Arm

The arm may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.

Steel wool or other abrasive material should never be used.

Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing any cleaning solution on a small area of the arm that is not visible, to verify compatibility.

Never submerge or allow liquids to enter the arm. Wipe any cleaning agents off of the arm immediately using a water-dampened cloth. Dry the arm thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

## 11.0 Troubleshooting the Arm

Symptom	Possible Cause	Remedy
Mounted device does not appear level or parallel to the floor.	Channel not plumb. Check with level.	Adjust channel to plumb, or reinstall channel.
	Weight of device not compatible with load rating of the arm.	Mount device on arm with compatible load rating (section 3.0)
	Swivel hardware loose.	Adjust swivel tension (section 7.3).
	Pivot hardware loose.	Adjust pivot tension (section 7.4).
	Mounting surface (e.g. wall, side of anesthesia machine, etc.) not structurally sound (does not hold mounting hardware).	Mounting surface must be reinforced or channel must be relocated.
	Channel loose at mounting surface.	Check for plumb and tighten, or relocate (reinstall) channel.
Mounted device drifts up or down.	Arm not counterbalanced correctly for weight of the device.	Perform counterbalance adjustment (section 7.1).
	Weight of mounted device not compatible with load rating of arm.	Use arm with compatible load rating (section 3.0).
Mounted device is difficult to move up or down.	Arm not counterbalanced correctly for weight of mounted device.	Perform counterbalance adjustment (section 7.1).
Arm pivots too freely.	Pivot tension is too loose.	Adjust pivot tension (section 7.4).
Arm is difficult to pivot.	Pivot tension is too tight.	
Device swivels too freely.	Swivel tension is too loose.	Adjust swivel tension (section 7.3).
Device is difficult to swivel.	Swivel tension is too tight.	
Device is difficult to tilt.	Tilt tension is too tight.	Adjust tilt tension (section 7.2).
Device will not maintain tilt position.	Tilt tension is too loose.	
Arm inadvertently slides down Channel.	Set screws (4) in slide are loose.	Reposition arm, tighten set screws in slide, and secure adjustable stop (section 5.2 and 5.4).
	Adjustable stop is loose or missing.	Install or secure adjustable stop (section 5.2).
Arm motion feels restricted or limited.	Improper cable routing is causing interference and/or cable binding.	Inspect cable routing and ensure proper anchorage, service loops, and path routing; Adjust as needed (section 8).