

## Installation Guide

### Zoe Medical PPM2/PPM3 Flush Wall Mount Kit


The purpose of this guide is to:

- Describe the attachment of the 6" Wire Chart Basket to 19" Wall Channel.
- Describe the attachment of Zoe PPM2 Mounting Adapter to 19" Wall Channel.
- Describe the attachment of Nightingale PPM3 Patient Monitor to ZOE PPM2 Mounting Adaptor.

#### Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown).


Item #	Description	Qty
1	19" Wall Channel	1
2	6" Wire Chart Basket with Molded Offset Slide	1
3	Zoe PPM2 Mounting Adapter with Molded Offset Slide	1
4	1/8 Short Arm Hex Wrench	1

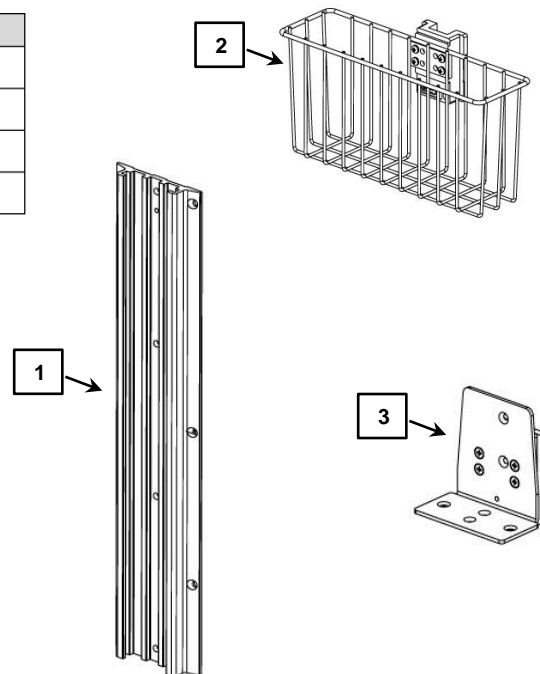

**Tools Required:** Phillips screwdriver (not provided)  
1/8 Short Arm Hex Wrench (provided)


### Installing 6" Wire Chart Basket to 19" Wall Channel

Slide Molded Offset Slide into the top of the 19" Wall Channel. Place the Molded Offset Slide to desired height, then tighten set screw found on the bottom right of Molded Offset Slide with a 1/8 Short Arm Hex Wrench. Turn set screw clockwise to tighten.

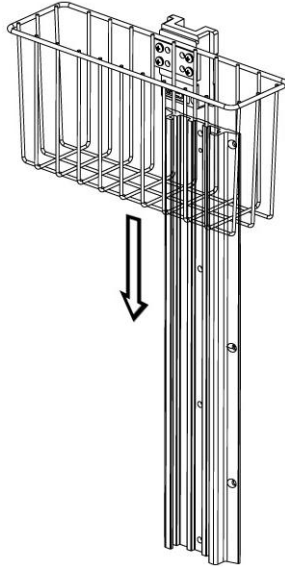


### Installing Zoe PPM2 Mounting Adapter to 19" Wall Channel

Slide Zoe PPM2 Mounting Adapter into the top of the 19" Wall Channel, slide to desired height. Tighten set screw found on the bottom right of Molded Offset Slide with a 1/8 Short Arm Hex Wrench. Turn set screw clockwise to tighten.

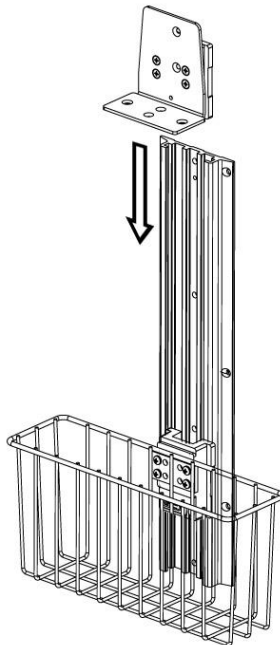

## Installing Nightingale PPM3 Patient Monitor to ZOE PPM2 Mounting Adapter

Line up the Nightingale PPM3 Patient Monitor with previously mounted ZOE PPM2 Mounting Adapter. The mounting area on the back of the Nightingale PPM3 Patient Monitor will slide directly onto the ZOE PPM2 Mounting Adapter.

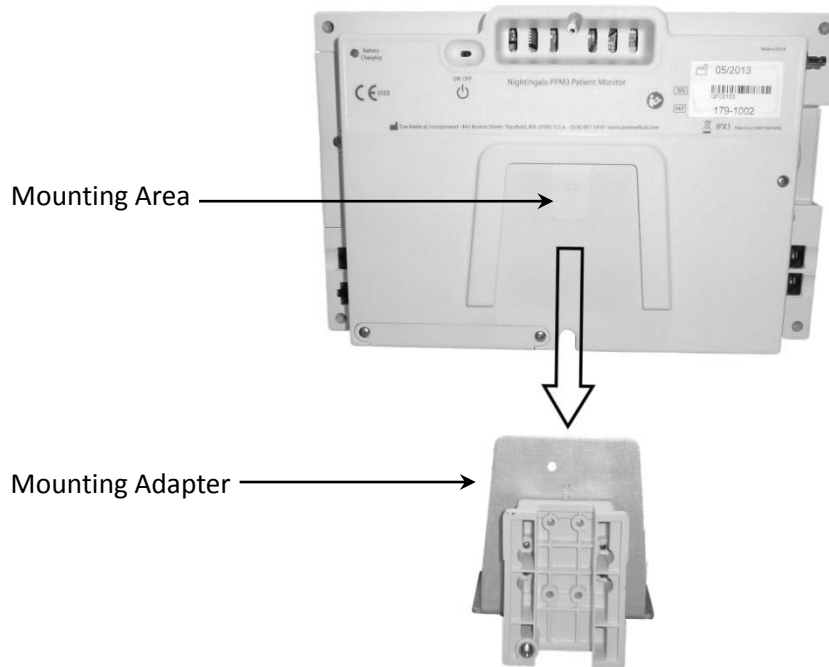

## Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

## Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the arm that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the arm immediately, using water dampened cloth. Dry the arm thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products before installing the arm in the Channel.