

Installation Guide

Top Plate with 10" Channel for Datex Ohmeda Aisys Anesthesia Machine

The purpose of this guide is to describe attachment of Top Plate to top of machine.

Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts; hardware not shown):


Item #	Description	Qty
1	Top Plate/10" Channel Assembly	1
2	M5 x 12mm Flat Head Machine Screw (FHMS)	4



Tools Required: Phillips screwdriver.

Attaching Top Plate to Top of Anesthesia Machine

1. Locate the four (4) threaded inserts shown in photo below left. **Installation Note:** It may be necessary to remove hole covers to expose the threaded inserts.

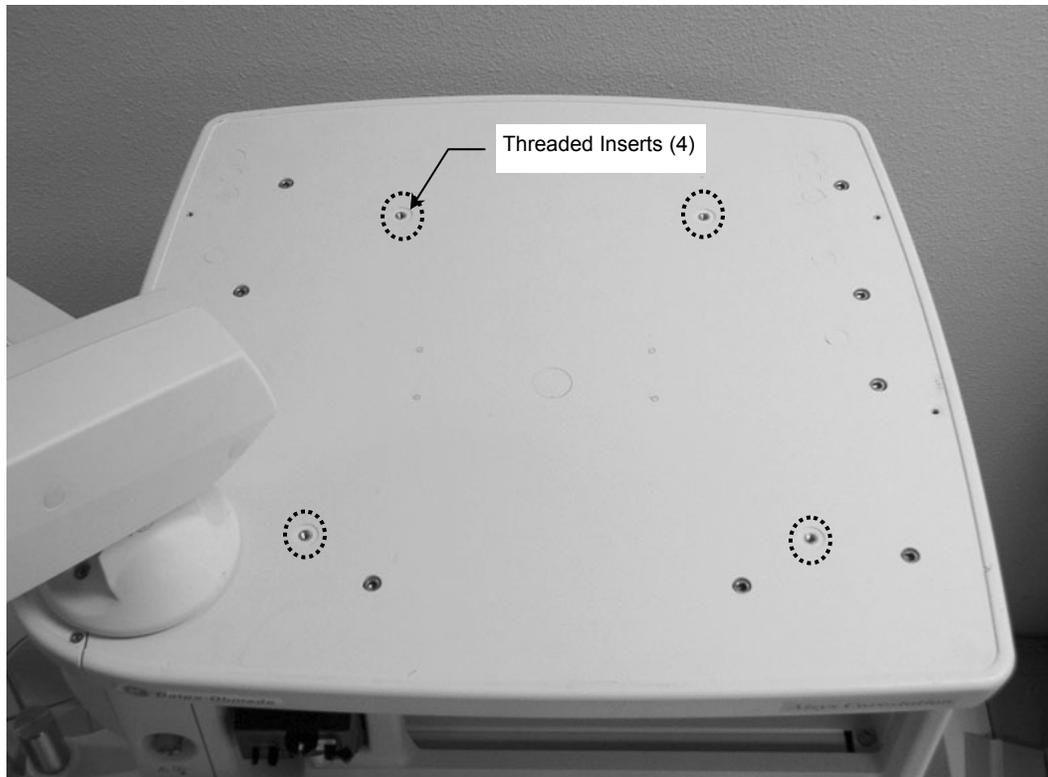

- Align four (4) mounting holes in Top Plate with corresponding threaded inserts in top of machine. Fasten Top Plate to top of machine with four (4) M5 x 12mm FHMS.



- Mount instruments in Channel according to instrument-specific instructions.

