

## OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION


APPLICATION FOR OSHPD PREAPPROVAL	OFFICE USE ONLY			
OF MANUFACTURER'S CERTIFICATION (OPM)	APPLICATION #: OPM-0165-13			
OSHPD Preapproval of Manufacturer's Certification (OPM)				
Type: <ul> <li>New</li> <li>Renewal</li> <li>Update to Pre-CBC 2013 OPA Number:</li> <li></li></ul>				
Manufacturer Information				
Manufacturer: GCX Corporation				
Manufacturer's Technical Representative: Rob Glaser				
Mailing Address:3875 Cypress Drive, Petaluma, CA. 94954-5635				
Telephone: (800) 228-2555 Email: rglaser@gcx.com				
Product Information	MAP L			
Product Name: GCX Wall Mount with M & VHM Arms	1 E			
Product Type: Cantilever	G			
Product Model Number:13", 19", 25", 31", 37" & 49" Wall Channels	w/M, VHM & VHM25 Arms			
General Description:				
DATE: 01/31/2017				
	N			
Applicant Information	CODE!			
Applicant Company Name: EASE Co.				
Contact Person: Jonathan Roberson, S.E.				
Mailing Address:5877 Pine Ave. Suite 210, Chino Hills, CA. 91709				
Telephone: (909) 606-7622 Email: J.Robe	rson@EASECo.com			
I hereby agree to reimburse the Office of Statewide Health F accordance with the California Administrative Code, 2016.	Planning and Development review fees in			
Signature of Applicant:	Date: 5/3/16			
Title:   Principal Engineer   Company Name:   EASE	Со.			
"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs" STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY OSH-FD-700 (REV 12/16/15)	Page 1 of 2			





## OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION


Registered Design Professional Preparing Engineering Recommendations				
Company Name: EASE Co.				
Name: Jonathan Roberson, S.E. California License Number: S4197				
Mailing Address:5877 Pine Ave. Suite 210, Chino Hills, CA. 91709				
Telephone: 909-606-7622 Email: <u>J.Roberson@EASECo.com</u>				
OSHPD Special Seismic Certification Preapproval (OSP)				
<ul> <li>Special Seismic Certification is preapproved under OSP- (Separate application for OSP is required)</li> <li>Special Seismic Certification is not preapproved</li> </ul>				
Certification Method(s)				
<ul> <li>Testing in accordance with:</li> <li>Other* (Please Specify):</li> <li>OPM-0165-13</li> </ul>				
*Use of criteria other than those adopted by the California Building Standards Code, 2016 (CBSC 2016) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2016 may be used when approved by OSHPD prior to testing.  Analysis DATE: 01/31/2017 Combination of Testing, Analysis, and/or Experience Data (Please Specify):				
List of Attachments Supporting the Manufacturer's Certification				
□     Test Report     ⊠     Drawings     ⊠     Calculations     □     Manufacturer's Catalog       □     Other(s)     (Please Specify):				
OFFICE USE ONLY – OSHPD APPROVAL VALID FOR CBC 2016 & ALL PRE-2016 CODE BASED PROJECTS				
Signature:				
"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs" STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY OSH-FD-700 (REV 12/16/15) Page 2 of 2				

FOURMENT ANCHORAGE       5877 Pine Ave, Ste. 210         Chino Hills, CA. 91709       Phn: (909) 606-7622         Office of Statewide Health Planning and Development       PREAPPROVAL OF MANUFACTURER'S CERTIFICATION         OPM-0165-13       THIS PREAPPROVAL CONFORMS TO THE 2016 CALIFORNIA BUILDING CODE					
		CTURER: GCX CORPORATION ENT NAME: GCX WALL MOUNT with M & VHM ARMS	Sheet: <u>1 of 9</u> Date: 11/23/16		
GENERAL NOTES					
1.		OSHPD PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2016 CBC	. THE DEMANDS		
2.	<ul> <li>(DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2016 CBC</li> <li>2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.</li> </ul>				
3.					
4.					
5.					
6.	. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.				
7.	7. SHEET METAL SCREWS SHALL BE TEKS SCREWS BY ITW BUILDEX (ICC ESR-1976).				
8.	8. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.				
9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING					
		PROVIDE SUPPORTING STRUCTURE REQUIRED TO SUPPORT WEIGHTS AND FORCES SHOWN, IN A	ADDITION		
	PI A	VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2016 CBC AND WITH THE DETAIL PREAPPROVAL. VERIFY THAT THE ACTUAL EQUIPMENT'S WEIGHT, CG LOCATION, ANCHOR LOCAT AND THE MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE DN THE PREAPPROVAL DOCUMENTS.	FIONS, ANCHOR DETAILS		
		/ERIFY THAT THE COMBINATION OF SDS & z/h RESULT IN SEISMIC FORCES (Eh , Ev) THAT ARE NOT /ALUES ON THE DETAILS.	GREATER THAN THE		
		DESIGN BACKING PLATE, BLOCKING, STUDS, ETC. WHICH THE UNITS ARE ATTACHED TO AS NOTED ON THE DRAWINGS.	No. 4197 EXP. 6-30-2018 No. 1/23/16 No. LTON		

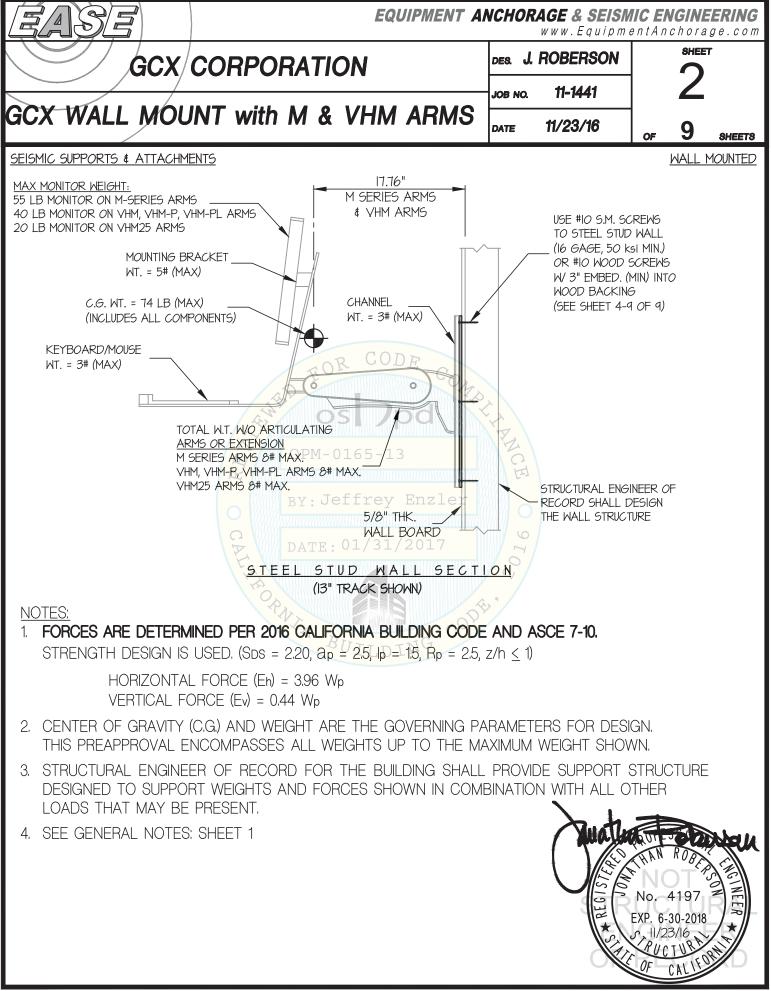


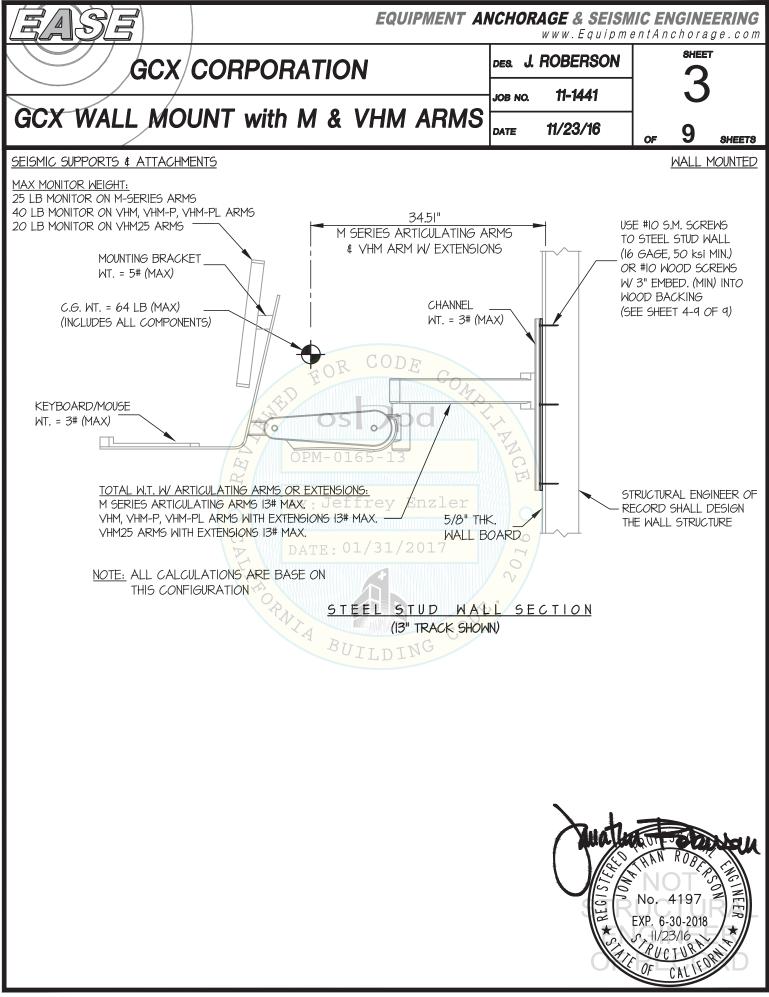

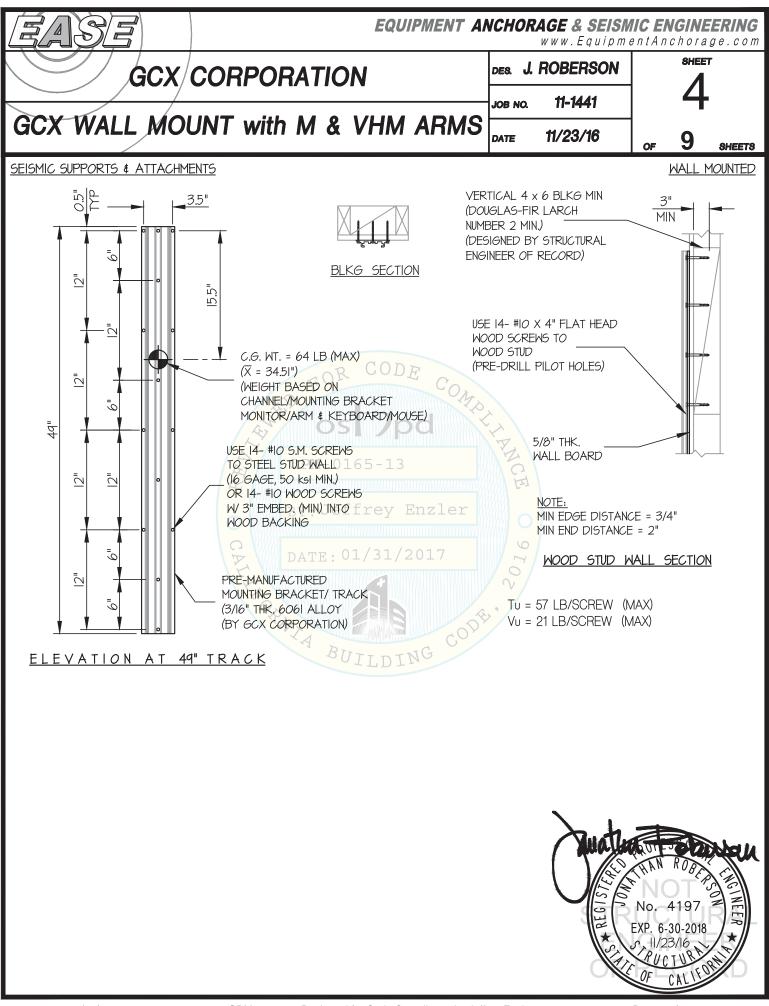

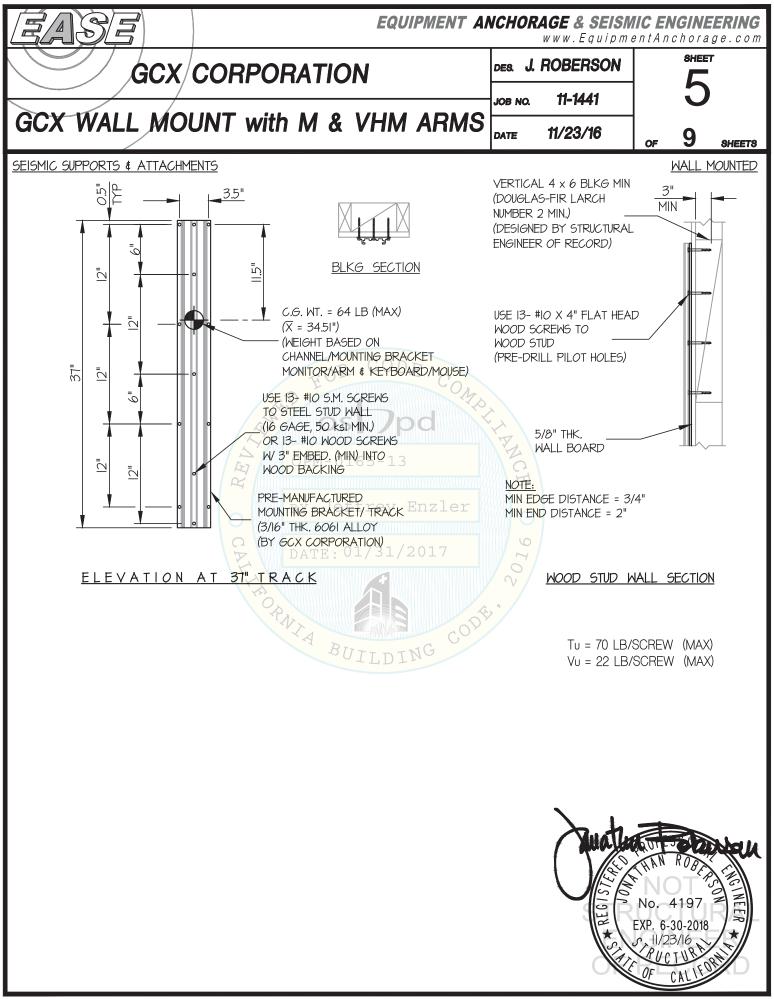

Page 7 of 11

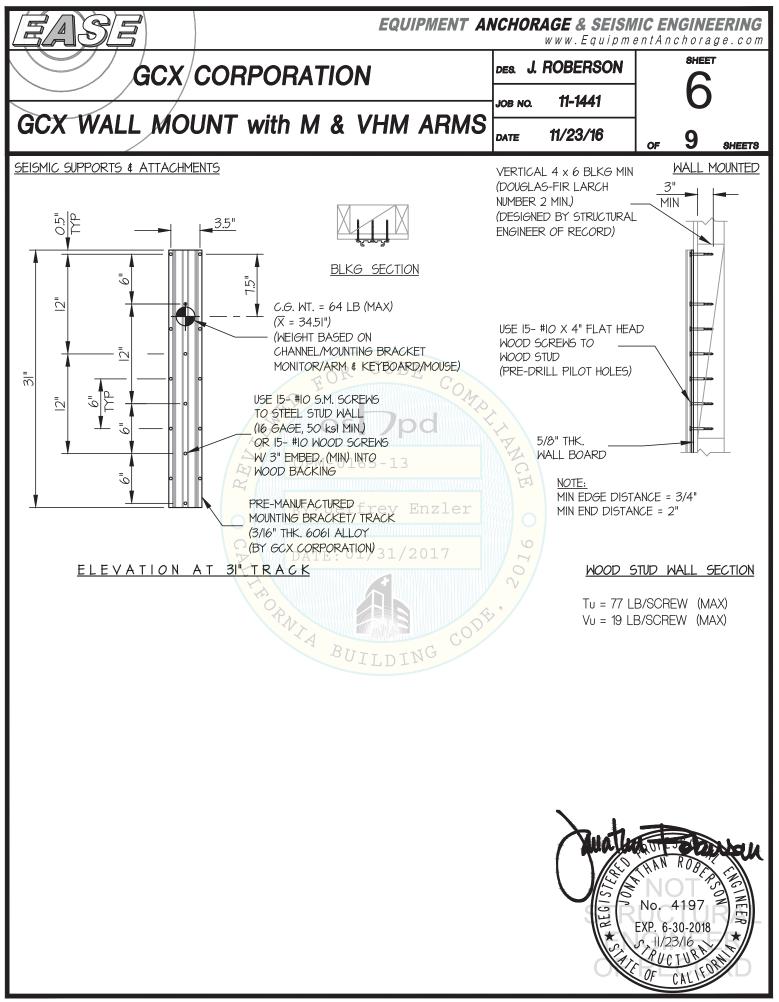

Page 8 of 11

