

Installation Guide

Utility Basket for 10 x 25mm to 10 x 35mm Rail

The purpose of this guide is to describe installation of Basket on Rail.

Parts Reference

The parts list below includes parts and hardware that will be used in this installation procedure.

Item #	Description	Qty
1	Utility Basket with Rail Clamp	1




Tools Required

Tools not required for installation of Basket.


Installing Utility Basket on Rail

Follow numbered callouts below.

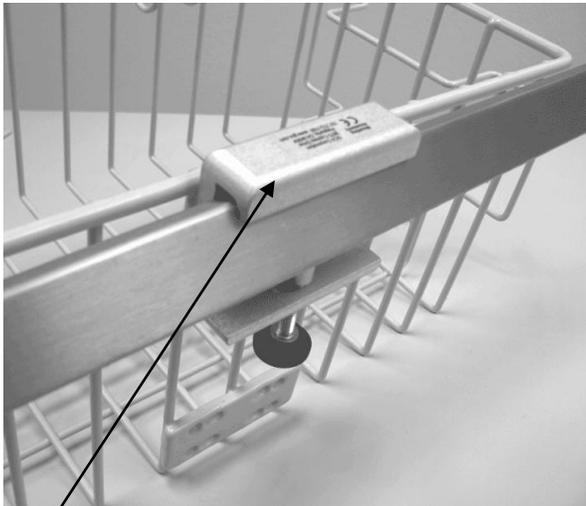

1. Hook the Rail clamp over the rail.

Installation Note: It may be necessary to loosen Clamping Knob before hooking Rail Clamp over the rail. Access to the knob may be easier from inside the basket.

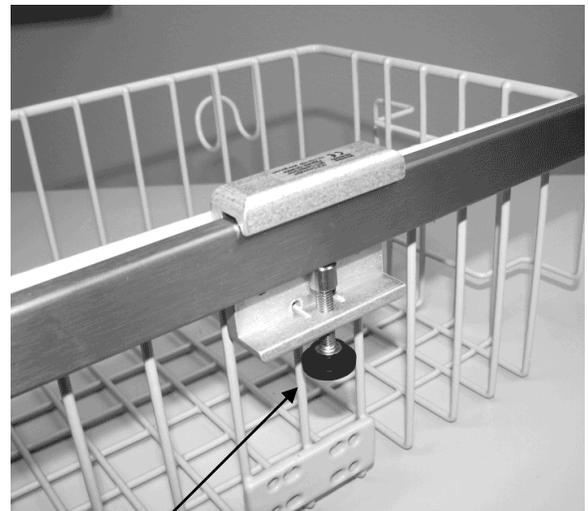

2. Tighten Clamping Knob.