

Installation Guide

7"/17.8cm Channel with 38mm Post Clamps

The purpose of this guide is to:

- Describe attaching Channel to 38mm Post.



Warning: Ensure that the weight of the mount and instrument to be installed does not exceed 60lbs (27.2kg).

Parts Reference

The following parts are included in this installation kit :

Item #	Description	Qty
1	Clamping Blocks	2
2	Channel	1
3	10-32 x 5/8" Socket Head Cap Screw (SHCS)	1
2	5/32" Hex Wrench	1

Tools Required:

- 5/32" Hex Wrench (provided)

Attaching Mount to Pole

Using the 5/32" Hex Wrench (supplied), attach Channel to the 38mm post with (2) Clamping Blocks and (4) 10-32 x 5/8" SHCS as shown below. Tighten screws equally (40 in/lb. minimum) until Clamping Blocks are secure.

