

Installation Guide

Dual Flat Panel Keyboard Bracket for Variable Height Roll Stand (VHRS)


The purpose of this guide is to:

1. Describe attachment of Keyboard Bracket to VHRS Post.
2. Describe attachment of Dual Flat Panel Display Mounting Bar to Keyboard Bracket.

Parts Reference

The following parts and hardware are included with this installation kit (see photos for major parts – hardware not shown).


| Item # | Description | Qty |
|--------|--|-----|
| 1 | Keyboard Bracket, Dual Flat Panel | 1 |
| 2 | #10-32 x 5/8" Flat Head Machine Screw (FHMS) | 3 |
| 3 | #10-32 x 5/8" Socket Head Cap Screw (SHCS) | 3 |
| 4 | 5/32" Hex Wrench | 1 |

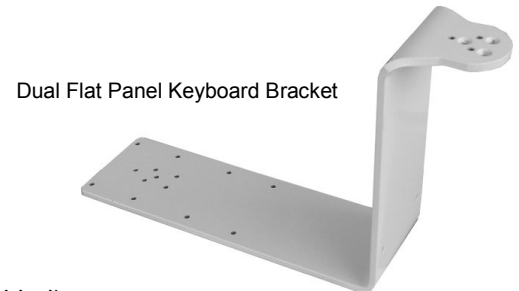

Tools Required: Phillips screwdriver (not provided), 5/32" hex wrench (provided).

Attaching Keyboard Bracket to VHRS Post

1. Fasten Keyboard Bracket to 3-hole mounting pattern with three (3) #10-32 x 5/8" FHMS.

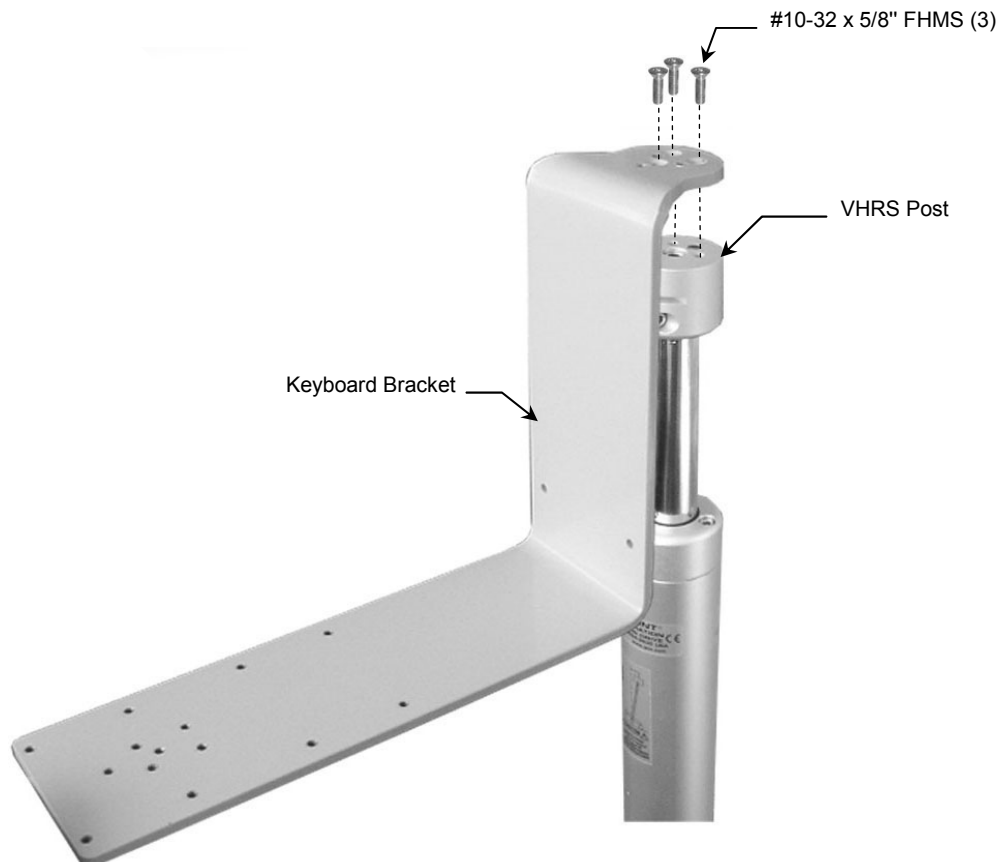

Attaching the Dual Flat Panel Display Mounting Bar and Keyboard Tray/Plate to the Keyboard Bracket

Hardware is provided in this installation kit for attaching the Dual Display Mounting Bar to the Keyboard Bracket. See upper callout below for attachment of Display Mounting Bar.

See lower callout for Keyboard Tray/Plate information.

