

GCX Mounting Assembly Installation Guide

Pivoting Flat Panel Pedestal Mount for (75mm/100mm VESA) For Landscape & Page Mode Display Operation




The purpose of these instructions is to:

1. Describe attachment of 75mm/100mm VESA compatible flat panels to the mounting hardware.

REFER TO QUALIFIED PERSONNEL

ATTACH THE FLAT PANEL DISPLAY

1. Select the appropriate length M4 FHMS (kit includes: M4 X 8mm FHMS (4), M4 X 12mm FHMS (4), M4 X 20mm FHMS (4), M4 X 30mm FHMS (4) & 3/8 X .171 X 1/2" Nylon Spacers (4)). Run two of the screws part way into the top two threaded inserts of the 75/100 mm mounting pattern. Align the screws with the slots on the mounting plate and insert. Insert the bottom two (2) M4 screws and secure all fasteners. **Note:** If spacers are needed due to an inset mounting location or the need for ventilation clearance, they should be used between the Mounting Plate and the display chassis.

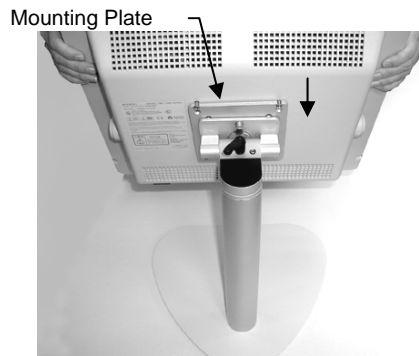

PIVOT OPERATION - The display can be pivoted 90 degrees left or right for page mode operation

1. Pull the Spring Plunger located near the top of the Mounting Plate. Pull the Plunger and rotate clockwise or counterclockwise until the Spring Plunger re-engages the Mounting Plate. Reverse procedure to return to landscape position.

Spring Plunger



TILT OPERATION

1. Adjust the overall tilting tension by evenly tightening or loosening the (2) #8-32 X 1/2" SHCS - (Tension Adjustment Screws) with the hex key provided. Use the Tilt Adjustment Lever to fine tune and lock the tilt position.
2. Turn Adjustment Lever counter clockwise to loosen. Grasp the top and/or bottom of the device and tilt to the desired angle.
3. Turn Adjustment Lever clockwise to tighten and lock position.

