

Installation Guide

LIL-0003-XX Mountable Tablet Enclosure with Locking Charging Dock Hardware Guide

The purpose of this guide is to:

1. Describe on how to assemble customer-provided tablet to Mountable Tablet Enclosure with Locking Charging Dock (Page 2).
2. How to mount to VHM-T, Variable Height Tablet Arm (Page 3).
3. How to mount to Patient Engagement Tablet Roll Stand, PTE-0001-60 (Page 5).
4. How to mount to GCX M-Series, VHM-25 Variable Height Arm and iPad Roll Stand IPA-0001-60 (Page 5-6).
5. How to lock Mountable Table Enclosure to Locking and Charging Dock (Page 6).

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):

Item #	Description	Qty
1	M4 x 10 mm Pan Head Machine Screw (PHMS)	4
2	M4 x 12 mm Pan Head Machine Screw (PHMS)	4
3	M4 x 14 mm Pan Head Machine Screw (PHMS)	4
4	M4 Lock Nut	4
5	Mountable Tablet Enclosure	1
6	Locking Charging Dock with Charging Cable	1
7	Keys	2
8	Key Extender	1

Tools Required:

Philips #2 Screwdriver (Not Provided)

7mm Wrench (Not Provided)

Tablet: Customer-supplied



Unlocked
Charging Dock



Two (2) Keys




Locked
Charging Dock




Key Extender

How to Assemble Mountable Tablet Enclosure with Locking Charging Dock:

1. On the rear of the Mountable Tablet Enclosure, pinch together the release clips.
2. While still pinching the release clips, flip the Mountable Tablet Enclosure over and slide the front of the enclosure away from the home button.

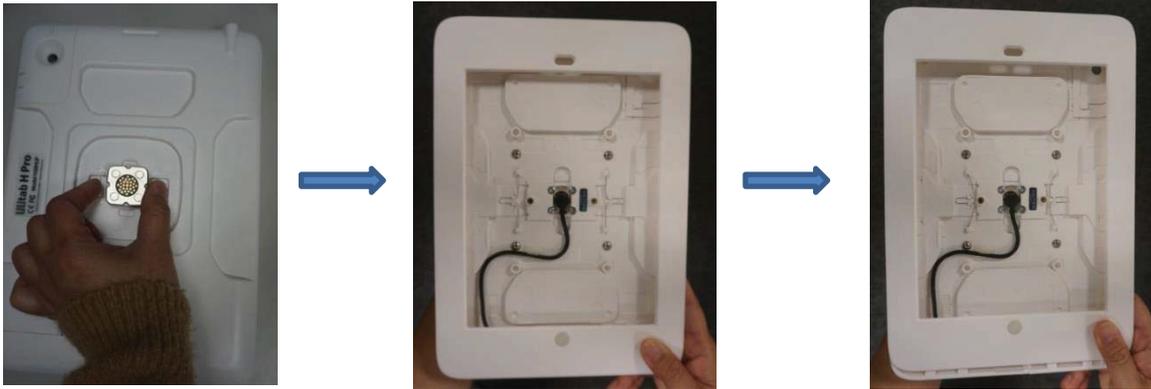


3. Before inserting your tablet, ensure the tablet charging port is on the same side as the USB Charging Cable. Lay the Mountable Tablet Enclosure face up, insert your tablet and connect the USB Charging Cable.
4. Slide the front of the enclosure onto the rear of the enclosure the same way you removed it. The front of the enclosure only fits in one direction, if it does not slide easily, reverse the orientation. Do **not** force it.

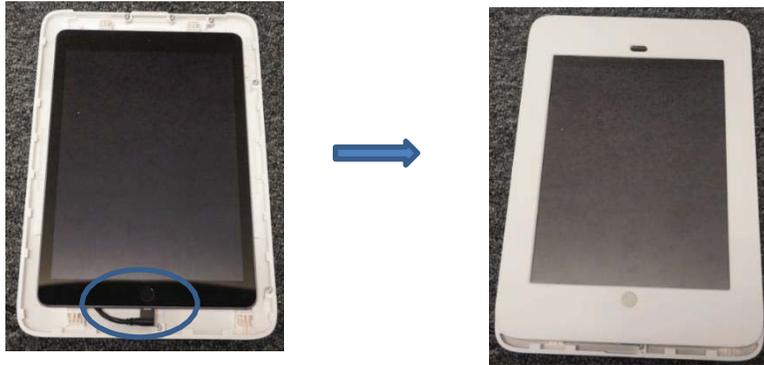


Mounting Tablet Enclosure with Locking Charging Dock to VHM-T, Variable Height Tablet Arm

1. Align the Locking Charging Dock 75mm VESA to the VHM-T, Variable Height Tablet Arm.
2. Attach to the VHM-T, Variable Height Tablet Arm, using a Philips #2 Screw Driver and 7mm Wrench with (4) M4 x 14mm Pan Head Machined Screw and (4) M4 Lock Nuts.

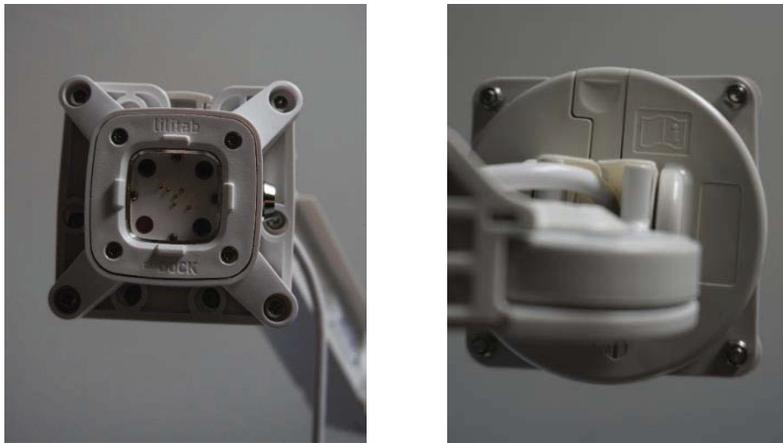

3. Cable Management: Open the cable plug on the front of the VHM-T, Variable Height Tablet Arm, to route the USB Charging Cable through the arm.

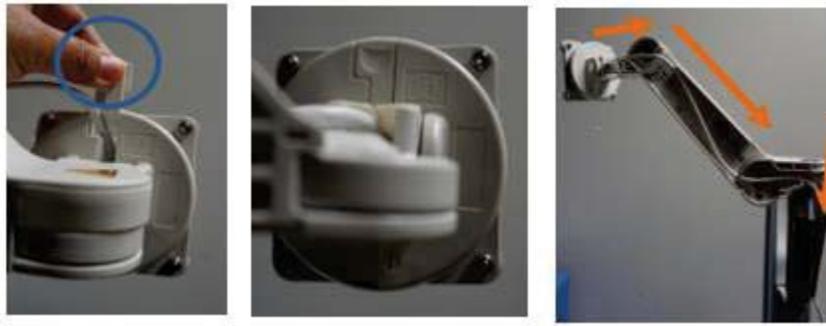

Mounting Tablet Enclosure on Patient Engagement Tablet Roll Stand, PTE-0001-60


1. Mount the Locking Charging Dock with the 75mm VESA using Philips #2 Screw Driver and 7mm Wrench with (4) M4 x 10 mm Pan Head Machined Screws and (4) M4 Lock Nuts.



Photo Includes Ball Joint VESA Mount, for Reference Only

Mounting Tablet Enclosure on M-series / VHM-25 Arms / iPad Roll Stand, IPA-0001-60

1. Mount the Locking Charging Dock with the 75mm VESA using a Philips #2 Screw Driver and 7mm Wrench with (4) M4 x 12 mm Pan Head Machined Screws and (4) M4 Lock Nuts.

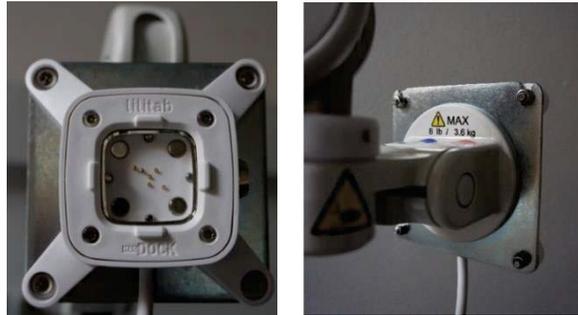

2. Route the cables through the flexible Cable Guide beneath the arm. For further cable management instructions, reference page 3 of DU-WMM-0001 Rev D for M-Series Arms or page 7 of DU-WS-0008-01 Rev B for VHM-25 Arms.



How to lock Mountable Tablet Enclosure:

Mount Mountable Tablet Enclosure to the magnetic Locking Charging Dock and secure with key provided. There is a key extender provided with this kit.

