

Installation Guide

Channel Mounting Kit for Philips M3176C USB Recorder – Single or Dual Side-by-Side


The purpose of this guide is to:

1. Describe mounting of Recorder Mount in channel (page 1).
2. Describe mounting of single recorder (pages 2).
3. Describe mounting of two recorders (page 2).

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):


Item #	Description	Qty
1	Recorder Mount Assembly, Single	1
2	Mounting Plate, Dual	1
3	#6-32 x 1/2" Flat Head Machine Screw (FHMS)	8
4	#10-32 x 7/16" FHMS	4
5	1/8" Hex Wrench	1

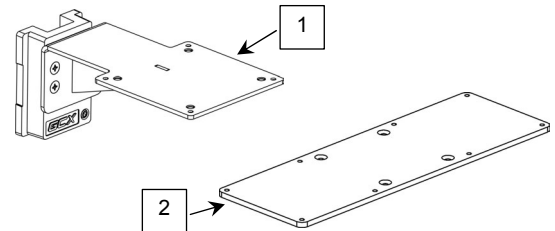

Tools Required: 1/8" Hex Wrench, Phillips screwdriver (not provided).


Mounting Recorder Mount in Channel

1. Insert Slide (rear of Mount) in channel and guide Mount to desired mounting position. Using the 1/8" hex wrench provided, tighten the set screw at the lower right corner of the Slide.

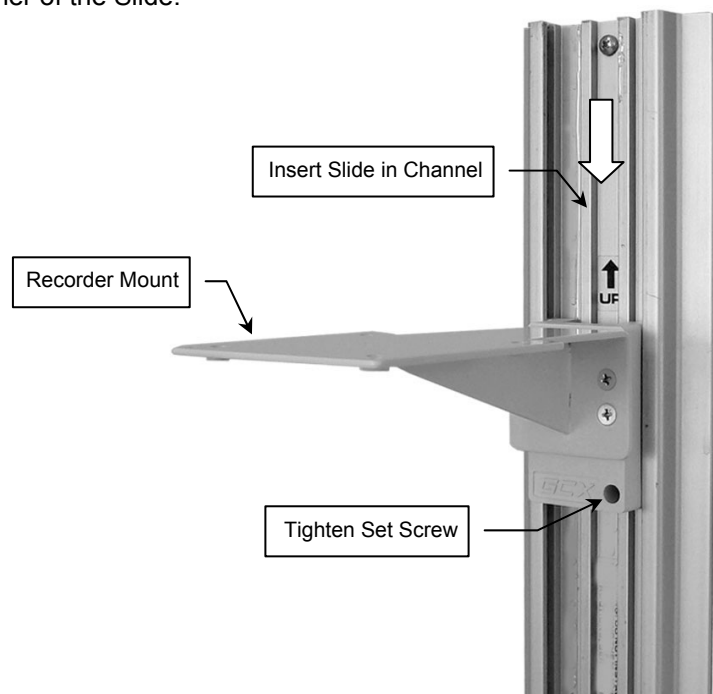


Mounting a Single Recorder

1. Fasten recorder to Mount with four (4) #6-32 x 1/2" FHMS as shown (right).

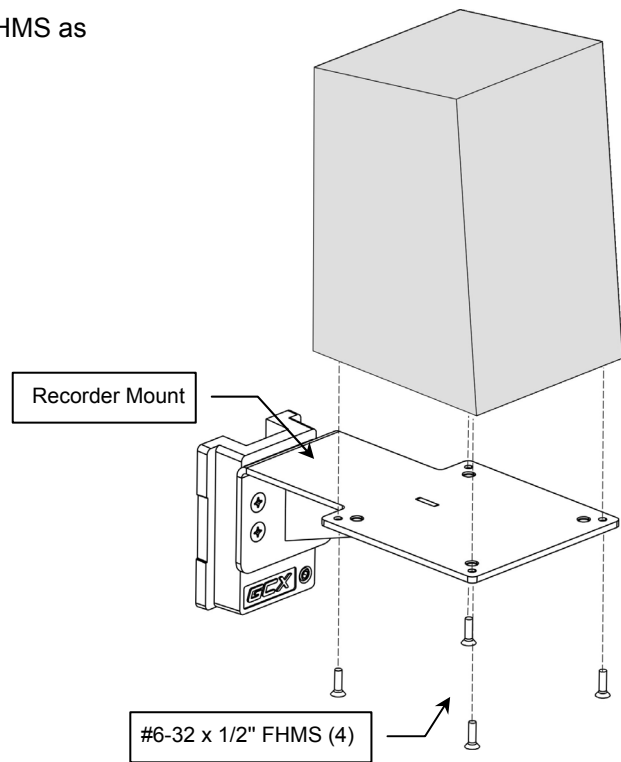

Mounting Dual Recorders

1. Fasten Dual Mounting Plate to Recorder Mount with four (4) #10-32 x 7/16" FHMS as shown below left.
2. Fasten each recorder to the Recorder Mount with four (4) #6-32 x 1/2" FHMS as shown below right.

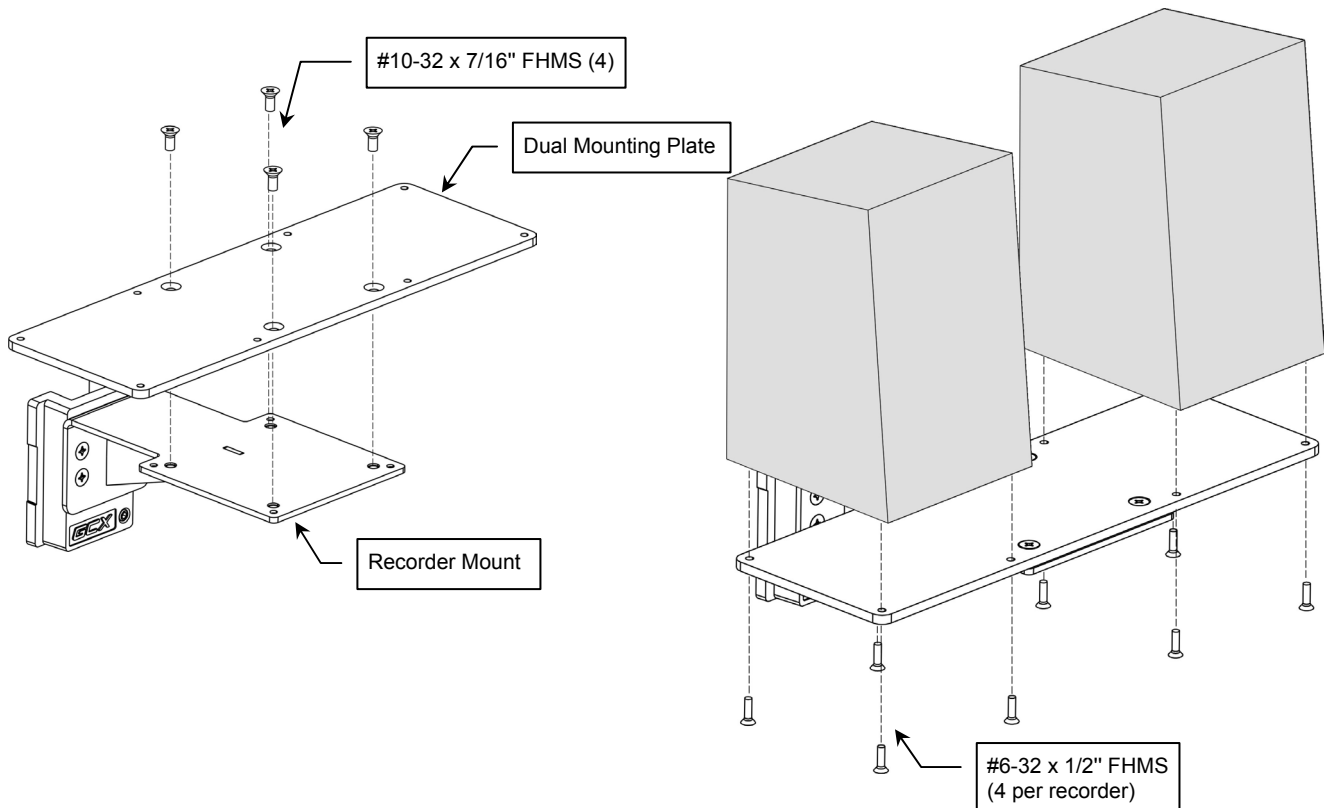

Periodic Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately using a water-dampened cloth. Dry the mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.