
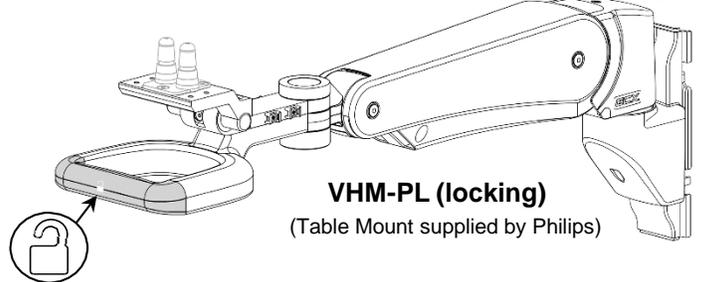


### Operation Guide



#### VHM-P (Non-Locking) and VHM-PL (Locking) Variable Height Arm (Slide-Below-Arm Configuration)

For Philips IntelliVue MP5/20/30/40/50, MX400/450/500/550/600/700/750/800/850, AD85/75

**WARNING:** MX750 and MX850 are compatible with one 4-slot rack only due to the limit of the maximum Arm Load Range.

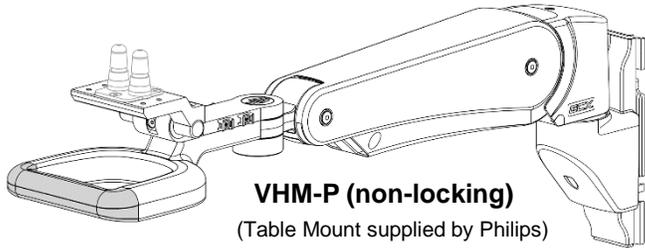

#### Safety Label Locations (VHM-P Series Arms)

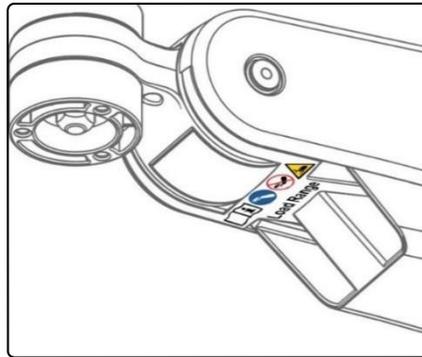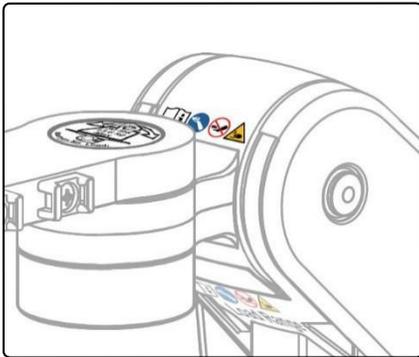

Refer to Installation Guide for additional information.



Remove the device only when the arm is at the highest position.



Do not remove the device when the arm is in a lowered position.





Potential pinch point that may cause personal injury.

Maximum weight for mounting plate with tilt function.

**Counter Balance Load Range:**  
25 - 45 lb / 11.3 - 20.4 kg  
or  
15 - 30 lb / 6.8 - 13.6 kg

#### Safety Label Locations (VHM-PL Series Arms)

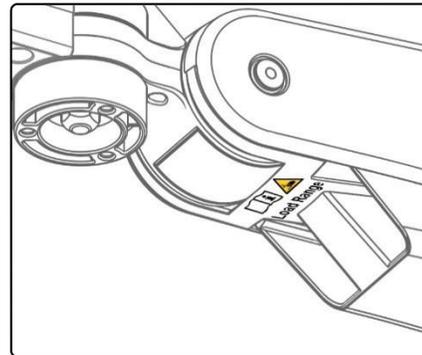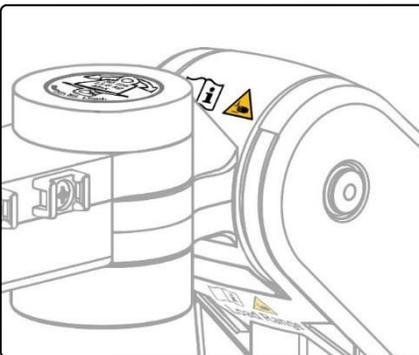

Refer to Installation Guide for additional information.





Potential pinch point that may cause personal injury.




Maximum weight for mounting plate with tilt function.

**Counter Balance Load Range:**  
20 - 45 lb / 9.1 - 20.4 kg  
or  
11 - 30 lb / 5.0 - 13.6 kg

VHM-P (non-locking only)

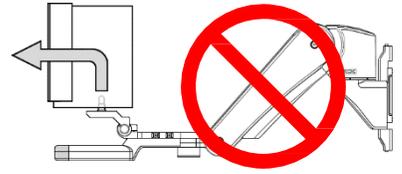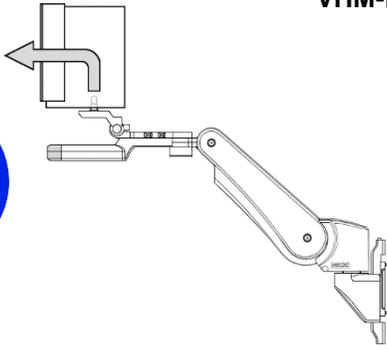


Pinch Points

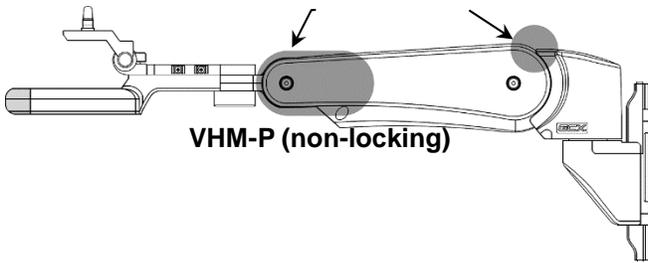


VHM-P (non-locking)

Pinch Points

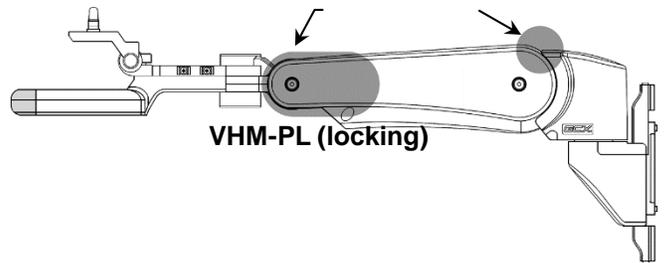

VHM-PL (locking)

VHM-P (non-locking)

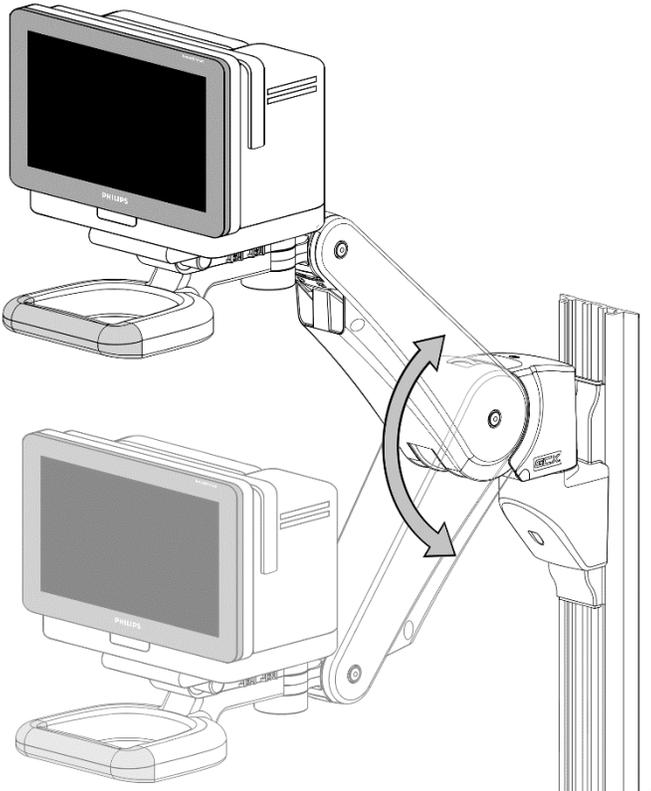

VHM-PL (locking)

