



Installation Guide

Mounting Power Supply on Roll Stand Post

The purpose of this guide is to describe mounting of power supply on roll stand post.

Min. Power Supply Size Using 12" [30.5 cm] Hook & Loop Straps	Max. Power Supply Size Using 18" [45.7cm] Hook & Loop Straps
1.4" [3.56 cm] x 2.3" [5.84 cm] or 7.4" [18.8 cm] perimeter	4.0" [10.16 cm] x 4.0" [10.16 cm] or 16" [40.6 cm] perimeter

1. Place power supply in Bracket as shown below left. Insert ends of hook-and-loop straps through rings, pull straps tight, and press firmly. **Installation Note:** Bracket is pre-assembled with 12" hook-and-loop straps. These straps securely hold most power supplies, but may be replaced with 18" straps (provided) for larger power supplies.
2. Loop power cable through Nylon retainer (below right). Fasten the Nylon retainer to the bottom of the bracket using the #8 flat washer and #8-32 x 1/4" PHMS included.

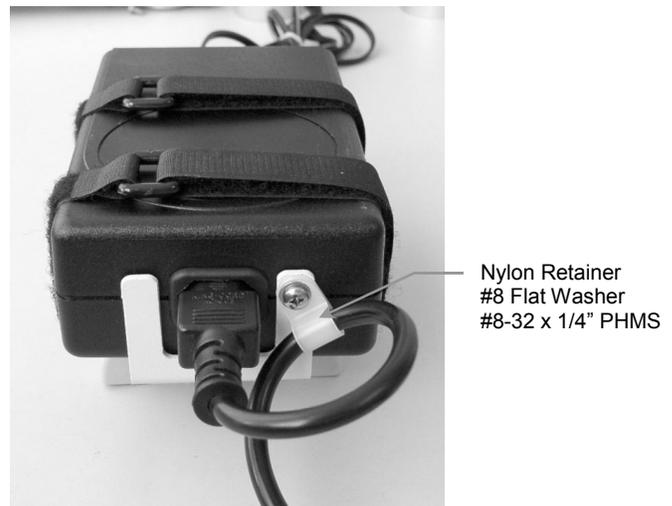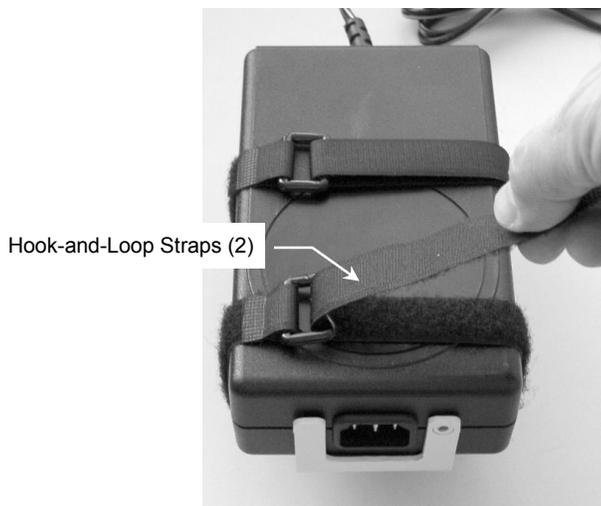

3. Fasten Clamping Blocks to Roll Stand Post with two (2) #10-32 x 9/16" PHMS (below). **Installation Note:** The power supply mounting position is determined by the application. Mounting shown below is typical for many applications but not required.

