

Installation Guide UPS Mount, VHRC/RC cart base (Fits all VHRC/RC carts with standard base, does not fit Slimline cart base)



Max UPS size: 5 3/4"/14.6cm (W) x 18"/45.7cm (L) x 7"/17.8cm (H)




Accommodates the following UPSs:

Manufacturer	Model	Dims (H" X D" X L")	Weight	Medical Grade
PowerVar	ABCE420-11IEC	6.6 (16.7cm) x 5.5 (13.9cm) x 17.7 (44.9cm)	30lbs (13.6kg)	yes
APC	SC650	6.6 (16.7cm) x 4.7 (11.9cm) x 14.5 (36.8cm)	27lbs (12.2kg)	no
APC	SC420	6.6 (16.7cm) x 4.7 (11.9cm) x 14.5 (36.8cm)	22lbs (9.97kg)	no
APC	SMT500/750	6.2 (15.7cm) x 5.4 (13.7cm) x 14.1 (35.8cm)	29lbs (13.1kg)	no
APC	SUA500/750US	6.2 (15.7cm) x 5.4 (13.7cm) x 14.1 (35.8cm)	29lbs (13.1kg)	no


Parts Reference: The following parts and hardware are included with this installation kit.

Item #	Description	Qty
1	Base Spacer	1
2	Tray assembly with Velcro straps	1
3	5/16-18 x 1 1/4" flange bolts	2
4	5/16-18 x 5/8" hex bolts	3
5	5/16" flat washers	3
6	1/2" socket wrench	1

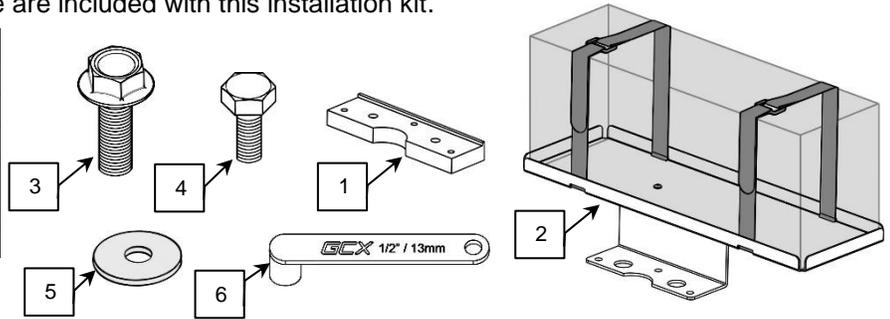


Tools Required: 1/2" Socket Wrench (provided).


Installation

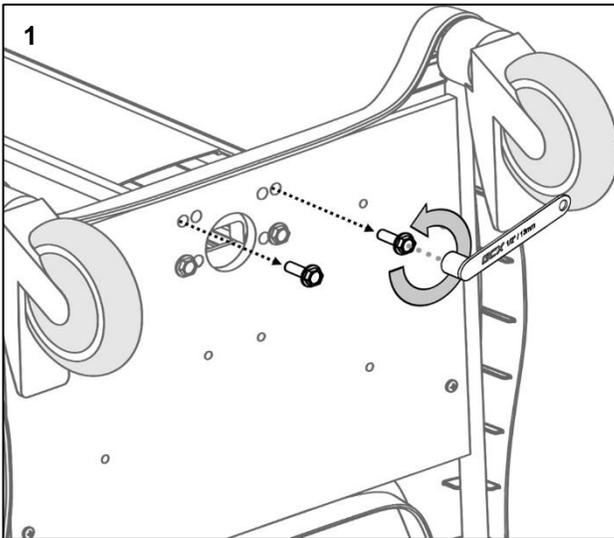


Lay cart down so bottom of cart base can be accessed. Remove and discard the (2) rear column flange bolts with a 1/2" socket wrench.

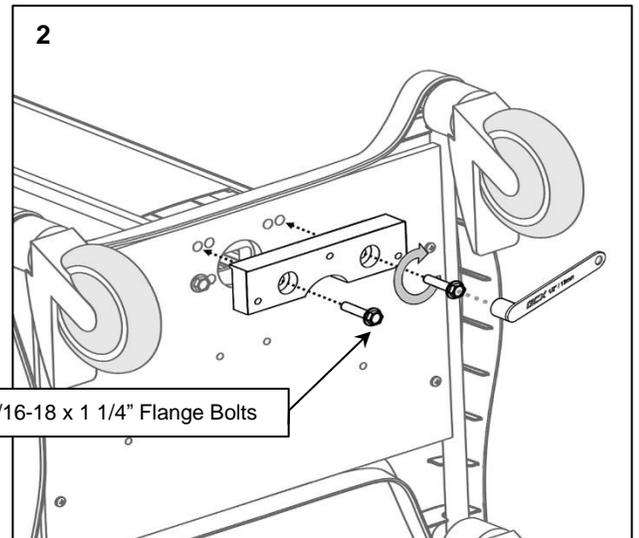



Install base spacer using (2) 5/16 x 1 1/4" flange bolts.

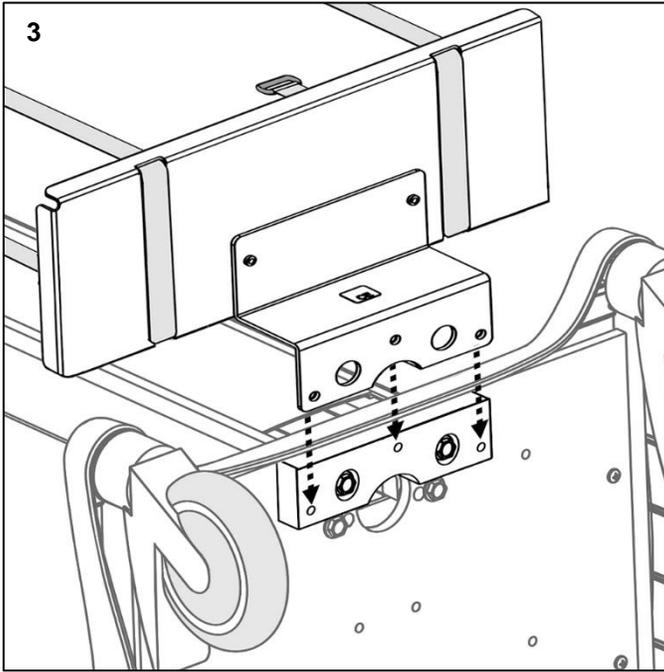

Align the (3) mounting holes on the tray assembly with the corresponding (3) threaded mounting holes on the base spacer.

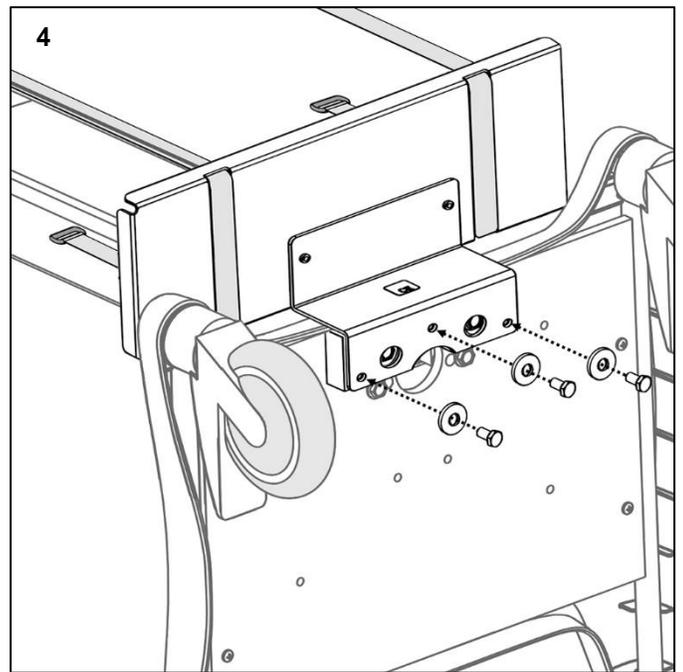

Using a 1/2" socket wrench, secure the tray assembly to the base spacer with (3) 5/16-18 x 5/8" bolts and (3) washers.

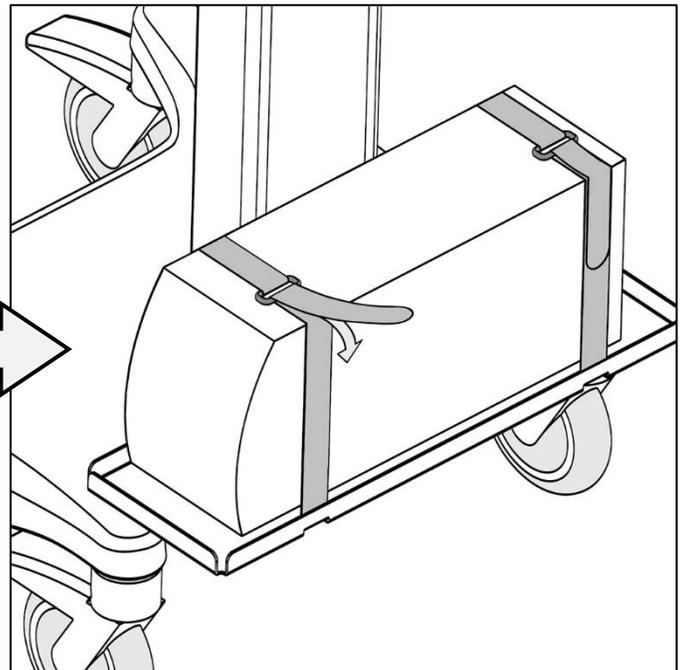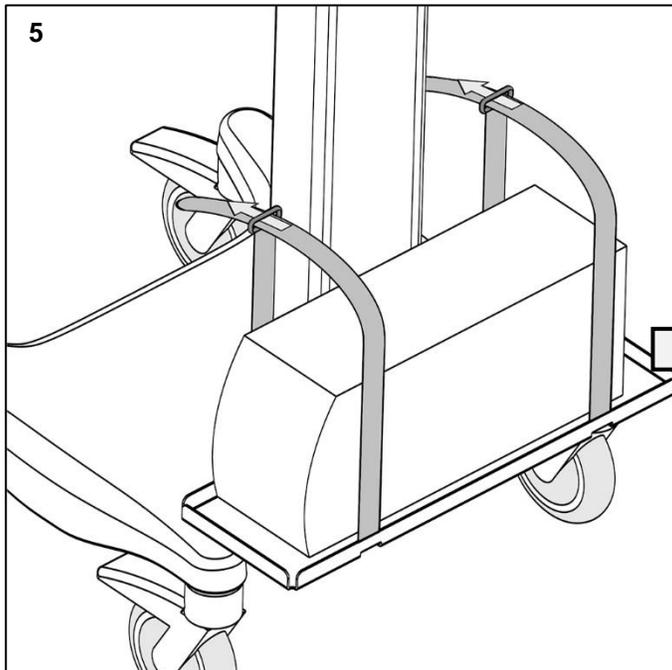

Install UPS to tray, and secure with the (2) Velcro straps.

Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the arm that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the arm immediately, using water dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products before installing the arm in the Channel.