

GCX Mounting Assembly Installation Guide

Flush Wall Mount Power Strip Assembly (Tripp-Lite UL 1363A Rating)

The purpose of this guide is to describe mounting of the Power Strip Assembly.

Parts Reference

The following parts and hardware will be used in this procedure:

Item #	Description	Qty
1	Power Strip – Channel Slide Assembly	1
2	Hook and Loop Strap, 18"	2
3	1/8" Hex Key	1

Tools Required: 1/8" hex key (provided).



Mounting Power Strip in Channel

1. Insert Slide (attached to rear of Power Strip) in Channel and slide Power Strip to desired mounting position.
2. Using 1/8" hex key provided tighten the socket set screw at the lower right corner of the Slide.

Installation Note: Two (2) Hook and Loop Straps are provided with this kit for cable management.

