

Installation Guide

4-Outlet Power Strip (Tripp-Lite UL1363A Rating) w/Cord Loop Hook for Roll Stand (2" Diameter Post)




The purpose of this guide is to describe mounting the Outlet Strip/Cord Loop Hook assembly on the roll stand.

Parts Reference

The following parts and hardware are provided for this procedure:

Item #	Description	Qty
1	Power Outlet Strip	1
2	Clamping Block, Rear Half	1
3	Cord Loop Hook (Front Clamping Block Attached)	1
4	Power Strip Adaptor Plate	1
5	10-32 x 9/16" Pan Head Machine Screw (PHMS)	2
6	# 10 x 5/8 Flat Head Sheet Metal Screw (FHSMS)	2
7	10-32 x 5/16" Pan Head Machine Screw (PHMS)	2

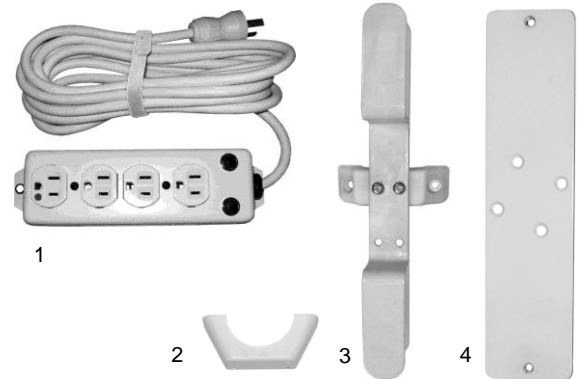


Tools Required: Phillips screwdriver.


Mounting the Power Outlet Strip/Cord Loop Hook on the Roll Stand

Installation Note: The Outlet Strip may be mounted vertically or horizontally on the roll stand post. Select the orientation that best suits your application and follow the procedure below:

1. Fasten rear Front Clamping Block with Cord Loop to threaded mounting holes on rear of Rear Clamping Block with two (2) #10-32 x 9/16" PHMS.

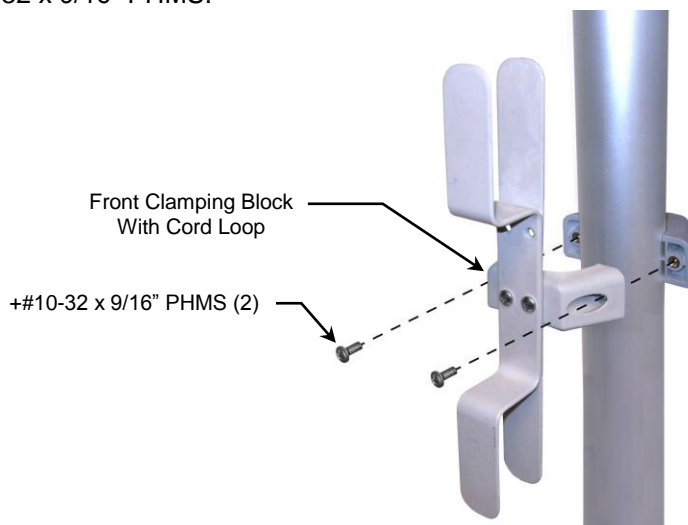

2. Fasten Power Strip Adaptor Plate to Front Clamping Block with two (2) # 10 x 5/8 (FHSMS).

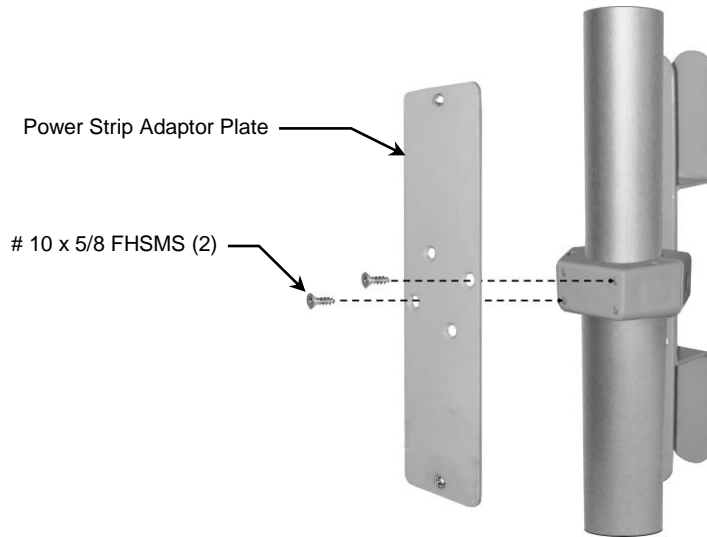

Installation Note: The Outlet Strip may be mounted horizontally or vertically on the roll stand post. Select the orientation that best suits your application and follow the procedure below:

Vertical Application



Horizontal Application



Fasten Power Outlet Strip to Adaptor Plate with two (2) 10-32 x 5/16" PHMS.

