


Installation Guide

VHRC Camera Mount


The purpose of this guide is to describe the assembly and installation of the VHRC Camera Mount.

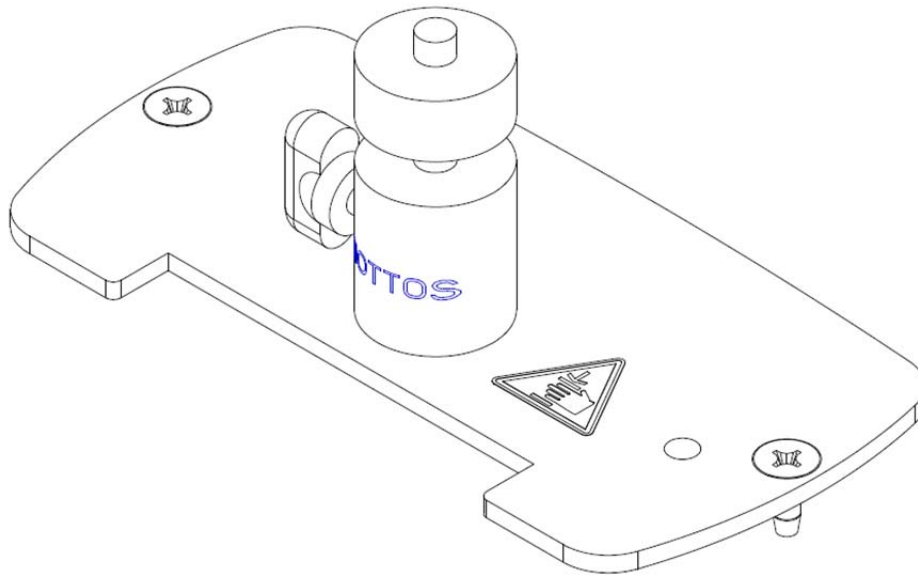

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):


Item #	Description	Qty
1	VHRS Top Camera Mount with Giottos Tilt/Swivel Head	1
2	#10 x 1/2 Flat Head Sheet Metal Screws (FHSMS)	2

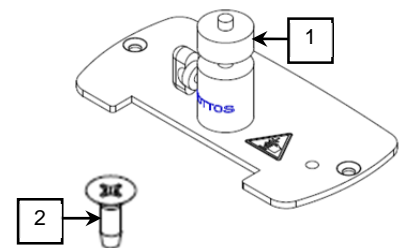


Tools Required: #2 Phillips screwdriver (not provided)

Installing Camera Mount in Channel

1. Remove existing channel cover by unscrewing two flat head fasteners. Discard top and fasteners

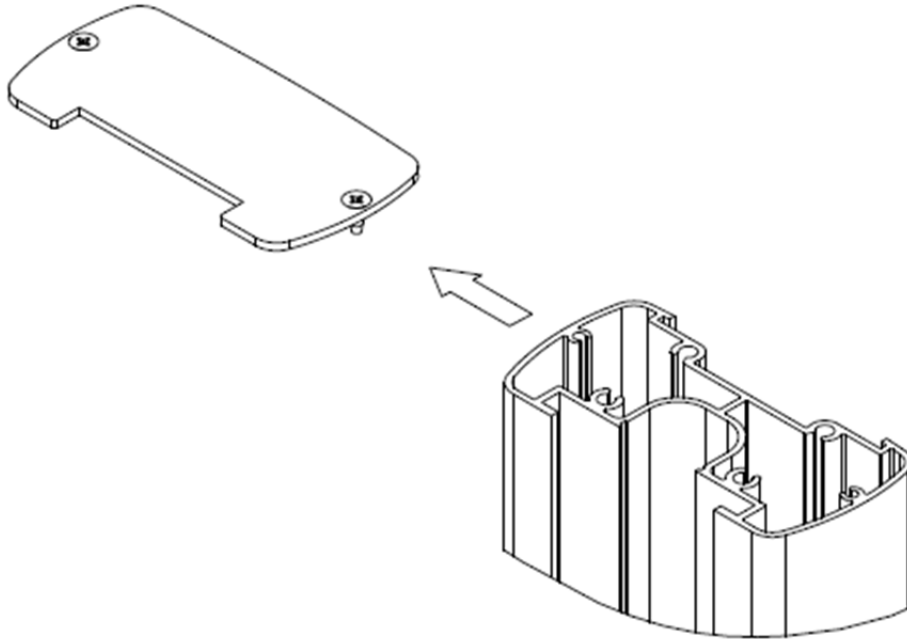


2. Install new mounting plate using two #10 x 1/2 FHSMS (included)

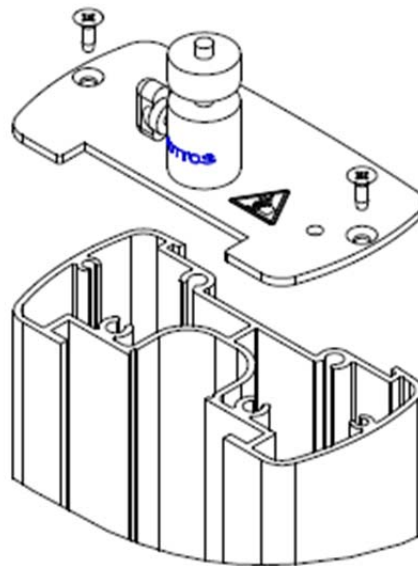

Installing Camera onto Mount

1. Align Camera mounting hole above the 1/4 - 20 thread of Camera Mount and turn knob counterclockwise to secure camera in place.

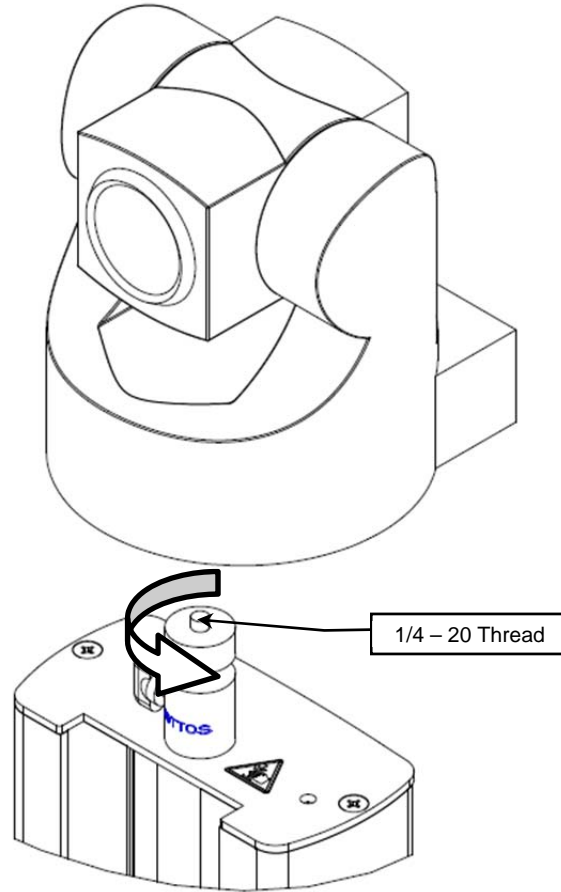


Camera Tilt/Swivel Adjustment

1. To adjust tilt/swivel tension, loosen the knob on the side of the camera mount while grasping camera. Adjust camera position, and retighten knob to secure camera in place.



Caution: Potential pinch point if camera is not stabilized during adjustment.

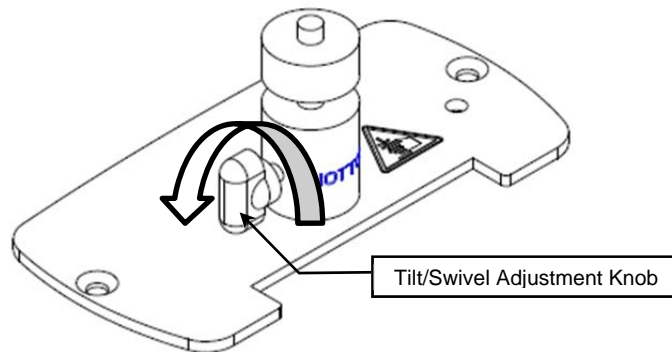

Routine Maintenance

Periodically check all mounting hardware. Tighten as necessary for optimal operation.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assemblies. Wipe any cleaning agents off of the mounting assemblies immediately, using a water-dampened cloth. Dry parts thoroughly after cleaning.