

## Installation Guide

### Adjustable Device Mount for Channel


*The purpose of this guide is to describe installation of the Mount.*

**Installation Note:** This mount is available in two sizes (shown below) for different device widths. This installation guide covers either mount. Only one Mount is included in this installation kit.

#### Parts Reference

The following parts and hardware are included with this installation kit:

Item #	Description	Qty
1	Mounting Assembly	1
2	1/8" Hex Wrench	1

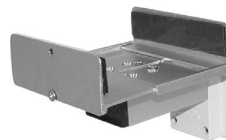

3" to 5" (P/N: WM-0030-01)




5" to 8" (P/N: WM-0030-02)

#### Tools Required


- Phillips screwdriver (not provided)
- 1/8" hex wrench (provided)

#### Installing the Mount

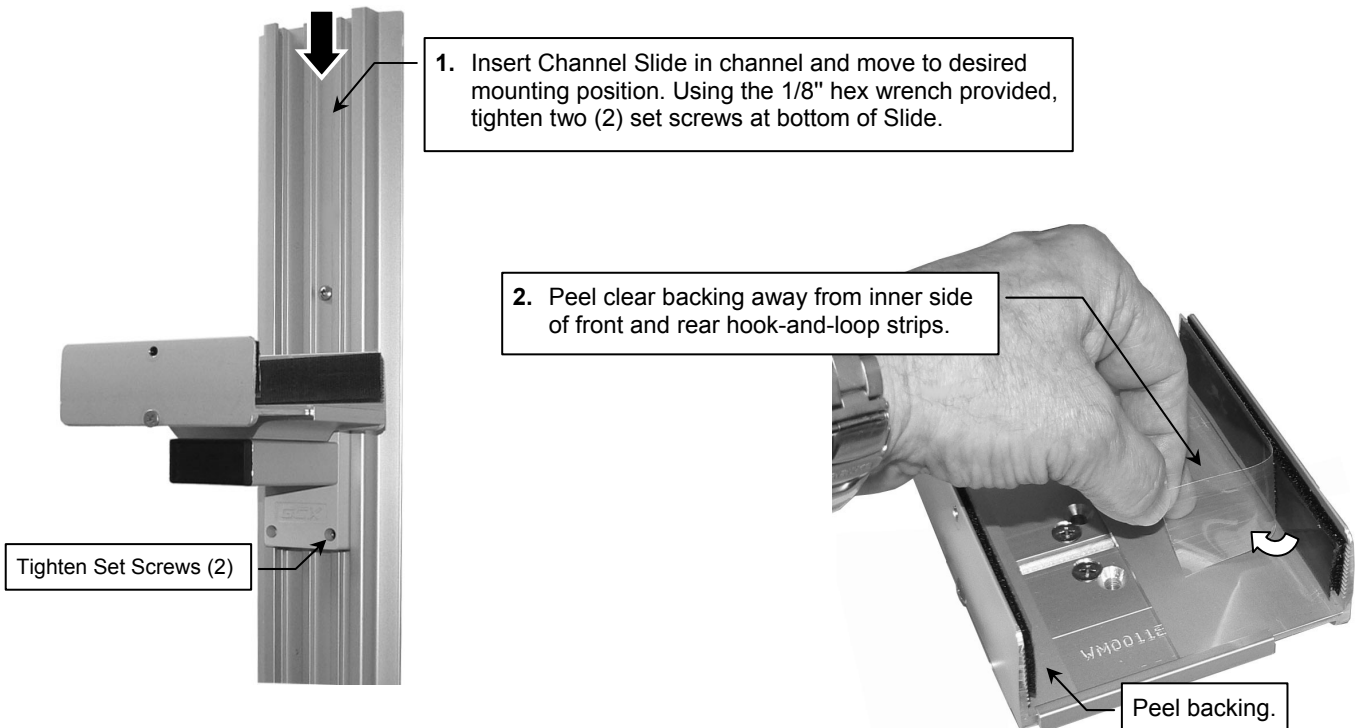

3. Clean sides of device with Isopropyl Alcohol where adhesive will make contact.

4. Insert device in Mount and tighten Clamping Screw.  
**Installation Note:** It may be necessary to first loosen the Clamping Screw to allow the device to fit in the Mount.

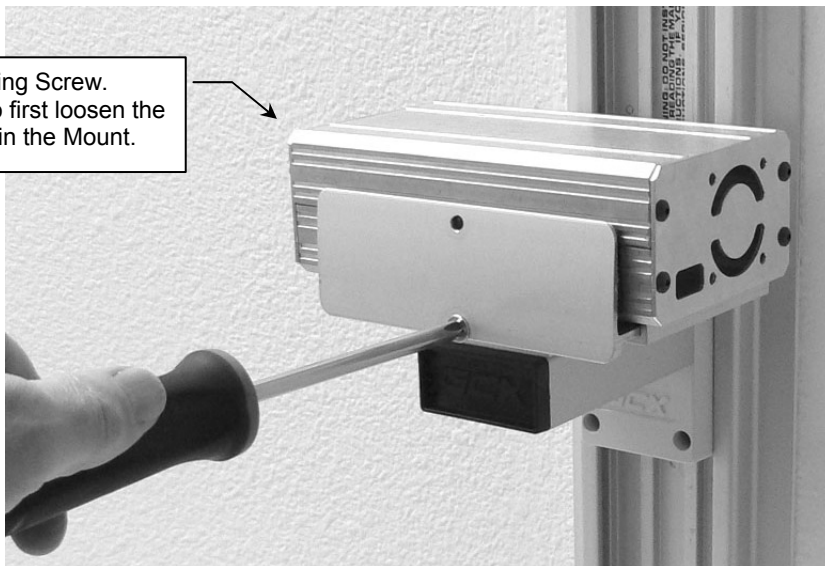

### **Routine Maintenance of the Mounting Assembly**

Periodically inspect all mounting hardware. Tighten or adjust as necessary for optimal operation and safety.

### **Cleaning the Mounting Assembly**

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately using a water-dampened cloth. Dry mounting assembly thoroughly after cleaning.