

## Configuration Guide

### Variable Height Roll Cart VHRC™ Series (with Large LCD Monitor)

The purpose of this guide is to provide the correct VHRC column weight range which is dependent on the total weight of GCX mounts and hospital equipment/devices that will be mounted to the height adjustable channel.

First determine which GCX mount(s) will be integrated within the height adjustable channel, then total their weight. Add this weight to the weight(s) of all medical equipment/devices that will be mounted. This total weight will determine the correct weight range [ and p/n ] for the gas spring assisted height adjustable VHRC. Maximum total weight on front and rear of column must not exceed 100 lbs/45.4 kg.

Single large flush mounted LCD display, (Optional) 16"/40.6 cm M Series arm with keyboard mount and CPU mounted in the lower portion of the height adjustable channel.

Note: This configuration always requires the 25 lb/11.3 kg base counterweight kit.

#### GCX Mount P/N's and Weights

<b>LCD Flush Mounts</b>	<b>Weight</b>
FLP-0010-04	7 lbs/3.2 kg
<b>16" M Series Arm with Swivel Head</b>	<b>Weight</b>
WMM-0005-03	3.5 lbs/1.6 kg
<b>Keyboard Mount w/ L/R Slide-out Trays</b>	<b>Weight</b>
WM-0023-49	5.5 lbs/2.5 kg
<b>CPU Mounts</b>	<b>Weight</b>
WM-0007-40	4.5 lbs/2 kg
WM-0007-41	5 lbs/2.3 kg
<b>Power Strip</b>	<b>Weight</b>
SR-0001-03	2 lbs/.91 kg

#### GCX VHRC Cart Models and Weight Range

Part Number	Weight Range
RST-0007-03	40-50 lbs/13.6-22.7 kg
RST-0007-04	50-60 lbs/22.7-27.2 kg
RST-0007-05	60-70 lbs/27.2-31.8 kg

#### Required Accessory for this Configuration

RST-0007-22: 25 lbs/11.3 kg Base Counterweight Kit

#### GCX VHRC Optional Accessories

RST-0007-24 - Rear Column Mounted Handle

1.2 lbs/.54 kg

RST-0007-26 - Rear Column Mounting Plate

Note: Rear mounted accessory weights do not need to be calculated into the cart weight range, but should be calculated in the maximum 100 lb/45.4 kg weight limit.

