

Installation Guide

12"/30.5cm 10x25mm Rail for 2"/5.1cm Diameter Post
(Refer to qualified personnel)


The purpose of this guide is to:

1. Describe Rail position adjustment
2. Describe attachment of Rail Assembly to post.
3. Reference Examples of Mounting Accessories

Parts Reference

The following parts and hardware are included in this installation kit:

Item #	Description	Qty
1	Rail Assembly	1
2	M4 Hex Wrench (not shown)	1
3	M5 Hex Wrench (not shown)	1




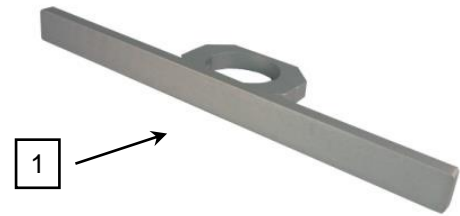


Tools required

- M4 hex wrench (provided)
- M5 hex wrench (provided)



Rail Position Adjustment and Attachment to Roll Stand post

- A. Using M5 Hex Wrench provided, unscrew two (2) M6 x 20mm SHCS to remove rear clamp (do not discard screws).
- B. Using M4 Hex Wrench provided, unscrew two (2) M6 x 12mm BHSCS to separate rail from clamp. Adjust rail position and re-attach to clamp using the same two (2) M6 x 12mm BHCS.
- C. Place assembly at desired position on post and re-install rear clamp using two (2) M6 x 20mm SHCS.

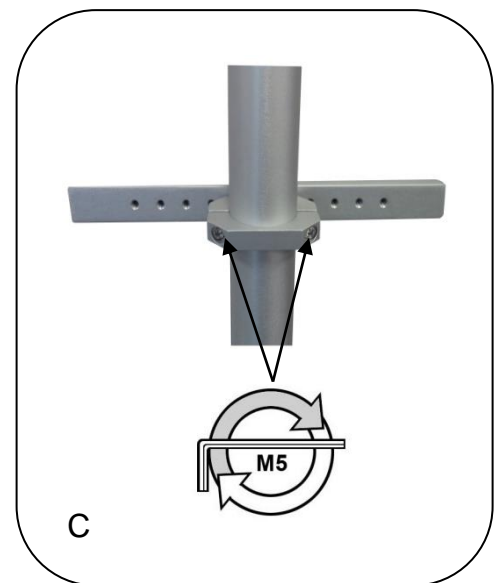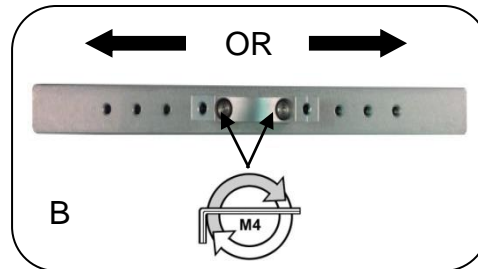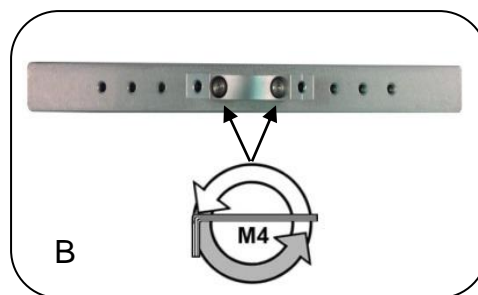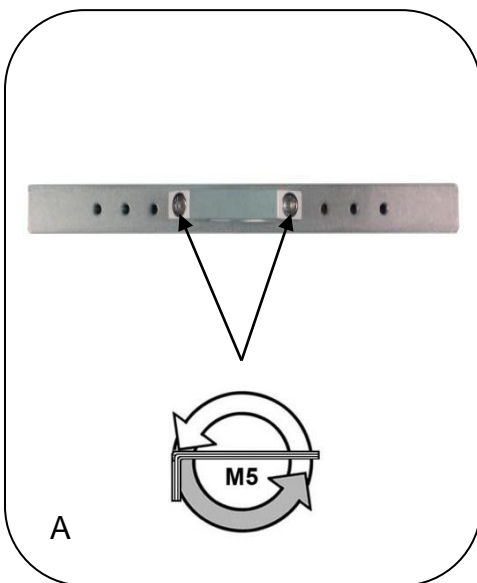

Examples of Rail Mounted Accessories

Accessories mounted on this assembly must not exceed 20lbs/9.1kg.

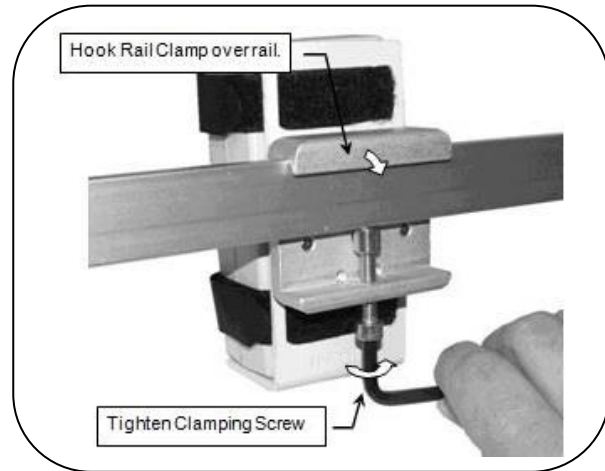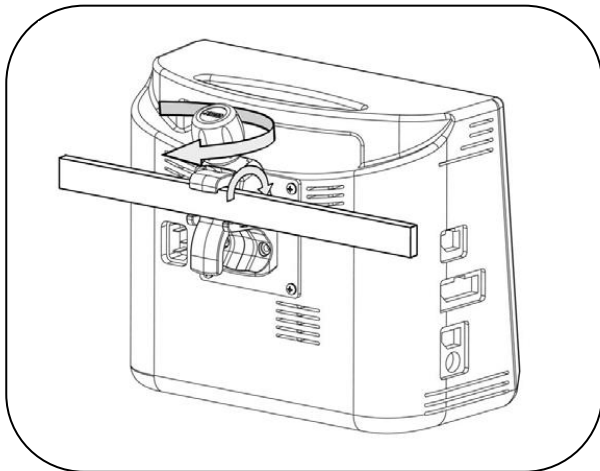

Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately, using water dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products before installing the arm in the Channel.