

Installation Guide

Horizontal Mount Adapter for Philips Table Top Mount

The purpose of this guide is to describe attachment of Philips Table Top Mount to Adapter and mounting of Horizontal Mount Adapter in Channel.

Parts Reference

The list below includes parts (see photo) and hardware (not shown) that will be used in this installation procedure.

Item #	Description	Qty
1	Horizontal Mount Adapter	1
2	M6 x 10mm Flat Head Machine Screw (FHMS)	3


Horizontal Mount Adapter



Tools Required: Phillips screwdriver (not provided).

Attaching *Philips Table Top Mount to Horizontal Mount Adapter



1. Fasten the Philips-supplied Table Top Mount to the Mounting Adapter with three (3) M6 x 10mm.

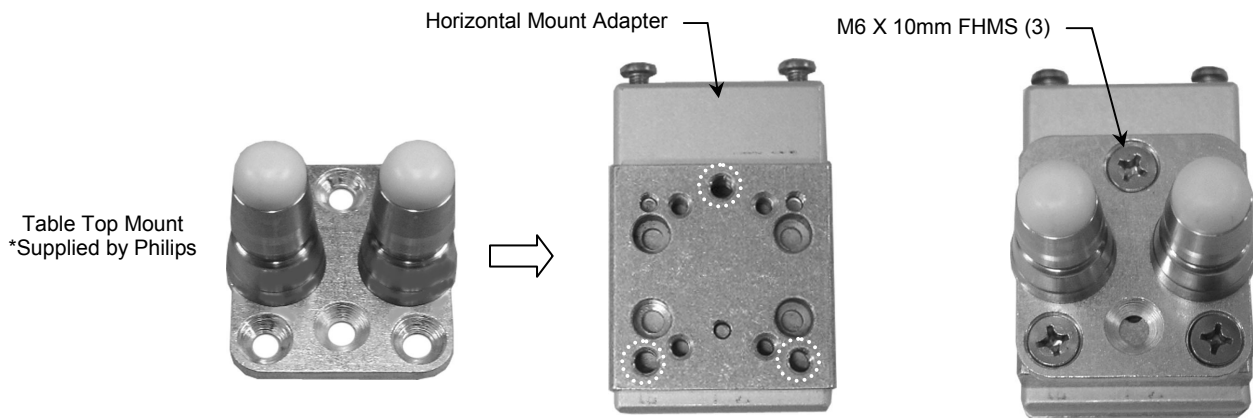

Mounting Horizontal Mount in Channel

Tools Required: Phillips screwdriver.

1. With two (2) set screws facing oriented toward the rear of the application, slide Horizontal Mount into Channel and move Mount to desired instrument-mounting position (example below left).
2. Tighten two (2) Phillips-head set screws at rear of Horizontal Mount to prevent movement of Mount (below right).

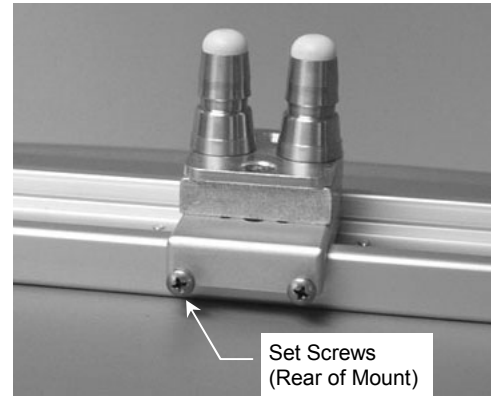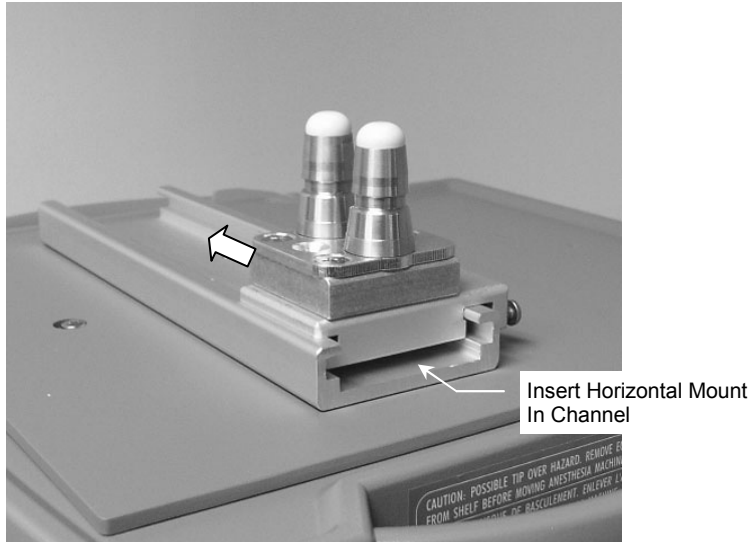

Mounting Instruments on the Table Top Mount

Mount instruments in accordance with Philips mounting instructions.

