

Installation Guide

Roll Stand Post Mount for 75mm VESA Data Capture Devices on Welch Allyn 7000-MS3 Roll Stand

(Refer to qualified personnel)

Purpose: To describe installation of Post Mounting Assembly to Welch Allyn Roll Stand



The purpose of this manual is to describe general installation of Post Mounting Assembly to Welch Allyn Roll Stand. This manual should be used in conjunction with any instrument-specific installation guides. Please read this manual and all instrument-specific installation material before installing or using this product.



Ensure that the weight of the instrument being mounted does not exceed the load rating limit of 10lb. Failure to do so may result in damage to equipment or injury.


Parts Reference







The following parts and hardware are included in this installation kit:
(Hardware not shown)

Item #	Description	Qty.
1	Post Mounting Assembly	1
2	Clamping Block	1
3	Cable Clip	3
4	10-32 X 5/8 Socket Head Cap Screw	2
5	M4 X 8mm Pan Head Machine Screws	4

Tools Required:

- 3/16" Hex Wrench (Provided)
- 5/32 Hex Wrench (Provided)
- Phillips Screwdriver (Not Provided)
- 2X 1/2"/13mm Wrench (Not Provided)

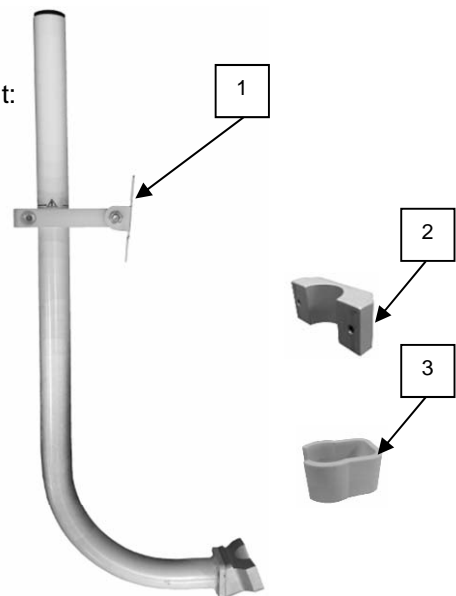

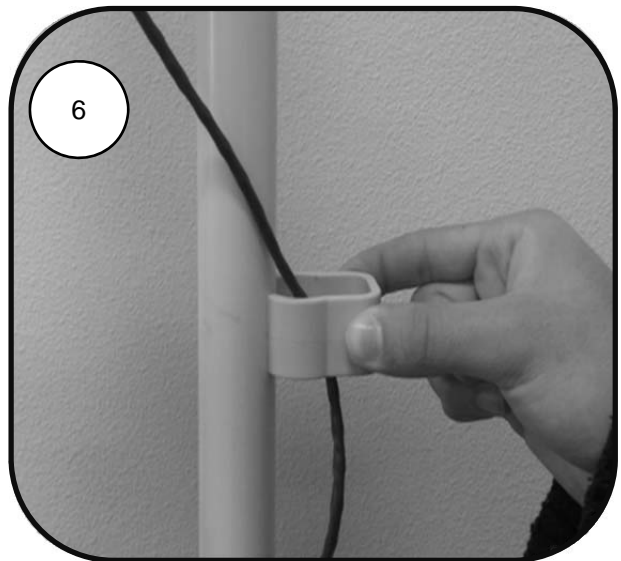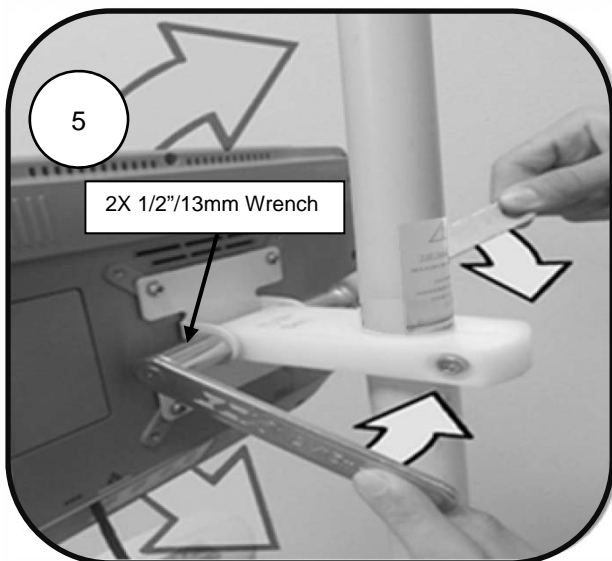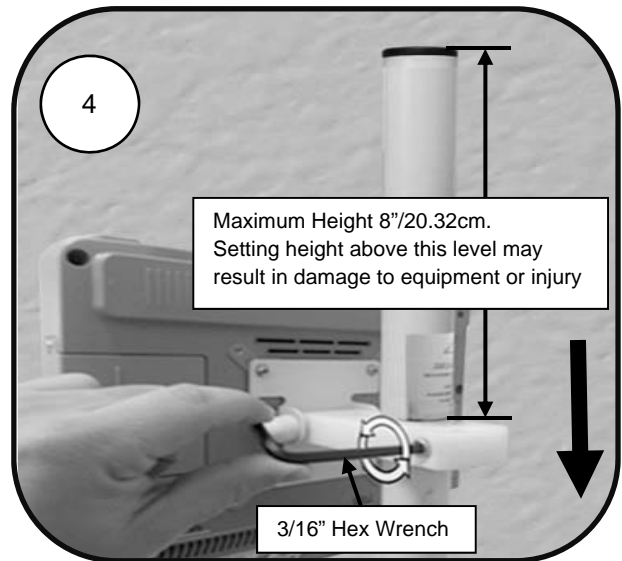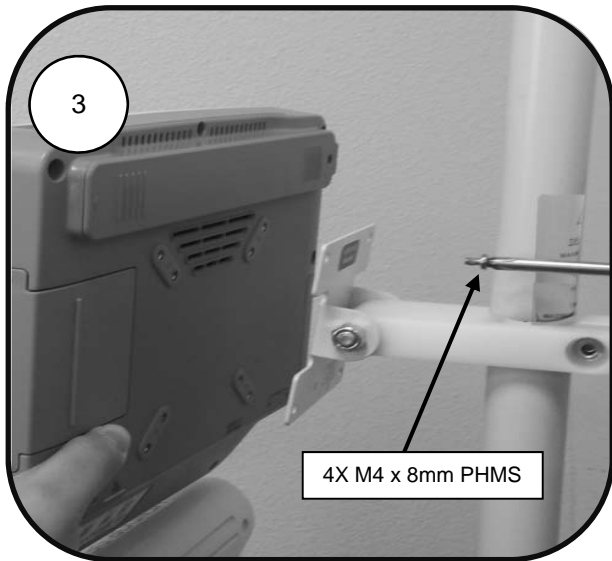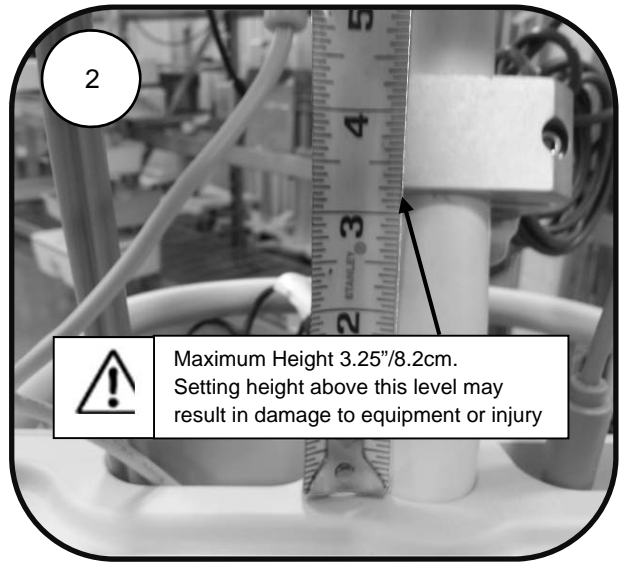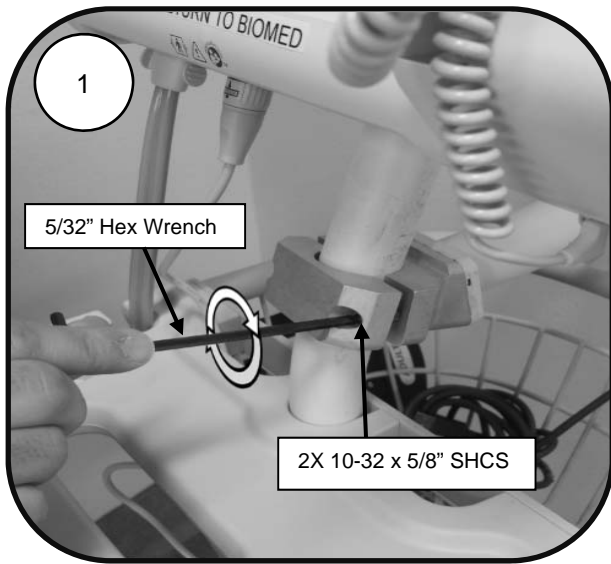



Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.

Steel wool or other abrasive material should never be used.

Damaged caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.

Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.