

Installation Guide


Mounting the M-Series Arm on a 38mm Post

The purpose of this guide is to describe attachment of the Arm to the Post.

Tools Required: 3/16" hex wrench (provided).

Mounting M-Series Arm on 38mm Post

1. Using the 3/16" hex wrench provided, mount Arm on post by fastening Clamping Block to Arm with four (4) 1/4-20 x 3/4" Socket Head Cap Screw (SHCS) as shown below. Torque fasteners to 35-45 in-lbs (4.0 – 5.1 N-m).

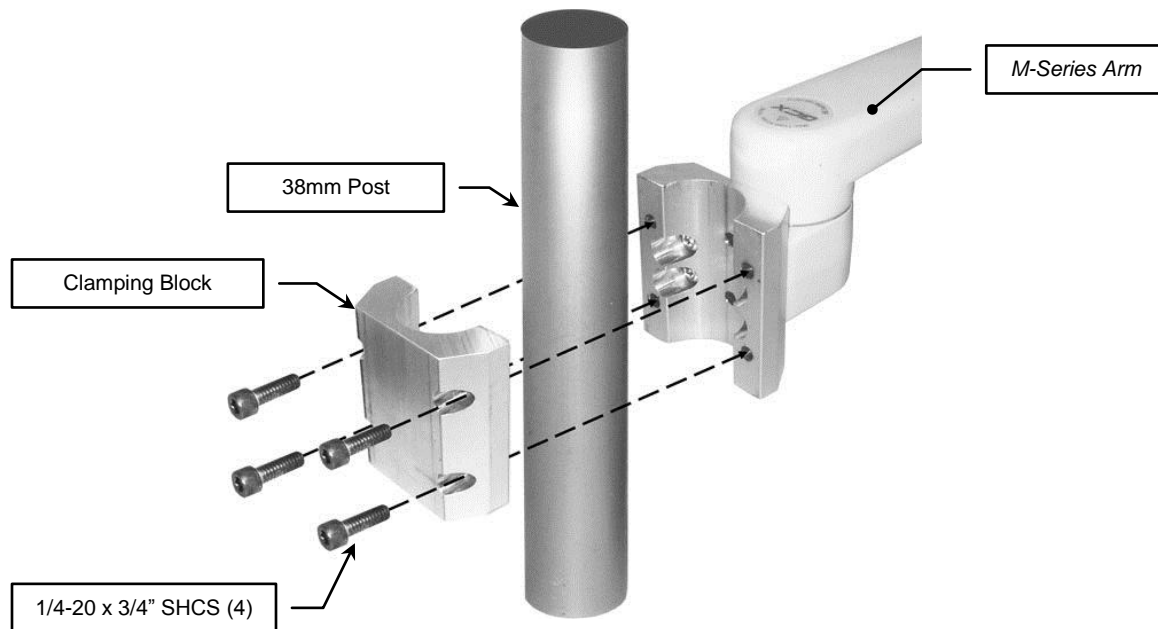

M-Series Arm Operation and Adjustment

Please refer to installation guide DU-WMM-0001 (included with Arm) for further instructions on the M-Series Arm.