


## Installation Guide

### Dräger Fabius GS Top Plate with 21" Channel


*The purpose of this guide is to describe attachment of Top Plate/Channel to top of machine.*

**Warning:** Do not mount equipment such that its center of gravity is outside the perimeter of the Top Plate.

1. Locate the four (4) dimples in the top of the anesthesia machine. Dimples indicate locations of threaded inserts beneath the top surface. If dimples are difficult to locate, use the Top Plate/Channel assembly as a template and mark the top of the machine through the four (4) mounting holes.
2. Using a 1/4" drill bit, drill in the locations marked in step 1 (drill only where dimples are present). **CAUTION:** Do not allow drill bit to penetrate more than 1/4" into top of machine. Drilling too deeply may damage threaded inserts and internal components of the machine. Remove all drilling chips.

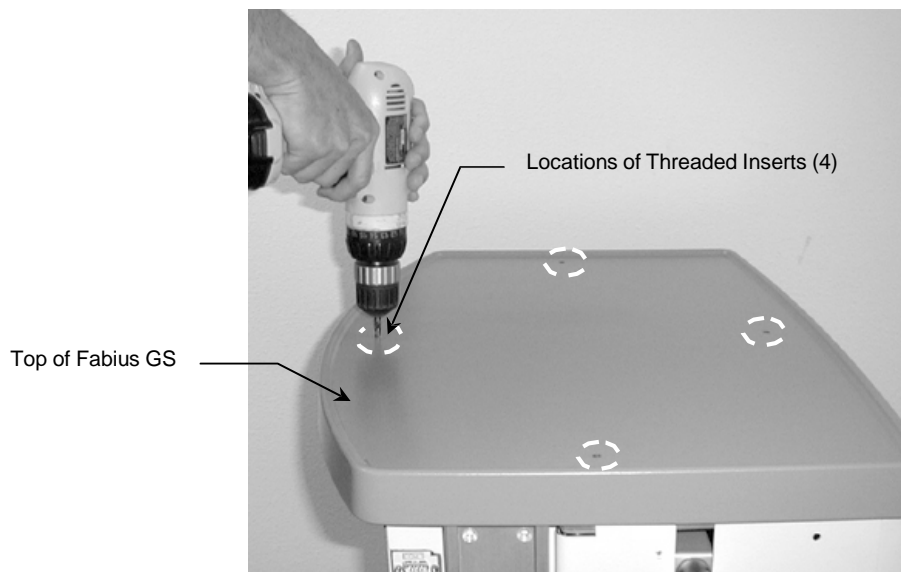

3. Align the four (4) mounting holes in the Top Plate/Channel with the corresponding holes in the top of the machine (drilled in step 2). Fasten Top Plate to machine with four (4) M5 x 30mm Pan Head Machine Screws (PHMS)

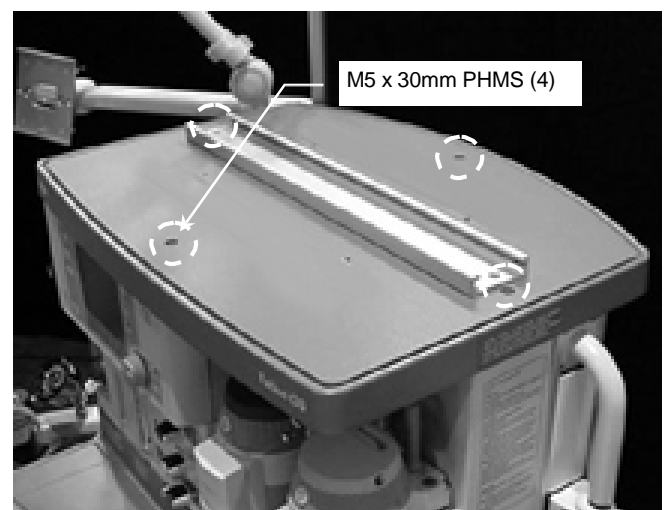

4. Attach the Tip Warning Label (*Figure 1*) to the front-center of the blue top shelf bezel.

*Figure 1*

