

GCX Mounting Assembly Installation Guide


Dräger Fabius GS Hewlett-Packard Camlock Top Plate

The purpose of this guide is to:



1. Describe attachment of Fabius GS Camlock Top Plate.

Attaching the Top Plate

1. Find the threaded inserts by locating the four (4) dimples in the top of the anesthesia machine.
2. Drill holes precisely in the center of the dimples using a 1/4" diameter drill bit. **CAUTION:** Do not allow the drill bit to penetrate more than 1/4" into the top surface. Drilling too deeply will damage insert threads and internal components of the machine. Remove all drilling chips.

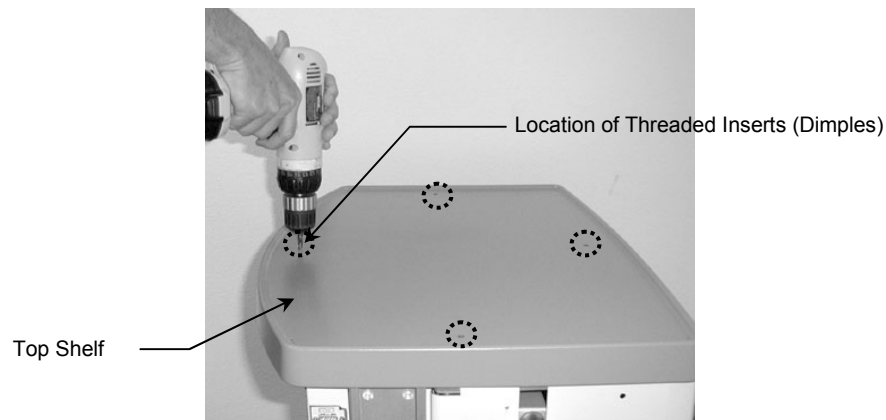


3. Thread the four (4) M5 x 30mm Pan Head Machine Screws through the Top Plate and into the holes drilled in step #2.

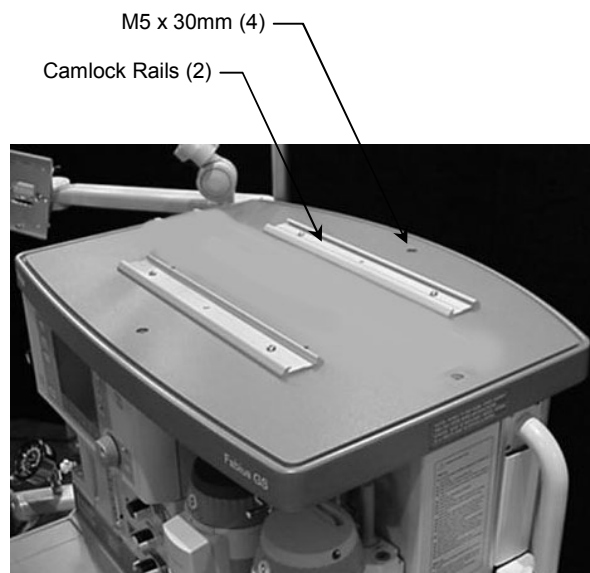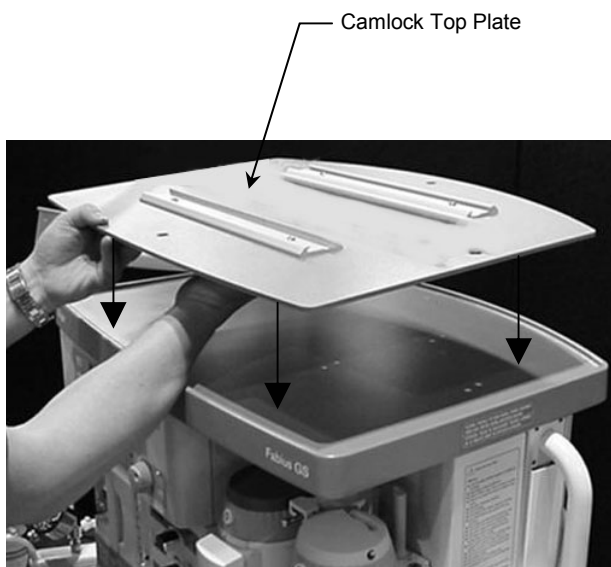

4. Attach the Tip Caution Label (*Figure 1*) to the front center of the blue top shelf bezel (see photo).

Figure 1

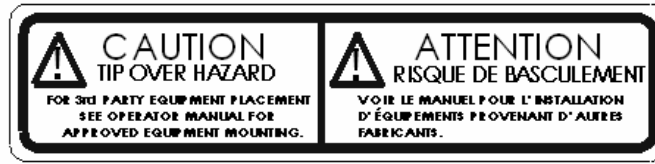

CAUTION Label Location

