

Installation Guide

6"/15.2cm Fixed Extension with 7.25"/18.4cm Channel for Dräger Accessory Track

The purpose of this guide is to describe installation and use of the Fixed Extension.



REFER TO THIS DOCUMENT FOR QUESTIONS ABOUT THIS PRODUCT.



WARNING: REFER TO QUALIFIED PERSONNEL FOR INSTALLATION.

WARNING: NOT FOR USE WITH GCX VHM SUPPORT ARMS WITH EXTENSION OPTION.

WARNING: USE OF MOUNTING HARDWARE AND MONITORING COMPONENTS OTHER THAN THOSE TESTED FOR TIP SAFETY MAY RESULT IN SERIOUS INJURY DUE TO TIPPING OF THE ANESTHESIA MACHINE.

WARNING: ALL DEVICES AND MOUNTING EQUIPMENT SHOULD BE POSITIONED INWARD, CLOSE TO THE ANESTHESIA MACHINE DURING TRANSPORT.



MAXIMUM LOAD: 60 lbs/27.2 kg

PRODUCT WEIGHT: 2.2 lbs/1 kg

Installer: When installation is completed, provide these instructions and all related documentation to the end user for future reference.

Parts Reference

The parts list below includes parts and hardware that will be used in this installation procedure.

Item #	Description	Qty
1	6"/15.2cm Extension Assembly with Top Cap	1
2	Nut Plate	2
3	M6 x 25mm Socket Head Cap Screw (SHCS)	2
4	VHM-P/PL (without Extension) Rear Cover	1
5	M5 Hex Key (not shown)	1


2



3



4

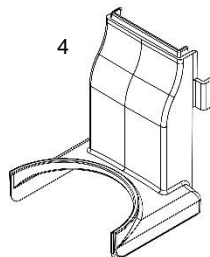


1





Tools Required: M5 hex key (provided).

Installation into the Accessory Track

1. Insert both M6 x 25mm SHCS through the top and bottom of the assembly and thread a Nut Plate onto the screws a few turns - leave approximately 20mm of thread exposed per screw.

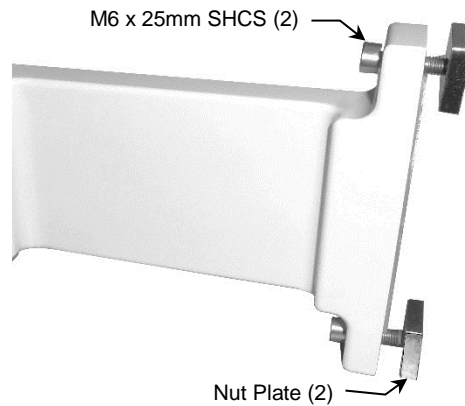


3. Guide Nut Plates into the accessory track. Move entire assembly up the track to the desired mounting position.
4. To secure, use the M5 hex wrench provided - tighten bottom screw first, then top screw.

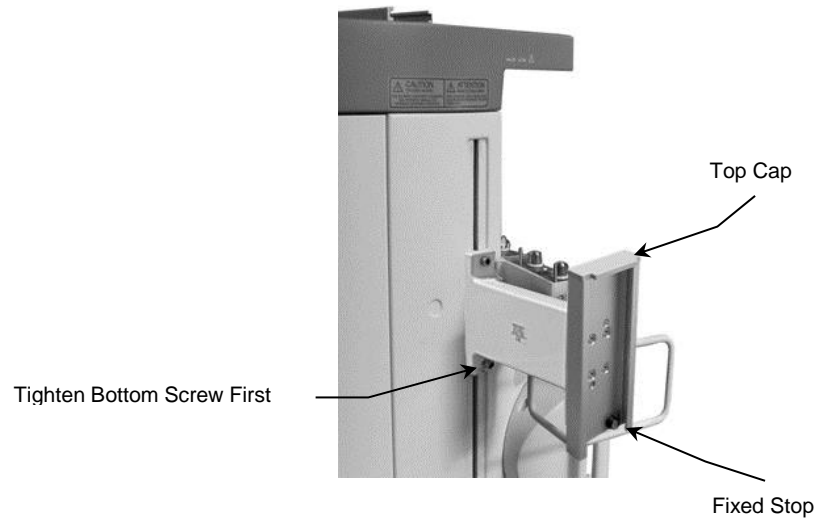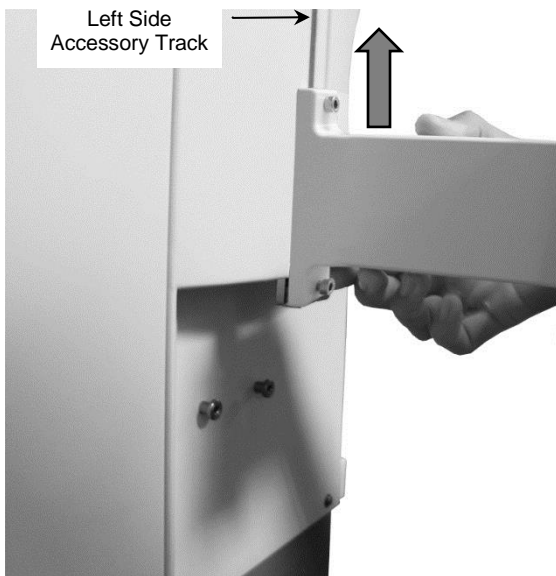

VHM-P/PL Arm without Extension:

- Mount arm in accordance with installation guide included with the arm except use the included Rear Cover in place of the one included with the Arm.
- Channel Top Cap is a press-fit component and is easily removed and installed.

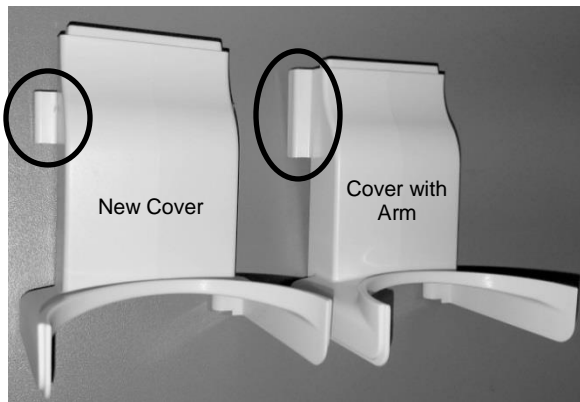

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened or adjusted as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the arm that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the arm. Wipe any cleaning agents off the arm immediately using a water-dampened cloth. Dry mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.