

## Installation Guide

### Dräger Fabius Plus Top Shelf Horizontal Channel Kit

The purpose of this guide is to:

- Describe installation of the Top Shelf Horizontal Channel onto Dräger Fabius Plus machine.




**Warning:** Carefully tighten screws by hand only. Do not use power tools.

#### Parts Reference

The following parts are included in this installation kit :

Item #	Description	Qty
1	Top Shelf Horizontal Channel	1
2	M5 x 8mm Flat Head Machine Screw (FHMS)	4


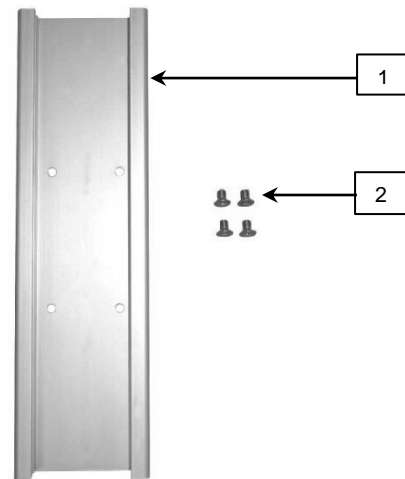


#### Tools Required:

Phillips screwdriver (not provided)


#### Removing Top Shelf From Fabius Plus

Use a Philips screwdriver and remove top two (2) Flat Head screws from right side channel. Loosen lower two (2) Flat Head screws enough so channel separates from Top Shelf. Repeat steps for left side channel. Lift Top shelf from machine using upward motion.

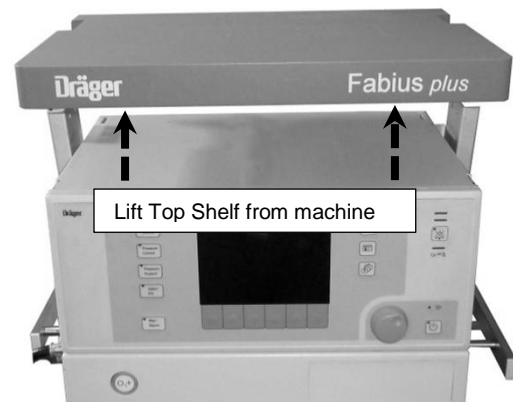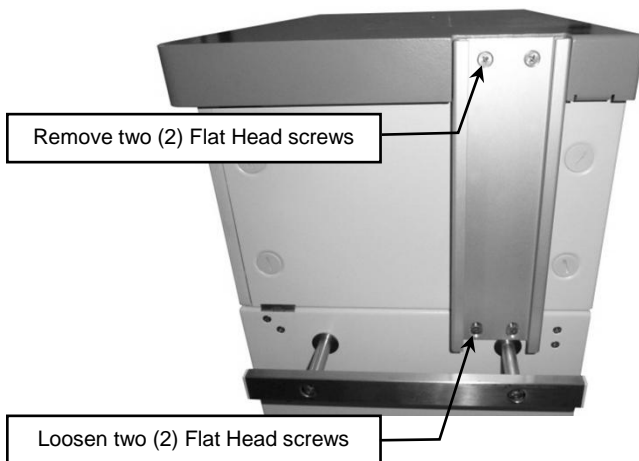

### Attaching Horizontal Channel To Top Shelf

Place channel with flat side facing upwards on a flat surface. Orient Top Shelf so that flat side in on the bottom and align mounting holes on Top Shelf with mounting holes on channel. Insert and fasten with four (4) M5 x 8mm FHMS.

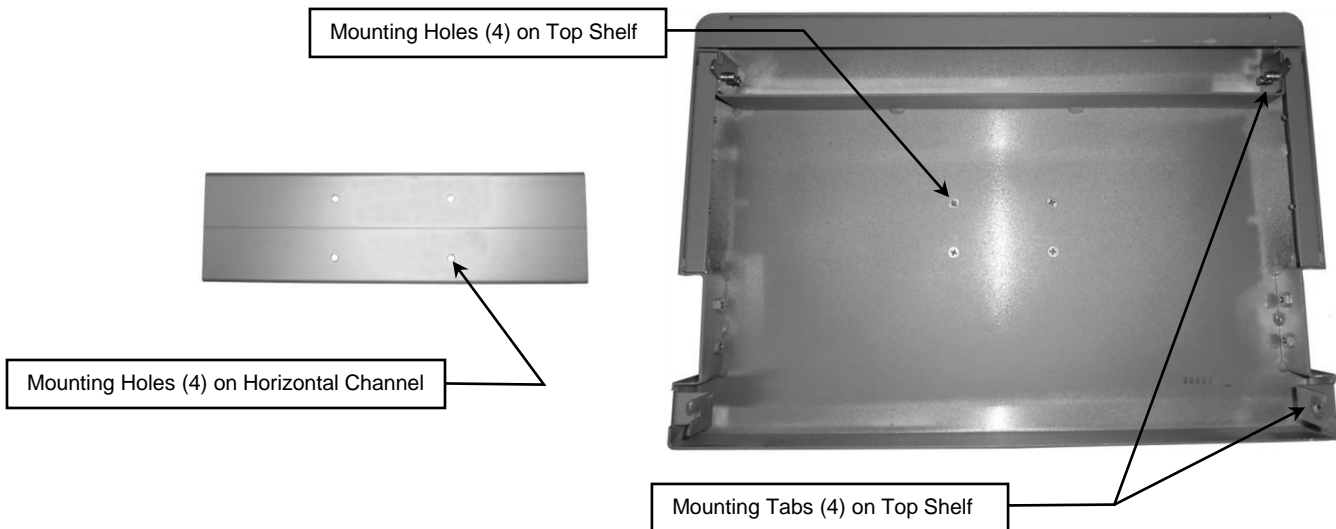

### Re-attaching Top Shelf To Fabius Plus

Orient Top Shelf with mounted channel so that the channel is on the top. Place Top Shelf over top of machine making sure to align mounting tabs with mounting slots. Press down on Top Shelf so tabs insert into slots. Starting with the right side channel, insert and fasten with two (2) Flat Head screws that were removed in step 1 into the upper mounting holes. Tighten the lower two (2) Flat Head screws on the right side channel as shown below. Repeat steps for the left side channel.

