

GCX Mounting Assembly Installation Guide


Datex-Ohmeda Aespire Core Pivot Arm – 6" and 12" Down Posts

The purpose of this guide is to describe attachment of the Down Post to the Aespire Core Pivot Arm.

Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts, hardware not shown):

Item #	Description	Qty
1	6" Down Post	1
2	12" Down Post	1
3	#10-32 x 5/8" Flat Head Machine Screw (FHMS)	3

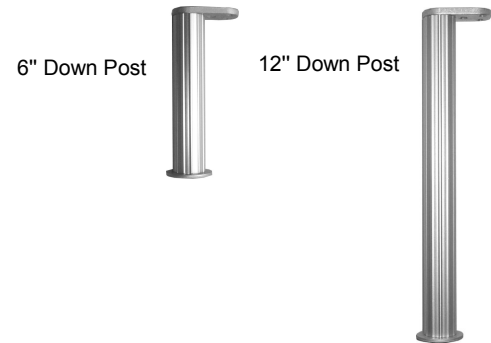


Tools Required for This Task: Phillips screwdriver.

Attaching Down Post (6" or 12") to Aespire Core Pivot Arm

1. Identify desired 3-hole mounting pattern on bottom of the Core Pivot Arm.



Bottom of Aespire Core Pivot Arm

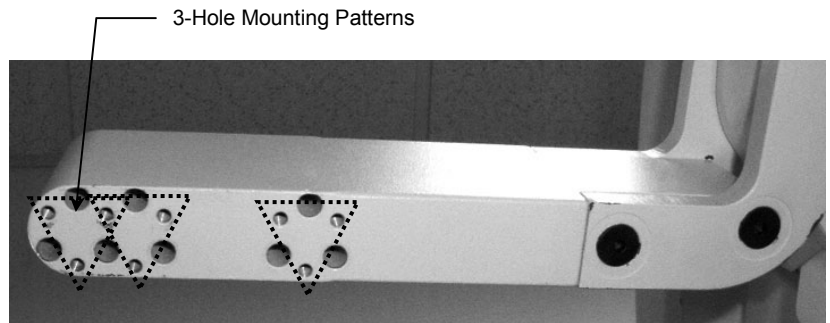

2. Fasten Down Post to Arm with three (3) #10-32 x 5/8" FHMS.

