

Installation Guide


Top Plate with Channel for Datex Avance CS2

The purpose of this guide is to describe installation of the Top Plate with Channel to the Datex Avance CS2.

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):


Item #	Description	Qty
1	Top Plate w/ 10" Channel	1
2	M6 x 16 mm Flat Head Machine Screws (FHMS)	4

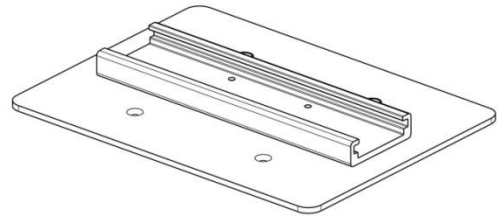

Tools Required: Phillips screwdriver (not provided).

Installing the Top Plate


- Using a sharp instrument, remove the four (4) thread covers of the four-hole mounting pattern to reveal threaded inserts.

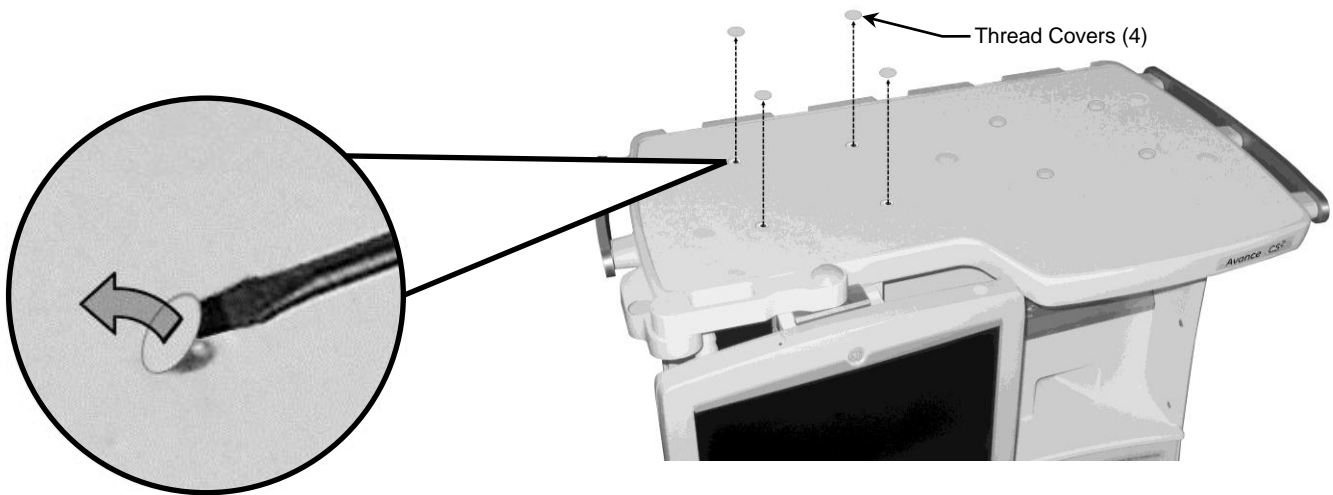


Installation Note: Top Plate may be mounted in two different locations: the left four-hole mounting pattern and the right four-hole mounting pattern.



2. Fasten Top Plate to machine with four (4) M6 x 16mm FHMS using a Phillips screwdriver.

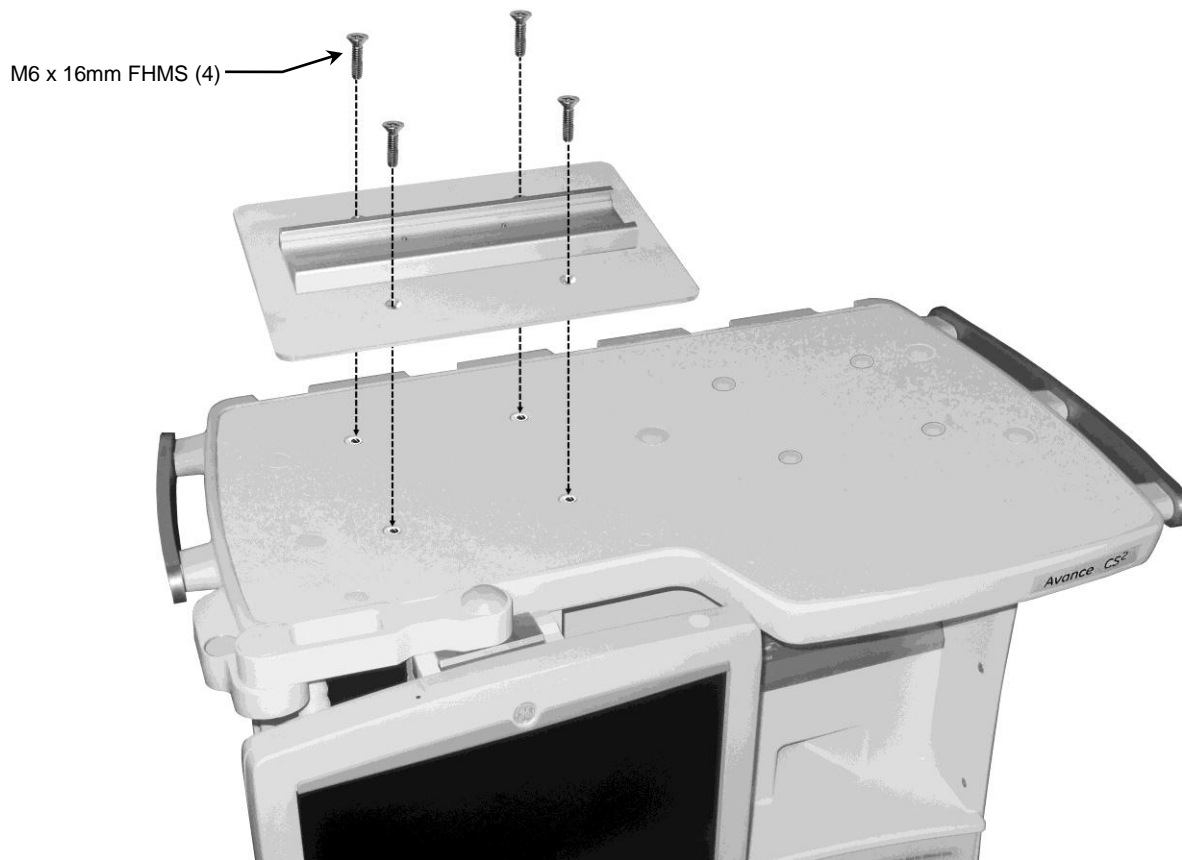

Installation Note: Two (2) Top Plates may be mounted side by side using both the left and right four-hole mounting patterns.

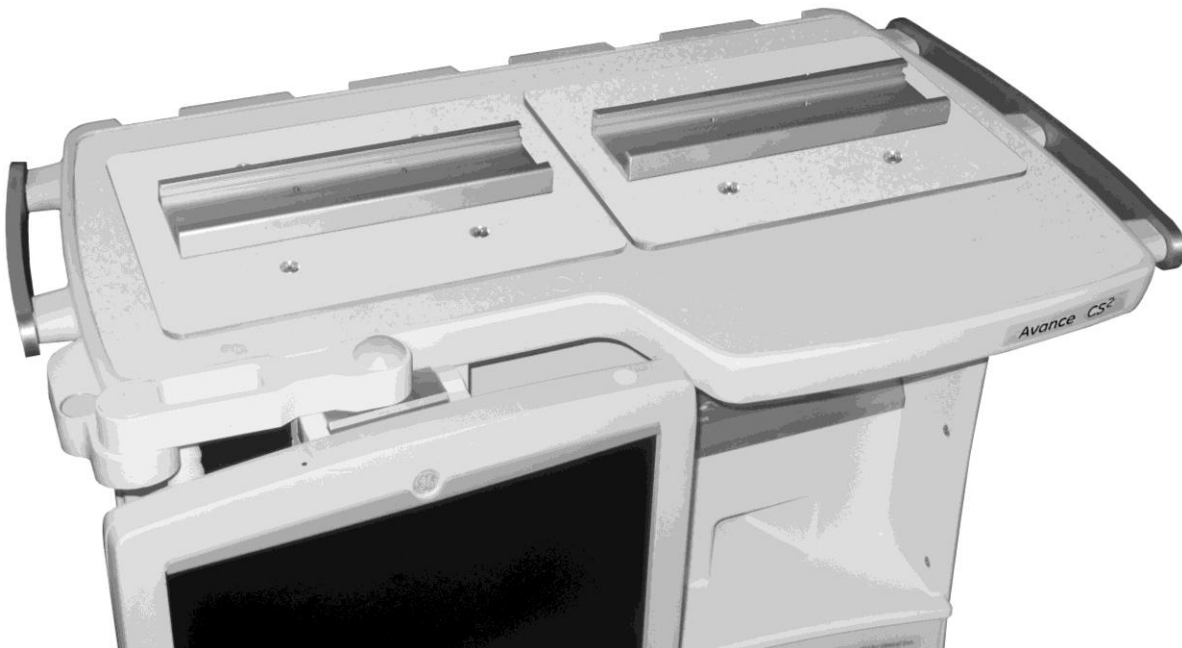

Routine Maintenance

Periodically check all mounting hardware. Tighten as necessary for optimal operation.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assemblies. Wipe any cleaning agents off of the mounting assemblies immediately, using a water-dampened cloth. Dry parts thoroughly after cleaning.