

Installation Guide

GE Healthcare Carescape Channel-Mounted Cable Management Arm


The purpose of this guide is to:

1. Describe attaching Cable Management Bracket to Arm (page 1).
2. Describe mounting of Cable Management Arm (page 1).
3. Describe routing of cables (page 2).

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):


Item #	Description	Qty
1	Cable Management Arm	1
2	Cable Management Bracket	1
3	10-32 x 1/2" Pan Head Machine Screw (PHMS)	3
4	1/8" Hex Wrench	1

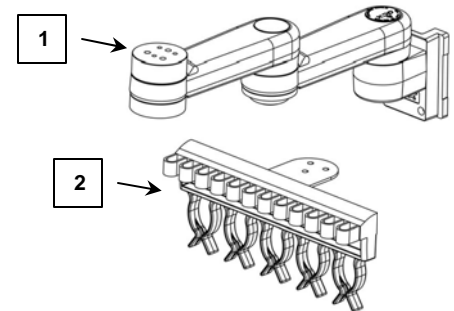


Tools required

- 1/8" hex wrench (provided)
- Phillips Screwdriver (not provided)


Attaching Cable Management Bracket to Arm

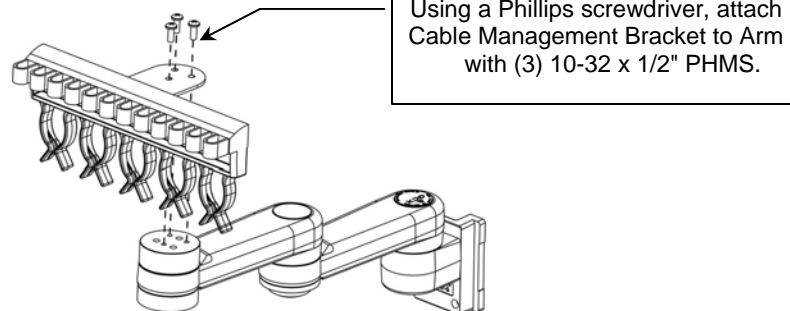

Mounting the Cable Management Arm

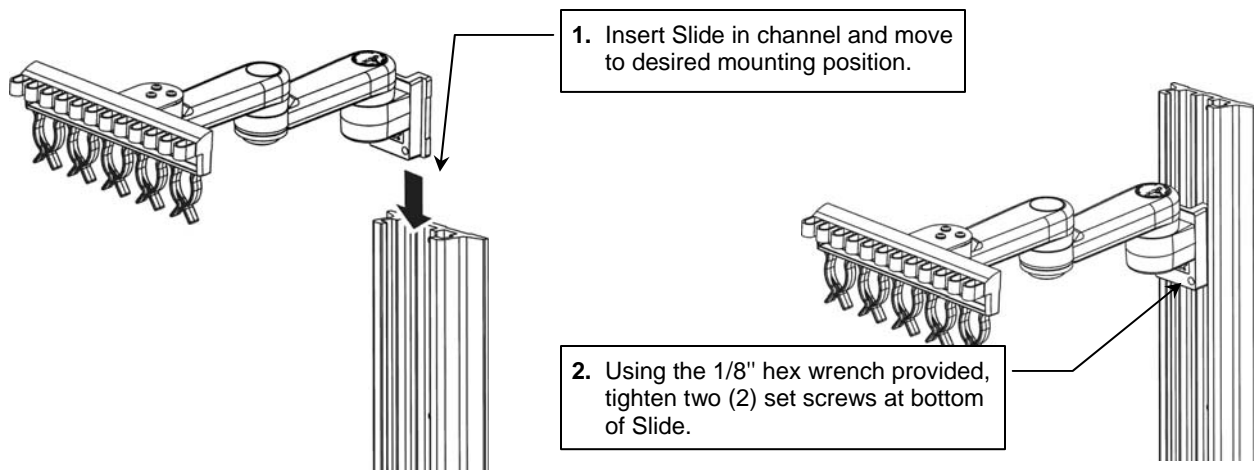


Routing Cables

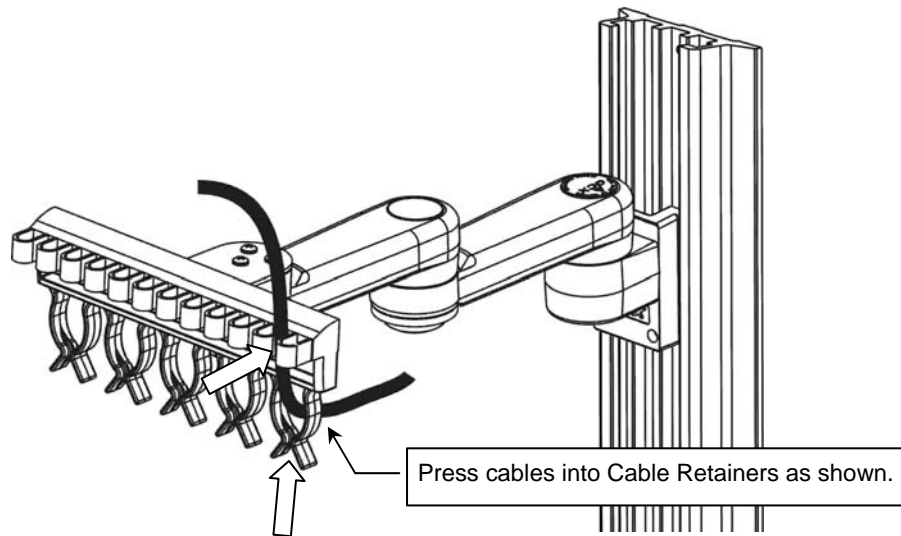

M-Series Articulating Arm Adjustments Swivel/Pivot Tension

To adjust swivel or pivot tension, loosen or tighten the swivel tension nut (front end) or pivot tension bolts (center and rear of arm) with a 1/2" socket head wrench.

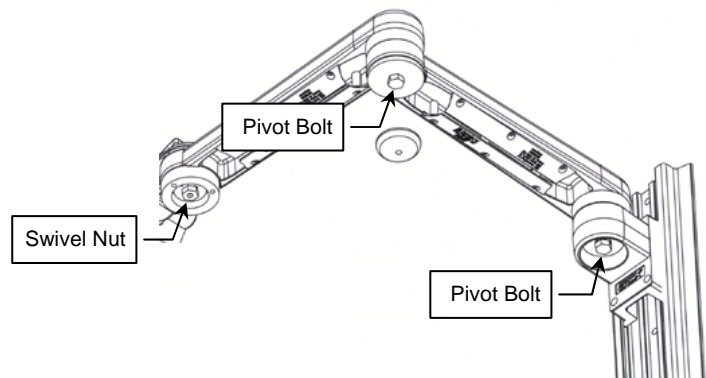

Routine Maintenance of the Mounting Assembly

Periodically inspect all mounting hardware. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately using a water-dampened cloth. Dry mounting assembly thoroughly after cleaning.