

Installation Guide


Horizontal Mount (0° or 10° Fixed-Tilt Version + Low Profile Tilt/Swivel)

The purpose of this guide is to describe installation of Horizontal Mount.

Tools Required: Phillips screwdriver.

Installing the Horizontal Mount

1. Orient plunger as shown below and slide Horizontal Mount to desired instrument-mounting position in Channel.
2. Using a Phillips screwdriver, tighten two (2) screws at rear of Horizontal Mount (*Rear View* below).
3. Mount instrument in accordance with separate instructions.

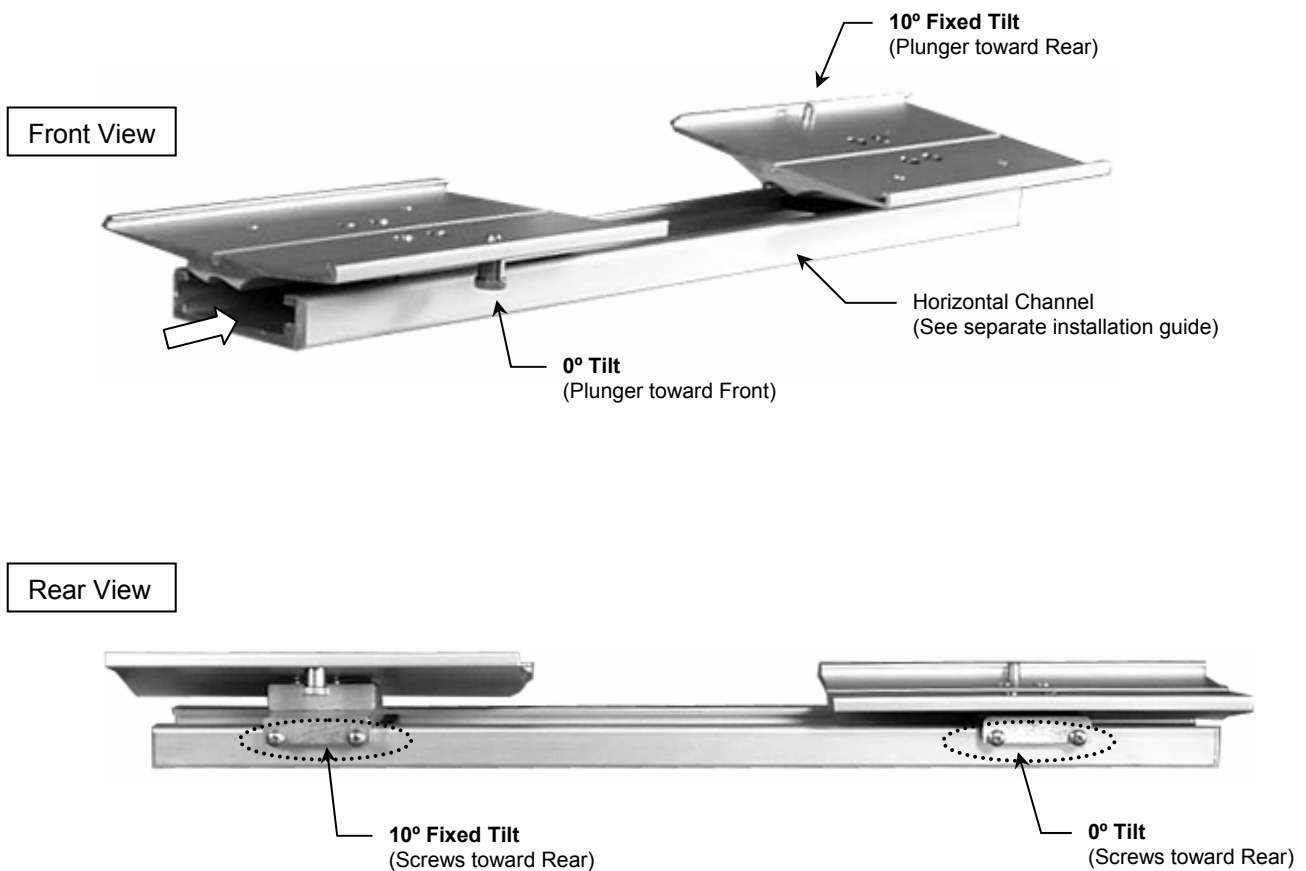

Low Profile Tilt/Swivel Version - Tilt and Swivel Tension Adjustments

Tilt tension is adjusted by tightening or loosening the socket set screw on the top. A 5/32" hex wrench is provided for this adjustment.

Swivel tension is adjusted by tightening or loosening screw on side of post immediately below Mounting Plate.

