

Installation Guide


HCI DVD Player Hanger for M-Series Arm

The purpose of this guide is to describe assembly and installation of the Hanger.

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):


Item #	Description	Qty
1	Adapter Plate	1
2	Down Post, 9"	1
3	Down Post, 12"	1
4	Down Post Disk	1
5	#10-32 x 3/4" Flat Head Machine Screw (FHMS)	6
6	#10-32 x 3/8" Socket Head Cap Screw (SHCS)	3
7	M4 x 8 mm Button Head Socket Cap Screw (BHSCS), Tamper-proof	4
8	M4 Split Lock Washer	4
9	5/32" Hex Wrench, Ball-end	1
10	5/32" Tamper-proof Hex Wrench	1

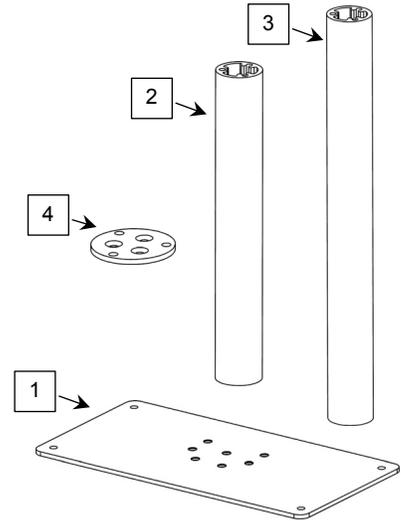


Tools Required

- Phillips screwdriver (not provided),
- 5/32" ball-end hex wrench (provided)
- 5/32" tamper-proof hex wrench (provided)

Installing the Hanger on M-Series Arm

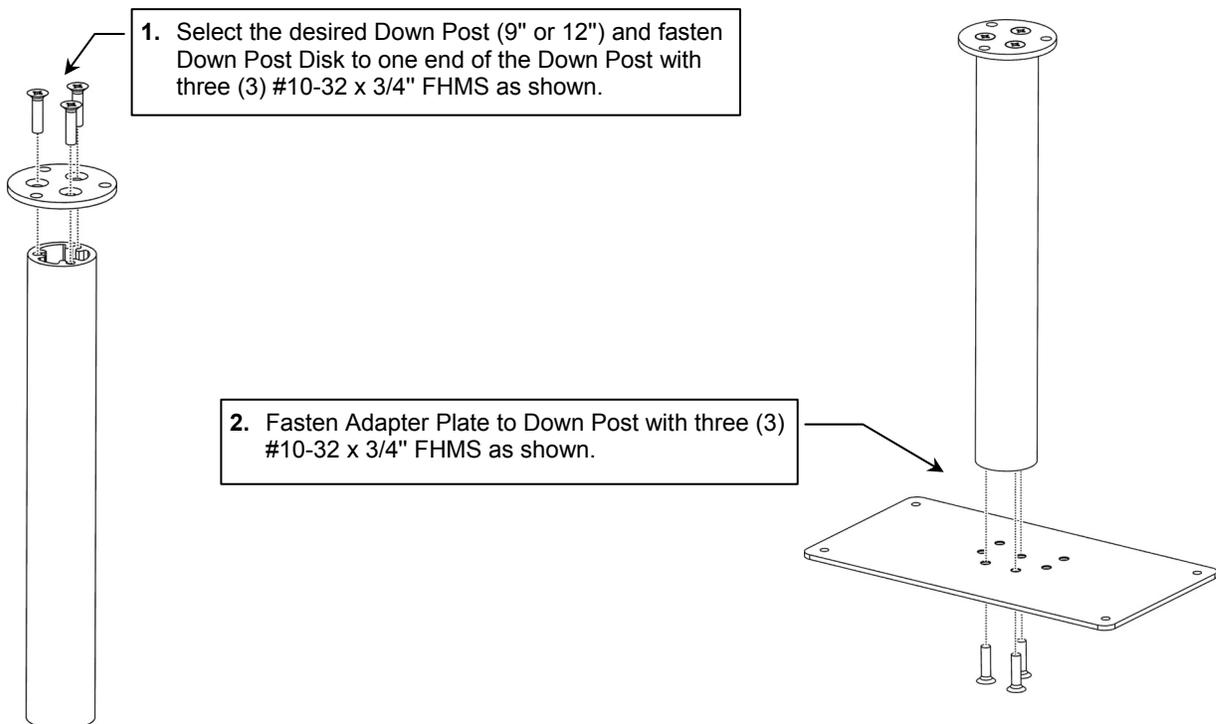

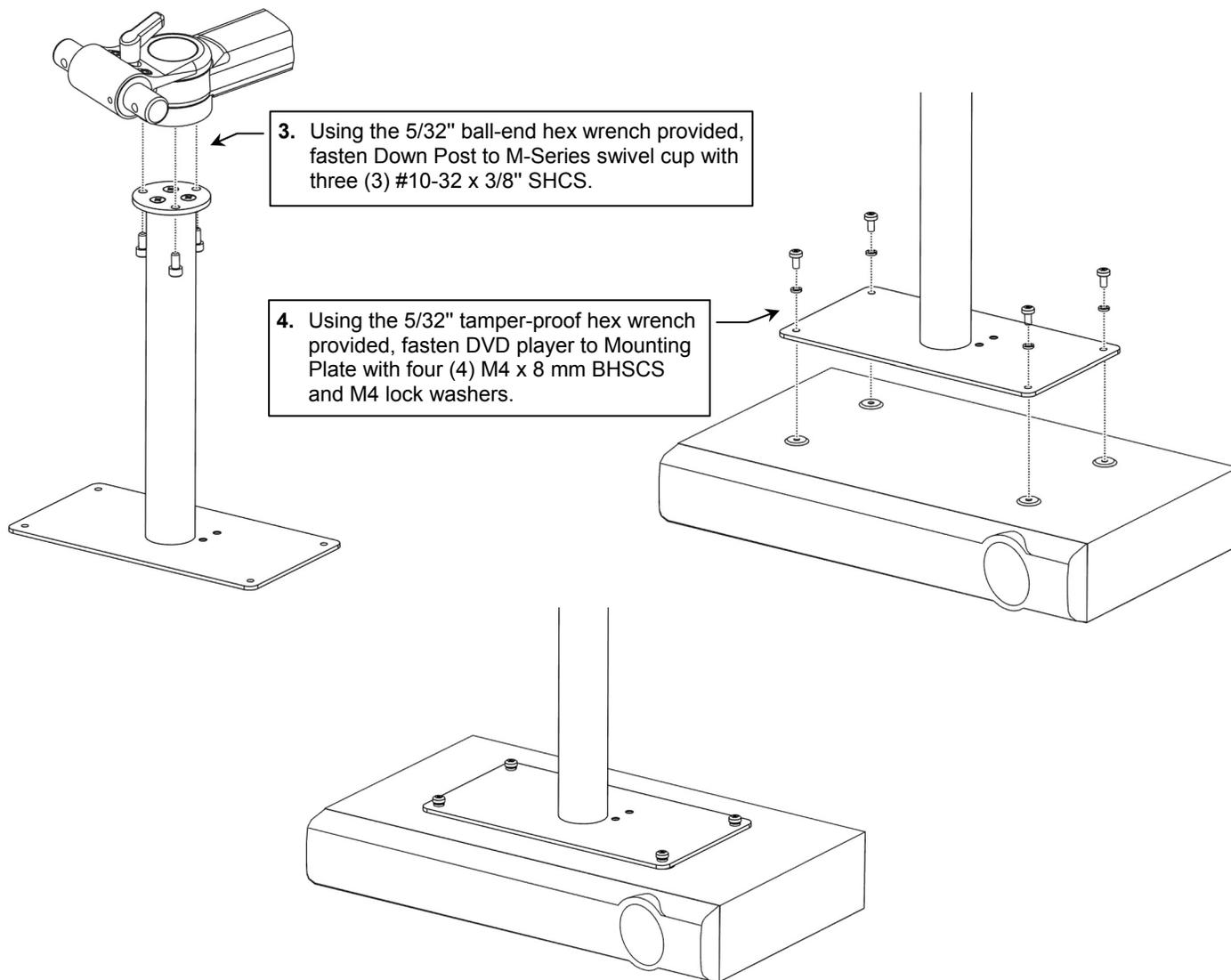

Routine Maintenance

Periodically inspect all mounting hardware. Tighten as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry all mounting assemblies thoroughly after cleaning.